Pantone 326C
 Pantone 1945C
 Black

18.5" W

5" H

NO PRINT AREA

SHINE reNEW®

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IMS-1453 | 1 US GALLON (3.8L)

DIRECTIONS FOR USE

Stain Removal Soak: Wear utility gloves while using Shine reNEW. To prevent potential staining of surfaces, place towel under the container where the instruments will soak. Dilute 1 part Shine reNEW with 7 parts warm water in a container. Open instruments and soak for 5-15 minutes. Heavily stained instruments may require 30 minutes of soaking. Heavy rust and stain may require light brushing with a soft brush during soaking. When cleaning tungsten carbide, soak no longer than 10 minutes. Remove instruments from soak, rinse thoroughly with water, and re-clean instruments with an enzymatic cleaner in an ultrasonic unit or clean in a dental instrument washer. Follow proper sterilization protocol. Discard used soak solution after daily use or change more often as needed. This product is designed to be used on stainless steel and tungsten carbide, not anodized aluminum. This material and its container must be disposed of in a safe way. Repeated soaking with Shine reNEW may improve the ability to remove heavy rust or stain. This product may not remove all rust and stain from severely stained or damaged instruments.

DANGER
 May be corrosive to metals. Causes severe skin burns and eye damage. Causes severe damage to the respiratory tract.

Keep only in original container. Do not breathe dusts or mists. Wash hands, forearms and face thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. **IF SWALLOWED:** rinse mouth. Do NOT induce vomiting. **IF ON SKIN (or hair):** Take off immediately all contaminated clothing. Rinse skin with water/shower. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Wash contaminated clothing before reuse. Absorb spillage to prevent material damage. Store locked up. Store in a corrosion resistant container with a resistant inner liner. Dispose of contents/container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation.

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NO PRINT AREA

DRAFT

SPECS	FG CODE:		LEGEND	Material/Die Cut area	REVISIONS	DATE, INITIAL, AND STATE WHO REQUESTED REVISION	WEIMAN
	STOCK CODE:			Printable Area		Date	
	DL NUMBER:	DL009	Bleed Area	Tear off perforation of PDD	CHECKBOX DENOTES PLATE CHANGE FROM LAST REVISION	<input checked="" type="checkbox"/> COLOR <input checked="" type="checkbox"/> COLOR <input checked="" type="checkbox"/> COLOR <input checked="" type="checkbox"/> COLOR <input checked="" type="checkbox"/> COLOR <input checked="" type="checkbox"/> COLOR <input checked="" type="checkbox"/> COLOR <input checked="" type="checkbox"/> COLOR <input checked="" type="checkbox"/> COLOR <input checked="" type="checkbox"/> COLOR	PRINT %: 75% PAPER SIZE: 11" X 17"
	APERTURE:		Carton Outer Flap	Carton Fold			
	CUSTOMER:		Copy/Text Safe Area	Add'l Label #2			
	CUSTOMER FG CODE:		Printed P&A Carton Label Guide				
	CUSTOMER ART CODE:						
	MATERIAL SIZE/CTN ID:	18.5" X 5" (W X H)					

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IMS-1453 | 1 US GALLON (3.8L)

DIRECTIONS FOR USE

Stain Removal Soak: Wear utility gloves while using Shine reNEW. To prevent potential staining of surfaces, place towel under the container where the instruments will soak. Dilute 1 part Shine reNEW with 7 parts warm water in a container. Open instruments and soak for 5-15 minutes. Heavily stained instruments may require 30 minutes of soaking. Heavy rust and stain may require light brushing with a soft brush during soaking. When cleaning tungsten carbide, soak no longer than 10 minutes. Remove instruments from soak, rinse thoroughly with water, and re-clean instruments with an enzymatic cleaner in an ultrasonic unit or clean in a dental instrument washer. Follow proper sterilization protocol. Discard used soak solution after daily use or change more often as needed. This product is designed to be used on stainless steel and tungsten carbide, not anodized aluminum. This material and its container must be disposed of in a safe way.

Repeated soaking with Shine reNEW may improve the ability to remove heavy rust or stain. This product may not remove all rust and stain from severely stained or damaged instruments.

DANGER

May be corrosive to metals. Causes severe skin burns and eye damage.
Causes severe damage to the respiratory tract

Keep only in original container. Do not breathe dusts or mists. Wash hands, forearms and face thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. **IF SWALLOWED:** rinse mouth. Do NOT induce vomiting. **IF ON SKIN (or hair):** Take off immediately all contaminated clothing. Rinse skin with water/shower. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Wash contaminated clothing before reuse. Absorb spillage to prevent material damage. Store locked up. Store in a corrosion resistant container with a resistant inner liner. Dispose of contents/container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation.

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©2017 Hu-Friedy Mfg. Co., LLC. SDS available on request or at Hu-Friedy.com
Manufactured for | Fabriqué pour : Hu-Friedy Mfg. Co., LLC
©2017 Hu-Friedy Mfg. Co., LLC.