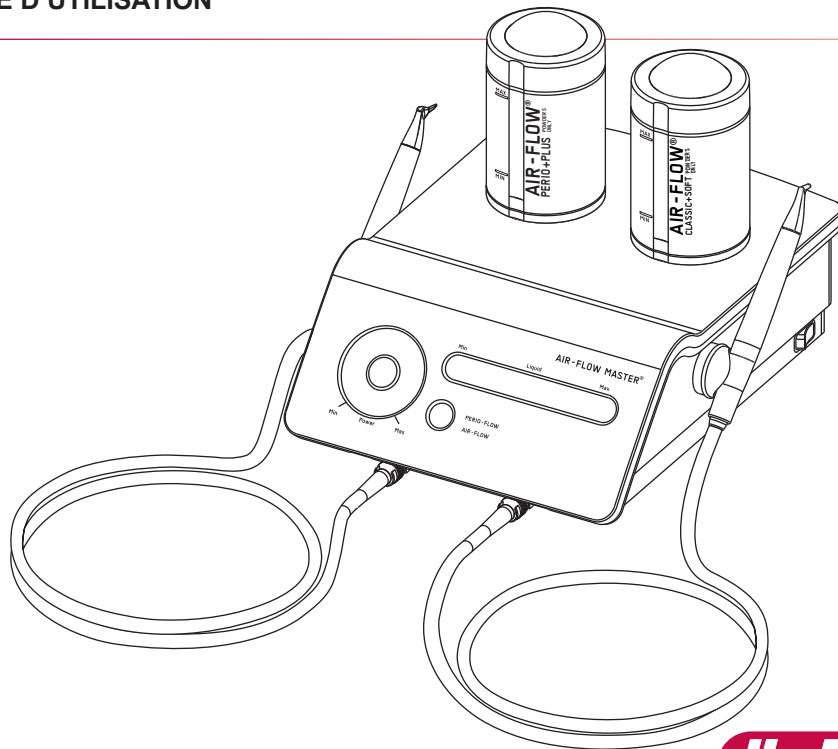
AIR-FLOW MASTER®

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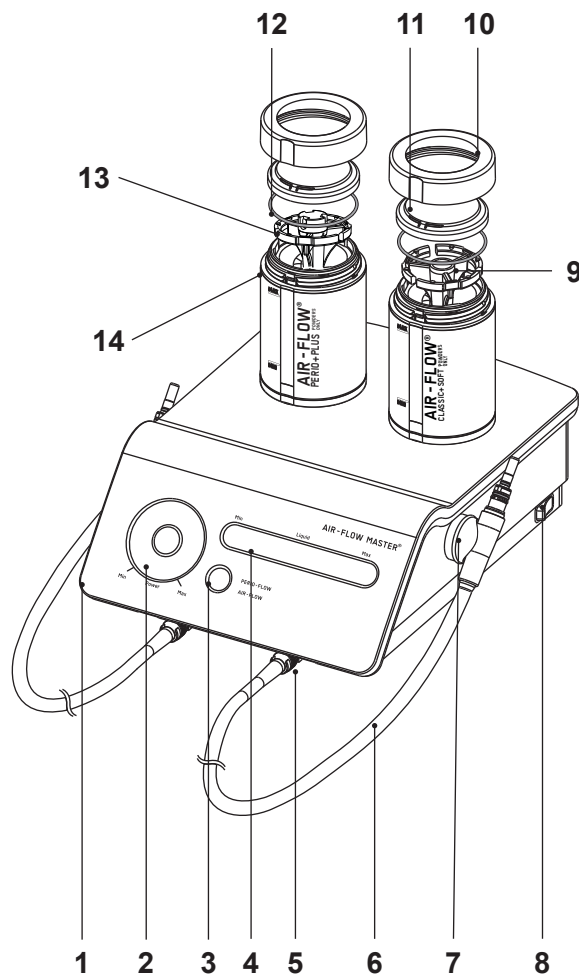
Rx ONLY Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

Hu-Friedy

EMS
ELECTRO MEDICAL SYSTEMS



COMPONENTS

- 1 Control panel
- 2 Power setting (air/powder)
- 3 Working mode selection
- 4 Water flow rate adjustment
- 5 Connector for the handpiece cord
- 6 Handpiece cord
- 7 Magnetic handpiece support
- 8 Purge switch
- 9 AIR-FLOW® CLASSIC+SOFT Powder diffuser
- 10 Cap ring
- 11 Cap dome
- 12 Cap seal
- 13 AIR-FLOW® PERIO+PLUS Powder diffuser
- 14 Inter-changeable colored ring

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


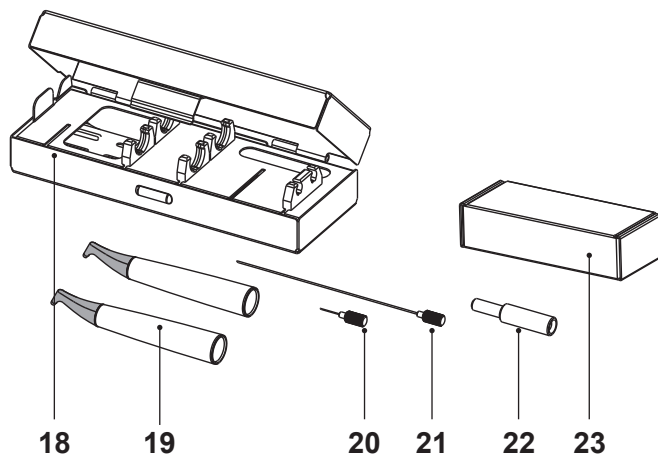
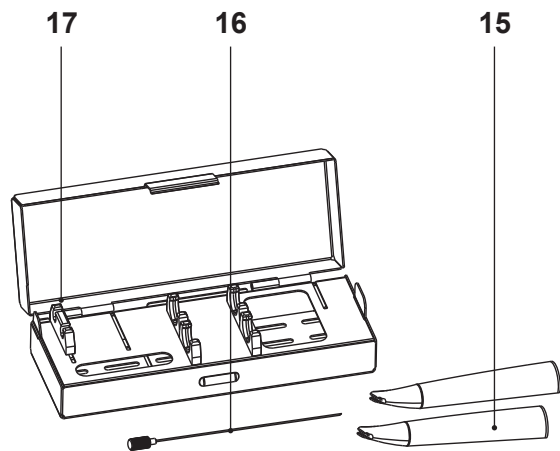
- 15 PERIO-FLOW® handpiece
- 16 Cleaning needle
- 17 PERIO-FLOW® SystemBox
- 18 AIR-FLOW® SystemBox
- 19 AIR-FLOW® handpiece
- 20 Short cleaning needle
- 21 Long cleaning needle
- 22 Needle guide
- 23 Maintenance Set

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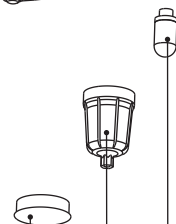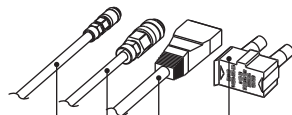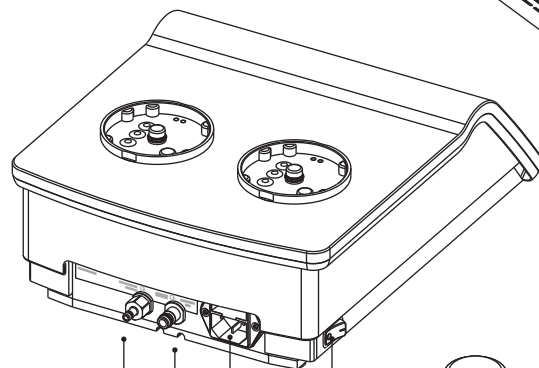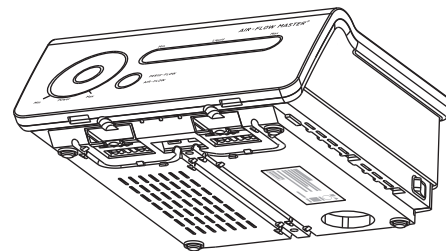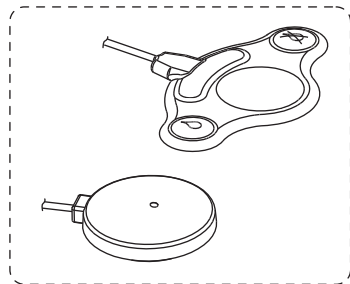35

36

- 24 Foot pedal (2 step or multifunction depending on order)
- 25 Water supply hose
- 26 Compressed air supply hose
- 27 Power cable
- 28 Fuses
- 29 Water hose connector
- 30 Air hose connector
- 31 Power socket
- 32 Power switch
- 33 AIR-FLOW® CLASSIC powder bottle
- 34 Cap
- 35 Air filter housing
- 36 Filter cartridge

Hu-Friedy & EMS supply units with various accessories. The "Packing list" shows exactly what your unit includes.

Technical modifications

Hu-Friedy & EMS reserve the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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DEAR CUSTOMERS,

Thank you for buying a new Hu-Friedy & EMS product. It meets the highest quality and safety standards.

Your AIR-FLOW MASTER is a combination of two air polishing units.

It is for use in dentistry with Hu-Friedy & EMS AIR-FLOW systems.

This product functions with the powders and the systems supplied by Hu-Friedy & EMS.

The confinement of the powder by a water jet makes it possible to steer the water jet with great precision and thus ensures a pleasant treatment for your patient.

This product removes dental plaque, soft deposits and surface stains from pits, grooves, interproximal spaces or smooth surfaces of the teeth.

Applications

- Plaque removal for placement of sealants,
- Surface preparation prior to bonding/cementation of inlays, onlays, crowns and veneers,
- Surface preparation prior to placing composite restorations,
- Cleaning prior to bonding of orthodontic brackets,
- Effective plaque and stain removal for orthodontic patients,
- Clean implant fixture prior to loading,
- Stain removal for shade determination,
- Plaque removal prior to fluoride treatment,
- Plaque and stain removal prior to whitening procedure.

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PERIO-FLOW system (optional)

This product must only be used by a qualified and trained personnel (dentist or dental hygienist).

This product should only be used with powders supplied by Hu-Friedy & EMS for a subgingival application.

This product remove the subgingival biofilm during periodic periodontal treatments. Its long fine nozzle was specifically developed for easy access inside the periodontal pocket.

For further details concerning the treatments, please refer to the instruction which are supplied with your AIR-FLOW or PERIO-FLOW systems. Please do not hesitate to contact your local Hu-Friedy & EMS dealer for further information.

 The instructions in this manual cover the combined use of an AIR-FLOW handpiece and a PERIO-FLOW handpiece. Please refer only to instructions related to AIR-FLOW handpiece when a PERIO-FLOW handpiece is not present.

Contra-indications:

Please refer to operation instructions of the related system.

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ABOUT THIS MANUAL

This operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Instructions concerning the components of the system are provided separately in the relative instructions for use. Read the instruction manuals originally delivered with each of the components before using this product. Do not hesitate to contact your local Hu-Friedy & EMS dealer for further information.

Always keep this instruction close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

Danger

Risk of injury

Caution

Risk of property damage or environmental harm

Please note

Useful additional information and hints

Wear gloves

Prohibited

Authorized

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly your approved Hu-Friedy & EMS repair center or your dealer.

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Hu-Friedy & EMS

CONTENTS

Technical modifications.....	7	Operating the unit	36	Maintenance.....	70
Installation and set up	2	Switching On	36	Replacing the dome	70
Connecting the "2 step" foot pedal	2	Working mode selection.....	40	O- ring	72
Connecting the "multifunction" foot pedal....	4	PERIO-FLOW mode detection	42	Valve assembly	74
Connecting to a second Hu-Friedy & EMS unit (optional).....	6	Setting the air/powder flow rate	44	Replacing the O-rings of the connectors...	76
Connecting the unit to the water supply	8	Setting the water flow rate	46	Replacing the water filter and the fuses	78
Connecting the unit to the compressed air supply	10	Before using the unit	48	Air filter	80
Connecting the unit to the mains power ...	12	Purge switch and mode.....	50	Safety precautions	82
Electrical Safety precautions	14	Powder exchange or refill.....	52	Storing the product	84
Connecting the cords and placing the handpiece supports	16	End of the treatment	54	Product disposal	86
Powder chambers	18	Powder and water purge	54	Warranty.....	86
Supragingival treatment using the AIR-FLOW CLASSIC+SOFT powder chamber.....	20	Cleaning, disinfecting and sterilizing	56	Accessories.....	86
Supragingival treatment using the AIR-FLOW PERIO+PLUS powder chamber	22	Cleaning the unit	56	Hu-Friedy & EMS-Service.....	88
Subgingival treatment using the AIR-FLOW PERIO+PLUS powder chamber	24	Cleaning the Handpiece cord	58	Symbols	90
Powder choice.....	26	Cleaning the dome	60	Technical data	94
Filling the powder chambers	28	Cleaning the Powder Chamber	62	Electromagnetic compatibility	96
Preparing for treatment.....	32	Unclogging the handpiece	64	Troubleshooting	98
Connecting the AIR-FLOW handpiece	32	Cleaning and sterilizing.....	66		
Installing the nozzle and connecting the PERIO-FLOW handpiece.....	34	Cleaning the handpiece	66		
		Sterilization of the handpiece and the cleaning needles	68		
		Packaging for Sterilization	68		
		Sterilization.....	68		

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
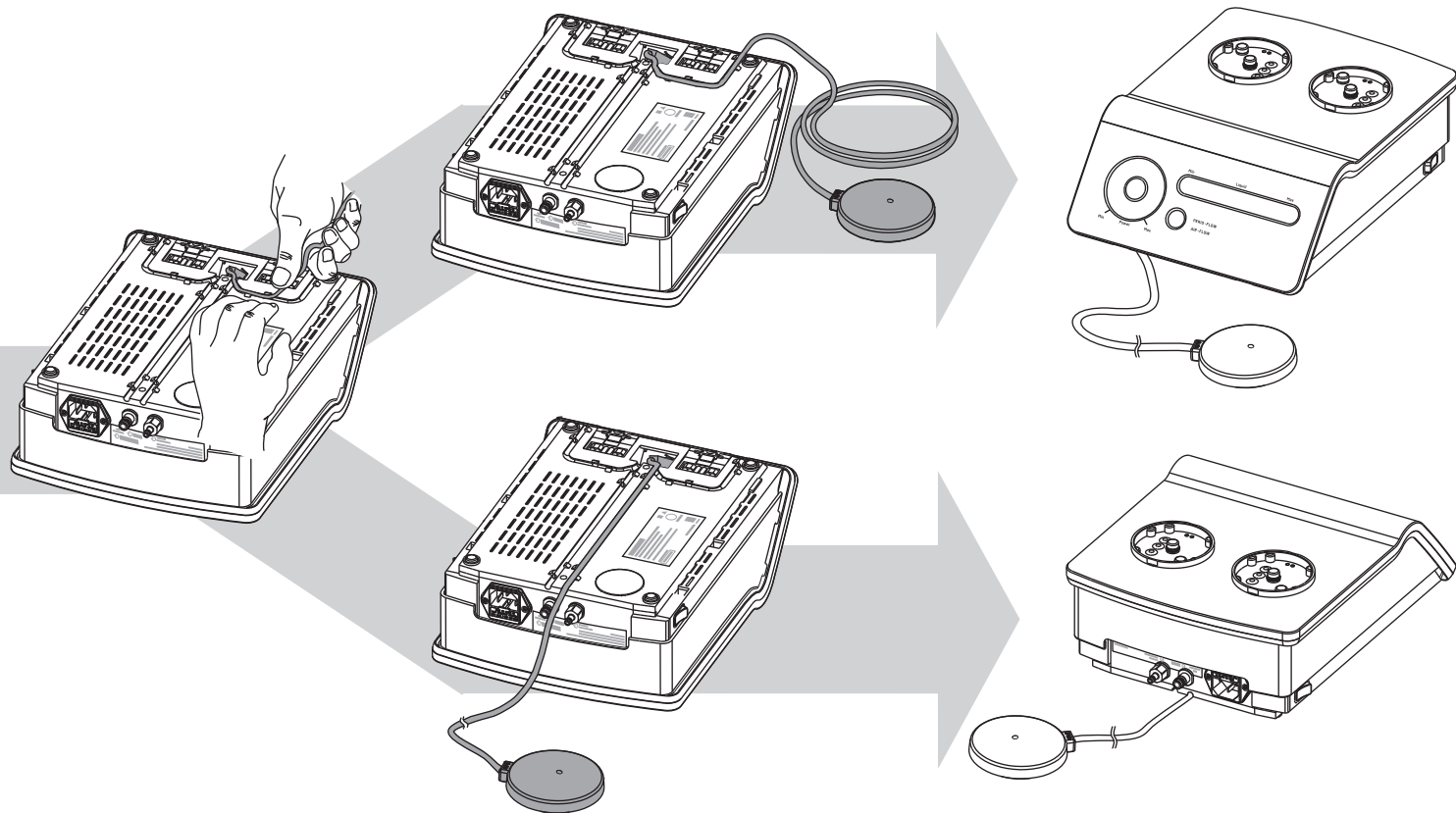
INSTALLATION ET BRANCHEMENT


Connexion de la pédale "2 step"

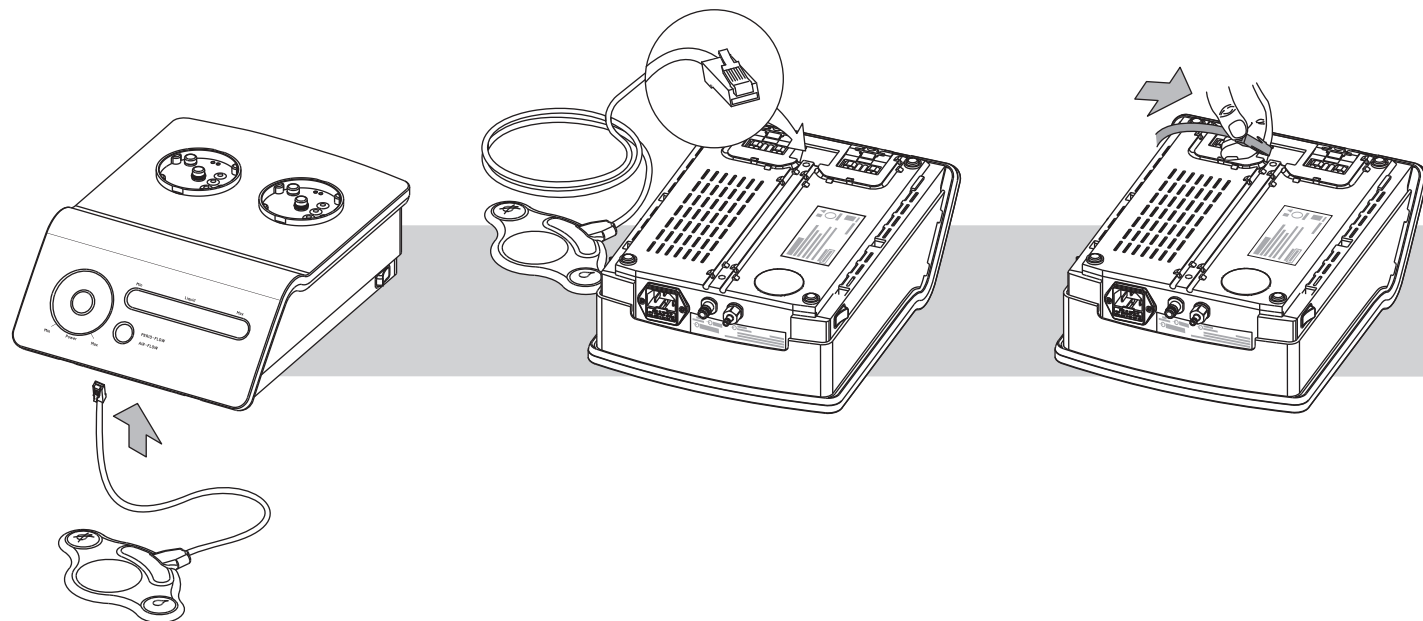
Connecting the "multifunction" foot pedal

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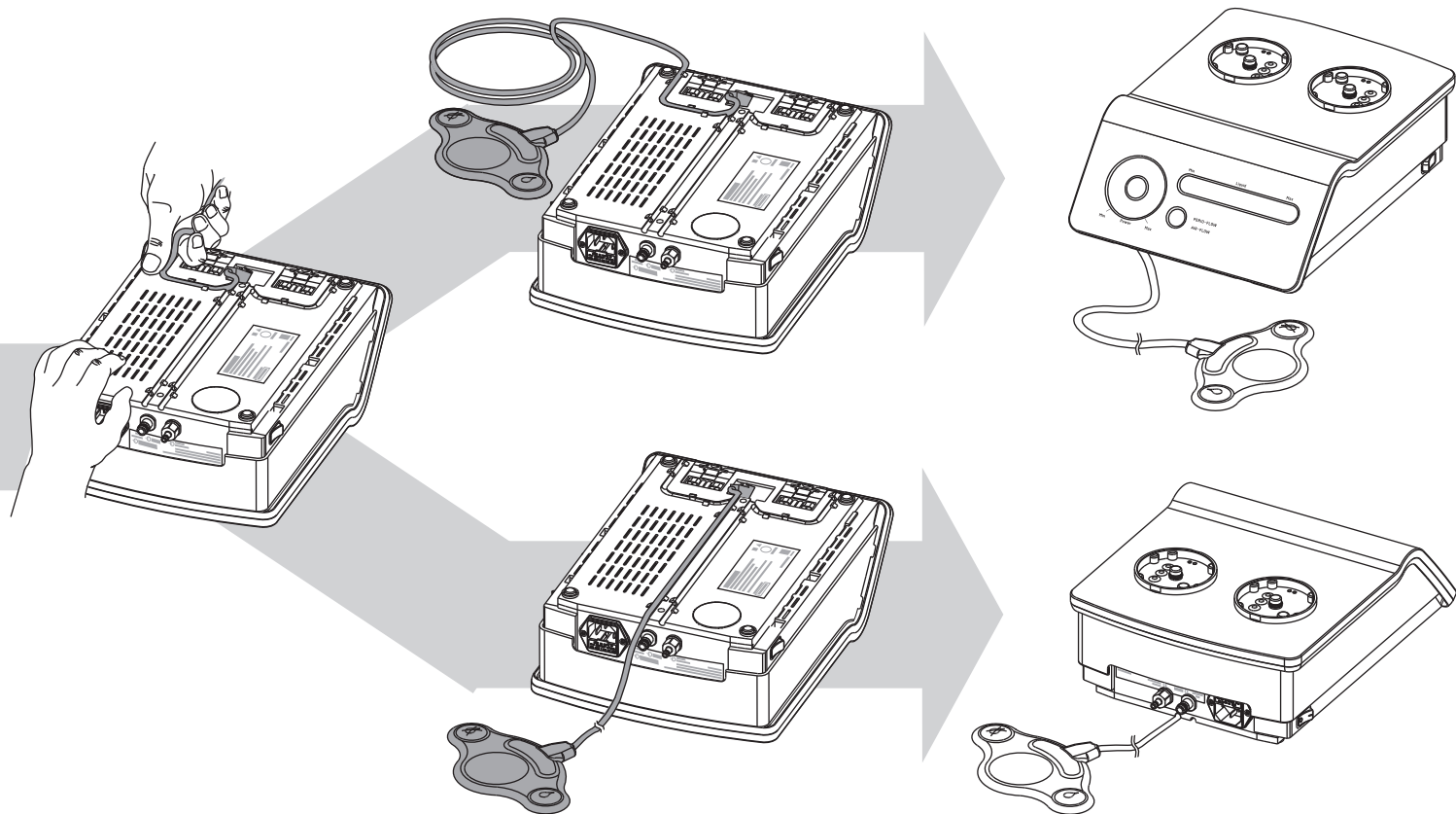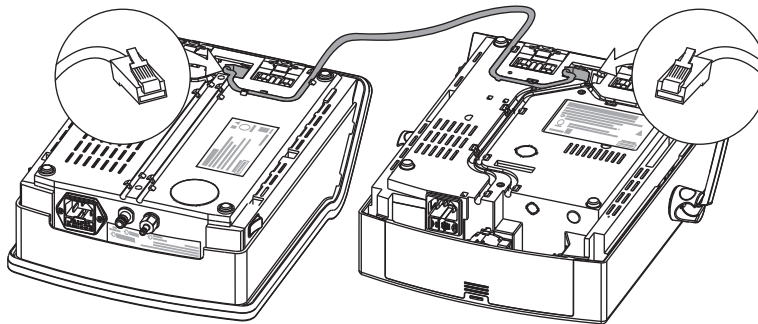Connecting to a second Hu-Friedy & EMS unit (optional)

The AIR-FLOW MASTER can be connected to another compatible Hu-Friedy & EMS device, so both devices can be operated using a single foot pedal. A special optional link cable allows this connection. This option is only available with the "multi-function" pedal.

The pedal will trigger only the handpiece which is lifted from its support whatever the unit.

For additional details on device compatibility, please contact the distributor in your country.

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

Connecting the unit to the water supply

BAR	1.0 - 5.0
PSI	15 - 72
kPA	100 - 500

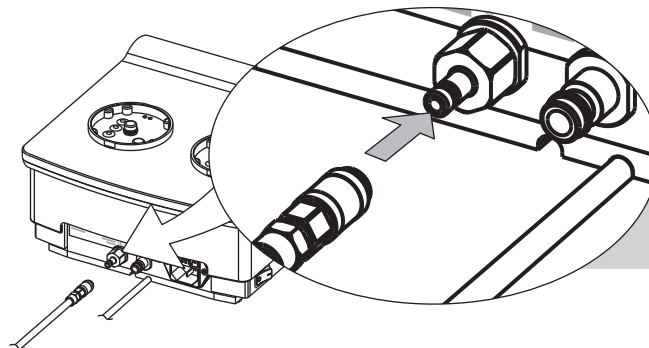Connect the unit to a water supply with the original Hu-Friedy & EMS water hose supplied with your product.

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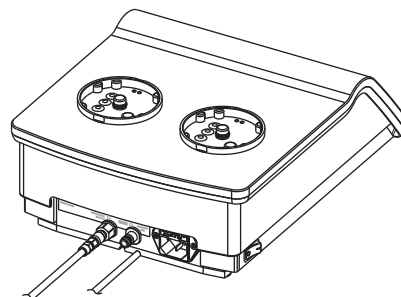
Connecting the unit to the compressed air supply

BAR	5.5 - 7.5
PSI	80 - 109
kPA	550 - 750

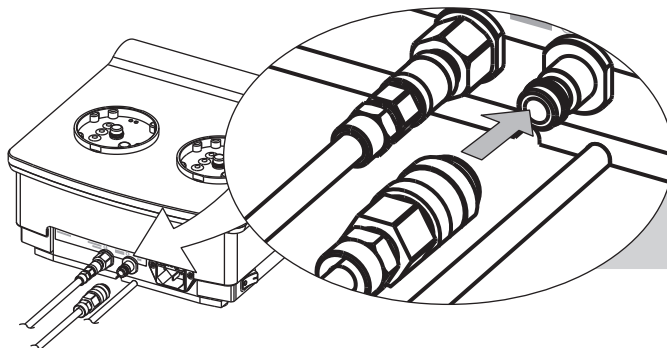
Use only dry and filtered compressed air.

Check that the supply pressure of compressed air does not exceed the specified value (mentioned on the unit).

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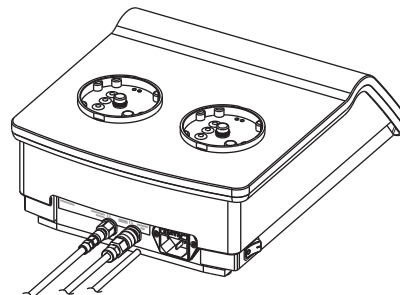
Connecting the unit to the mains power

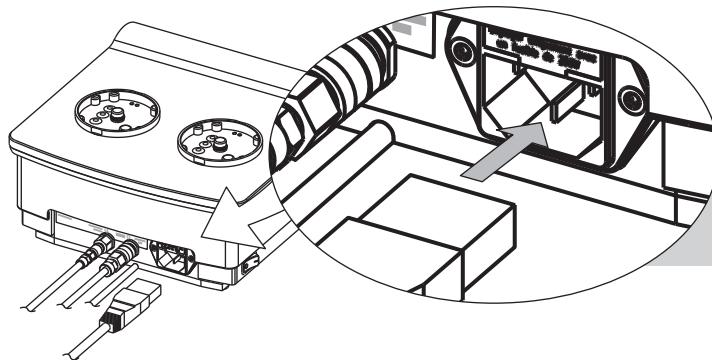100-240 VAC 50-60Hz


 To prevent damage to the unit, check that its rated voltage corresponds to the local line voltage.

Use the unit only in a FI protected mains supply (FI = Residual current protection).

 For USA or Canada: connect only to Hospital Grade receptacle.

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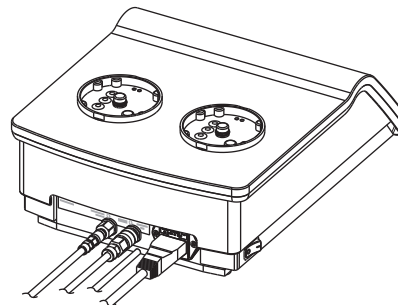Electrical Safety precautions

To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

No modification shall be made on this product.

The main switch of the unit must be accessible at any time.

When the main switch is on the "0" position, the unit is disconnected from the supply network.

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
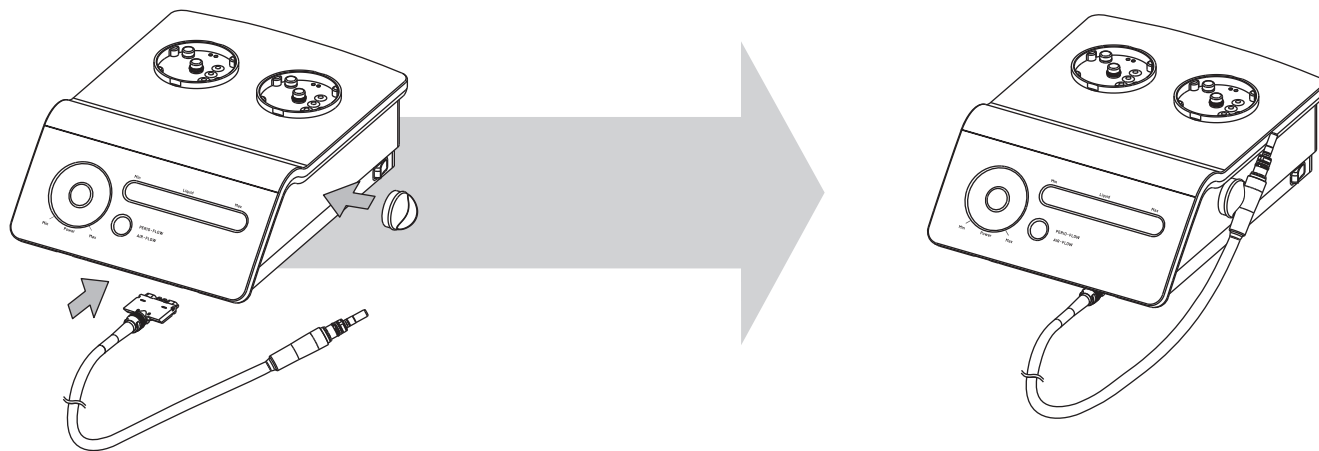
Connecting the cords and placing the handpiece supports

Check that the cords are fully dry before connecting to the unit.

Place the magnetic handpiece holder only on the metallic part of the unit or the furniture.

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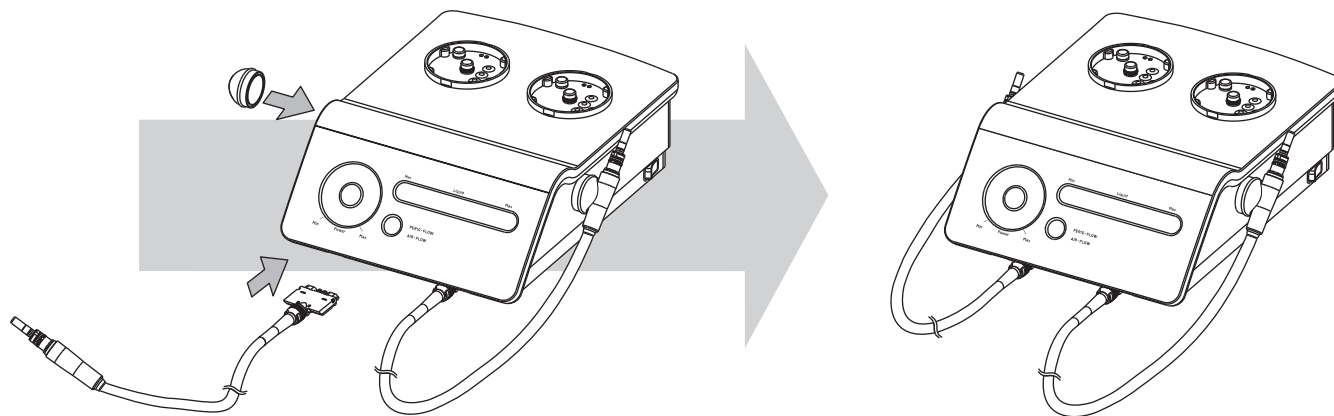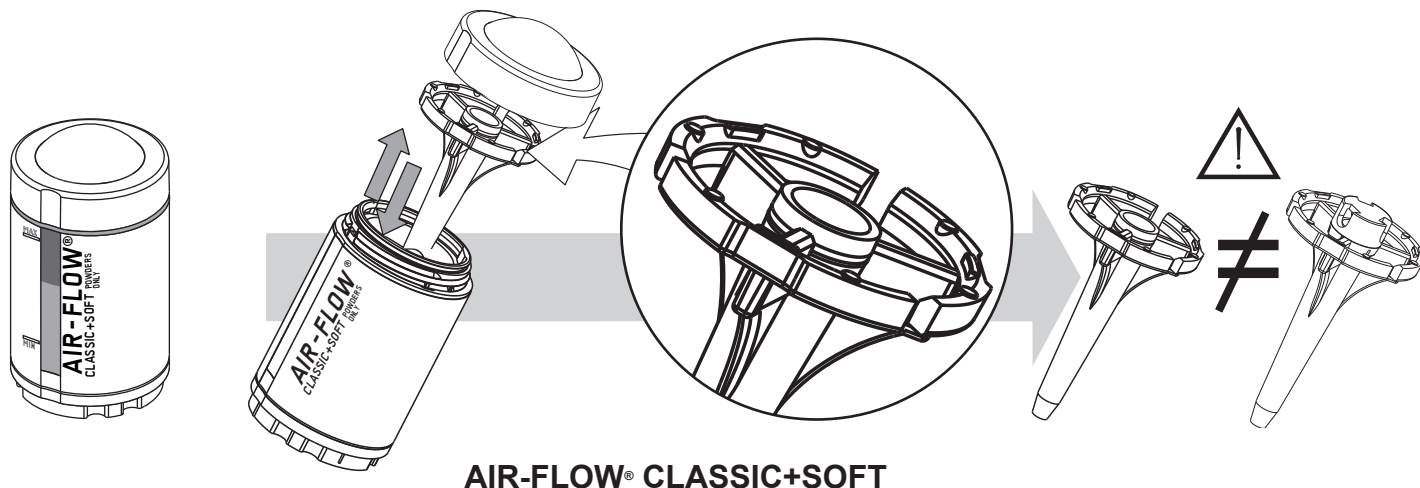
Powder chambers

AIR-FLOW® CLASSIC+SOFT

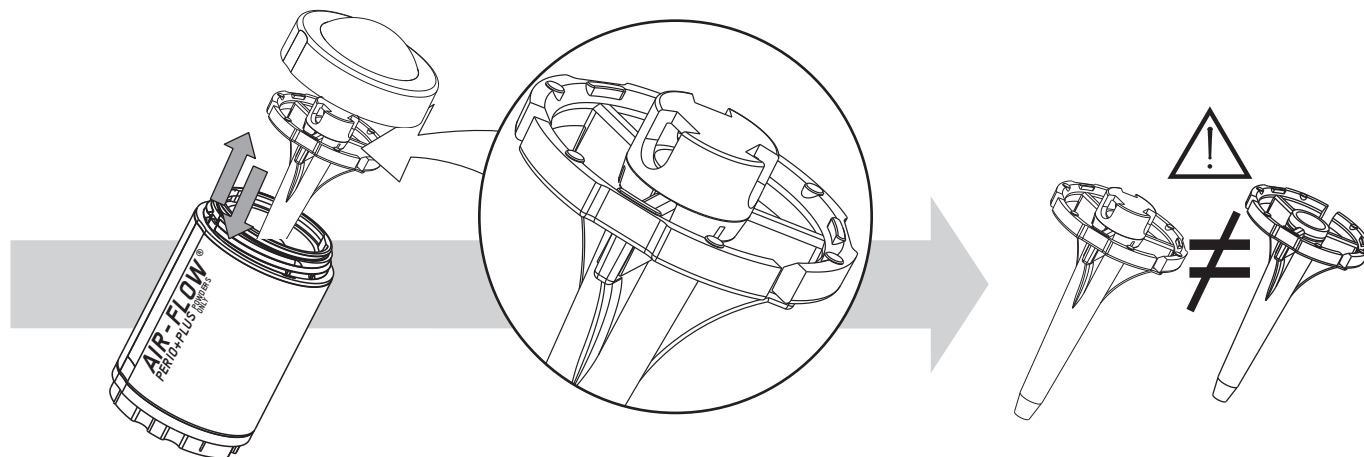
There are two powder chambers having each a specific diffuser.

 The AIR-FLOW CLASSIC+SOFT powder chamber can be used exclusively for supragingival treatment together with the CLASSIC, CLASSIC COMFORT and SOFT powders.

 The AIR-FLOW PERIO+PLUS powder chamber can be used for the subgingival treatment using the specific powder PERIO.

 Do not invert the diffusers, they are not compatible.

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
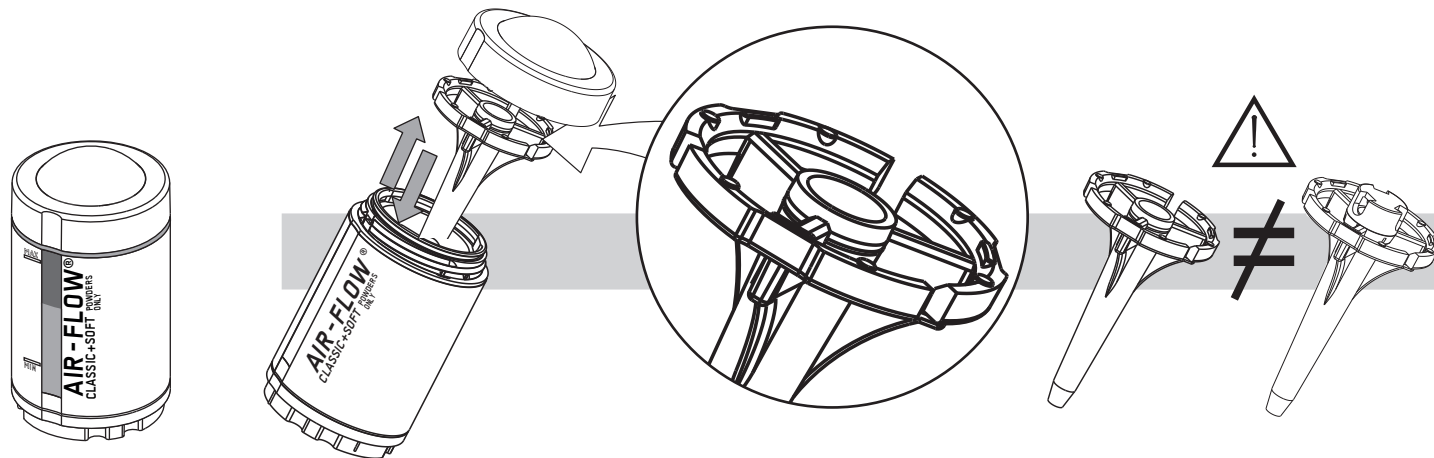
SUPRAGINGIVAL TREATMENT USING THE AIR-FLOW CLASSIC+SOFT POWDER CHAMBER

AIR-FLOW® CLASSIC+SOFT

-  The AIR-FLOW CLASSIC+SOFT powder chamber must be used only for supragingival treatments.
-  The AIR-FLOW CLASSIC+SOFT powder chamber can only be used with CLASSIC, CLASSIC COMFORT and SOFT supragingival powders.
-  Do not invert the diffusers, they are not compatible.
-  Do not use this powder chamber with the PERIO powder.
-  Use the AIR-FLOW handpiece

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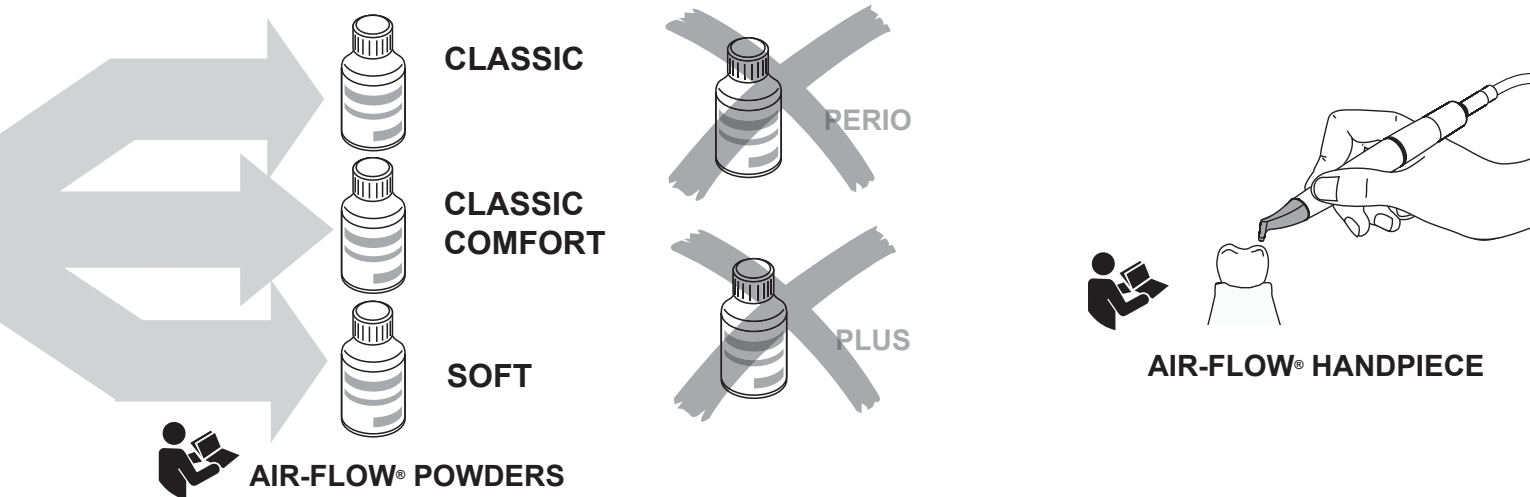
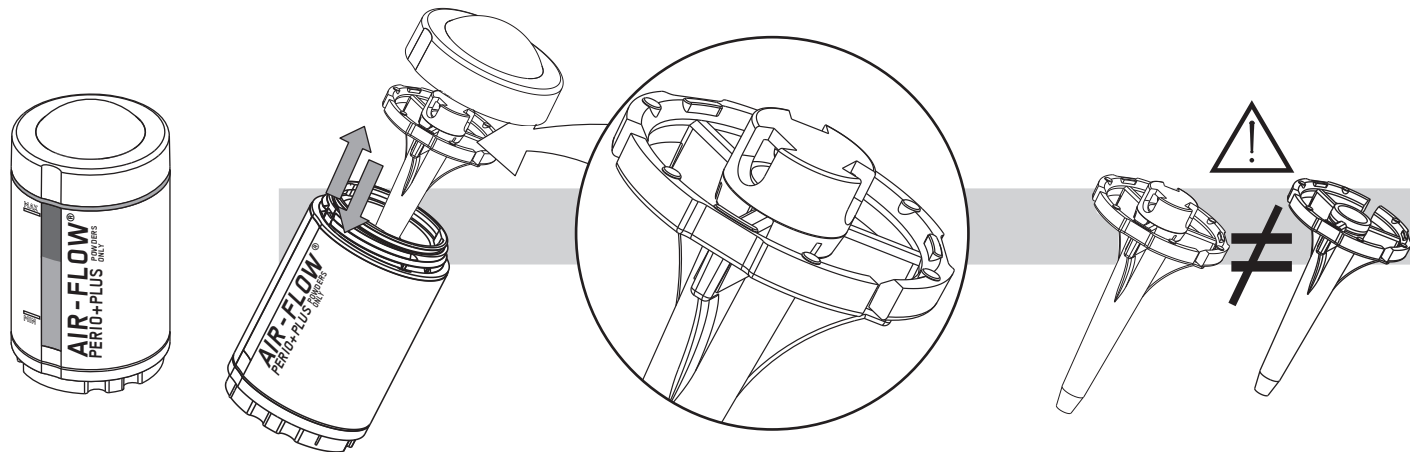
SUPRAGINGIVAL TREATMENT USING THE AIR-FLOW PERIO+PLUS POWDER CHAMBER

AIR-FLOW® PERIO+PLUS

-  The AIR-FLOW PERIO+PLUS powder chamber can be used for supragingival treatments using PLUS powder together with the AIR-FLOW handpiece.
-  Do not invert the diffusers, they are not compatible.
-  Do not use CLASSIC, CLASSIC COMFORT or SOFT powders with this powder chamber.
-  Use the AIR-FLOW handpiece.
-  Please contact your local retailer in order to verify if the EMS & Hu-Friedy AIR-FLOW PLUS Powder is available in your country.

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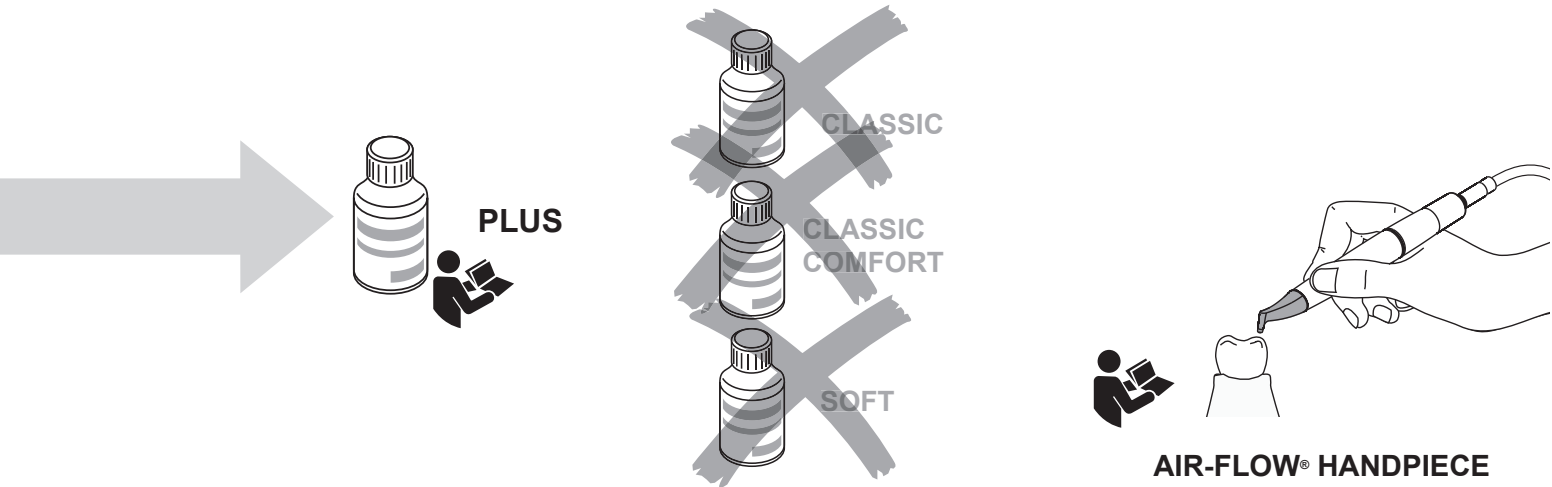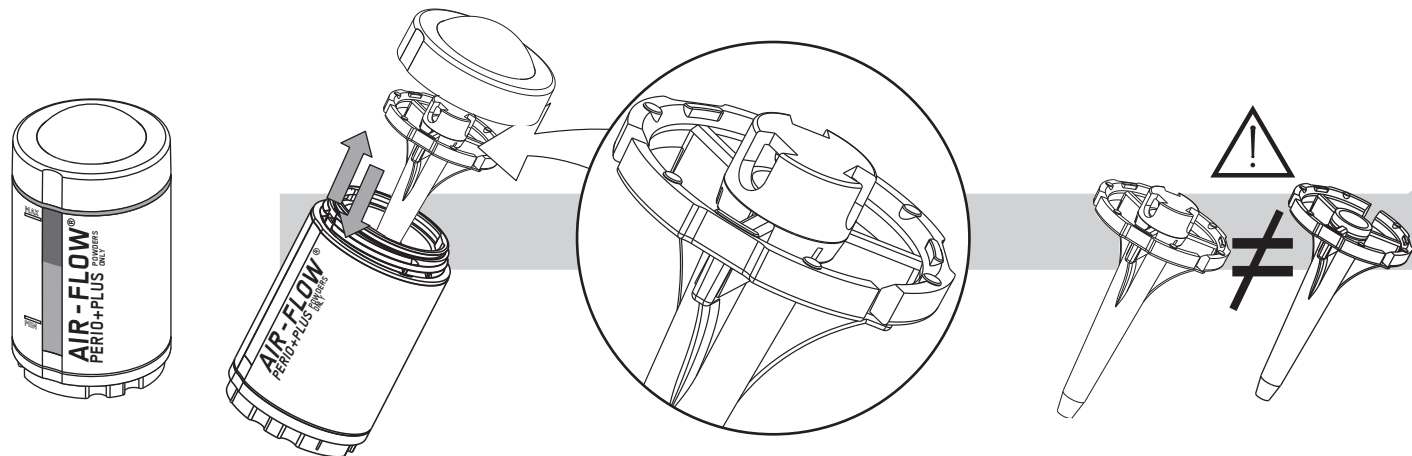
SUBGINGIVAL TREATMENT USING THE AIR-FLOW PERIO+PLUS POWDER CHAMBER

AIR-FLOW® PERIO+PLUS

 For a subgingival treatment with the AIR-FLOW PERIO+PLUS powder chamber use only the PERIO and PLUS powders.

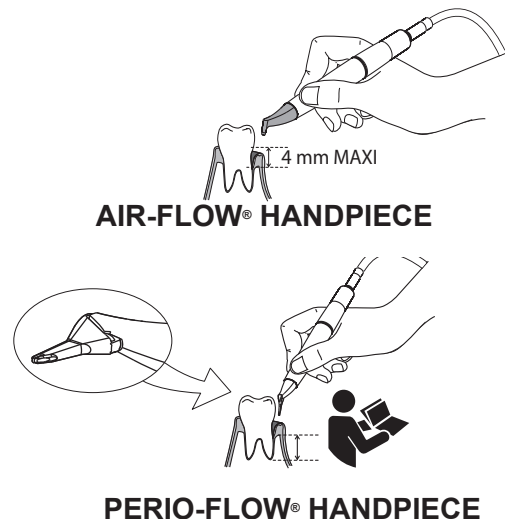
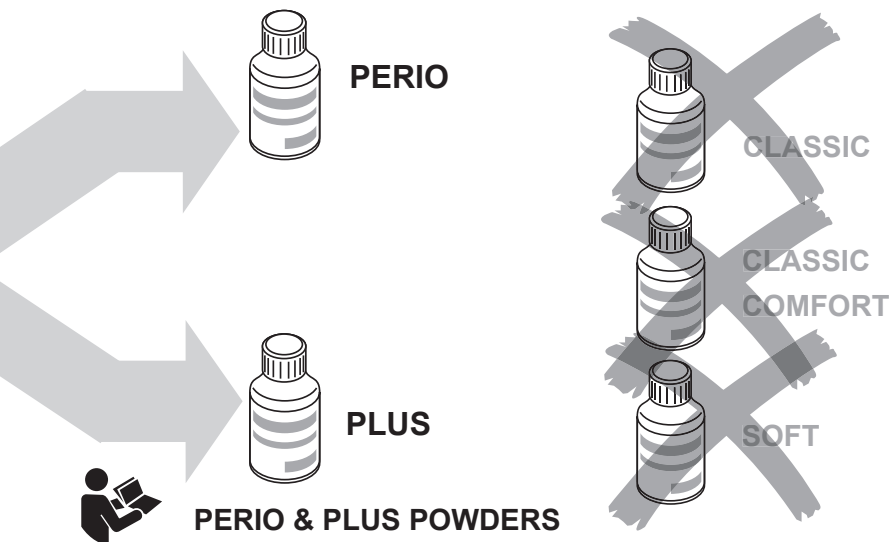
 Do not use CLASSIC, CLASSIC COMFORT or SOFT powders for subgingival treatments.

 Do not invert the diffusers, they are not compatible.

 Read the instructions delivered with the handpiece.

 Please contact your local retailer in order to verify if the EMS & Hu-Friedy AIR-FLOW PLUS Powder is available in your country.

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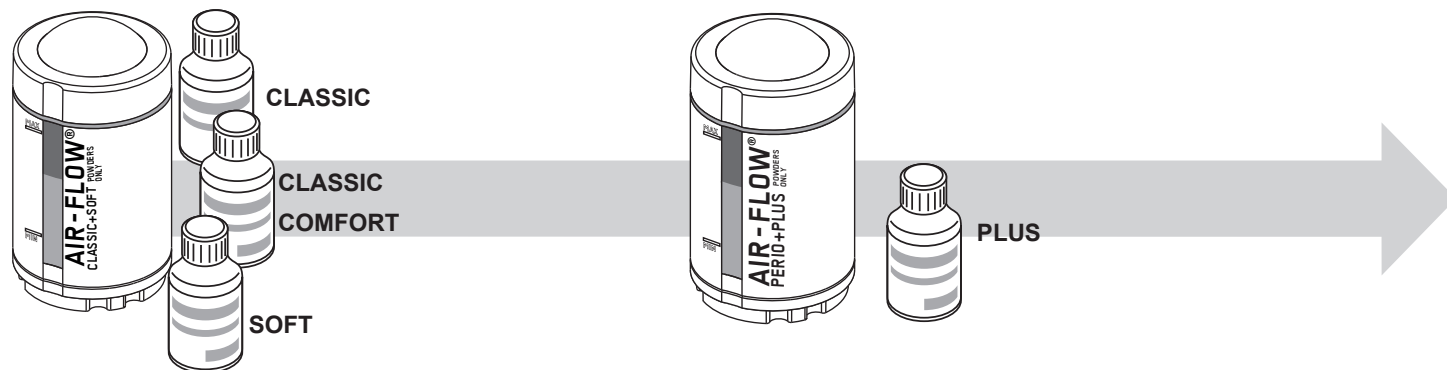
Powder choice

AIR-FLOW® CLASSIC+SOFT POWDER CHAMBER

AIR-FLOW® PERIO+PLUS POWDER CHAMBER

SUPRAGINGIVAL

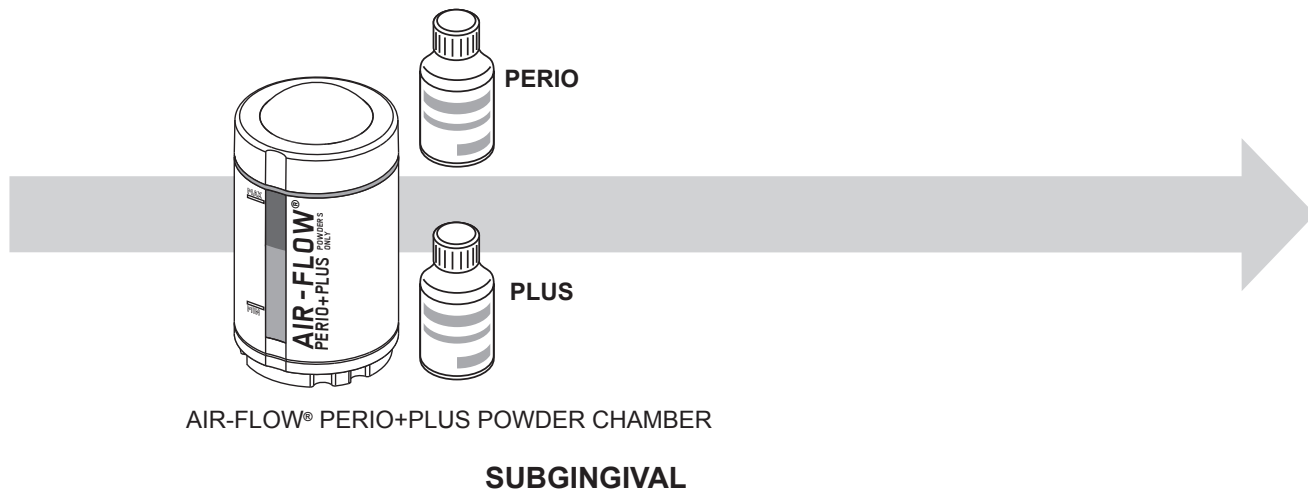
The powders compatible with the AIR-FLOW CLASSIC+SOFT powder chamber can only be used for supragingival treatments.

Read carefully the instructions given with the powder before using it. Specific utilisation of each powder is described in the instruction for use delivered with the powders.

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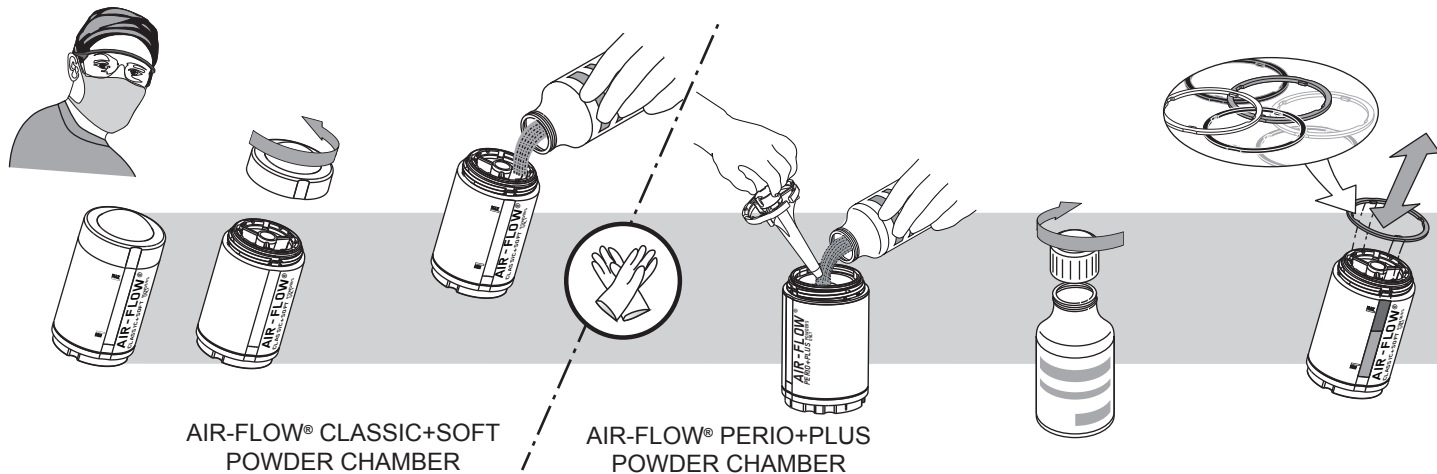
Filling the powder chambers


 Check the condition of the threads on the powder chamber, on the cap and on the O-ring seals. Please refer to the information provided in the chapter "Maintenance".

 Make sure that the powder chambers are absolutely dry. Moisture can cause the powder to cake and provoke a blocking of the circuit.

 To avoid clogging the unit do not fill the powder chambers above the line "Max". Pour the powder into the AIR-FLOW CLASSIC+SOFT powder chamber. The AIR-FLOW PERIO+PLUS powder chamber having a different design follow the instructions as shown above.

 Put the powder diffuser back to its position as soon as you finish to fill the powder chamber.

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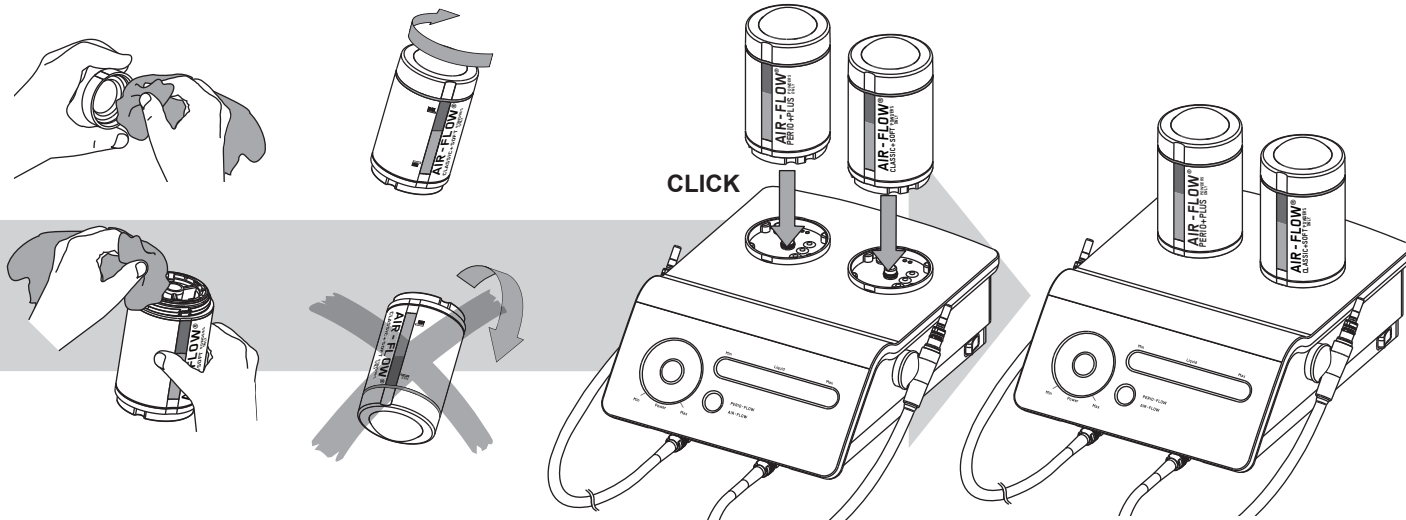 Use only powder provided by Hu-Friedy & EMS and corresponding to the application.

 Close correctly the bottle after use. The conditions of storage are described in the chapters "Storing the product" and "Technical data".

A set of interchangeable colored rings is provided to help recognize the chosen powder flavors. Please refer to the information provided in the chapter "Maintenance".

Clean the threads of the powder chamber and of the cap before screwing them on.

 Ensure that the powder chambers are closed tight. Otherwise air and powder could leak.

 Do not turn the powder chambers upside down after filling.

 Please contact your local retailer in order to verify if the EMS & Hu-Friedy AIR-FLOW PLUS Powder is available in your country.

 The powder should not remain over a determined time inside the powder chamber. Read the instructions delivered with the powders.

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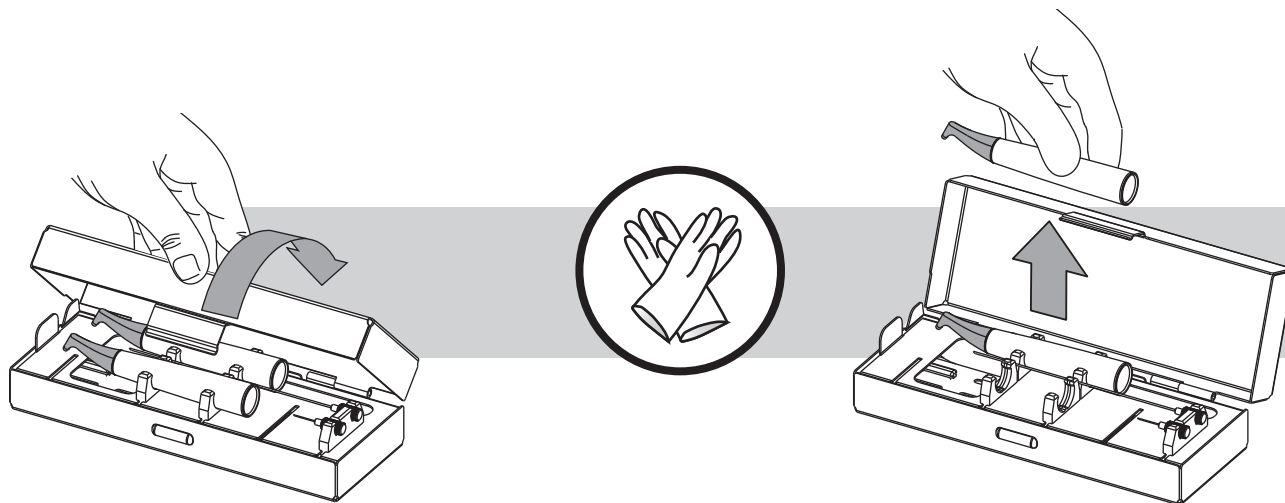
PREPARING FOR TREATMENT

Connecting the AIR-FLOW handpiece


For further details concerning the use of the handpiece, please refer to the operating instructions of the AIR-FLOW systems.

The AIR-FLOW handpiece can be used in combination with a AIR-FLOW PERIO+PLUS powder chamber in some applications.

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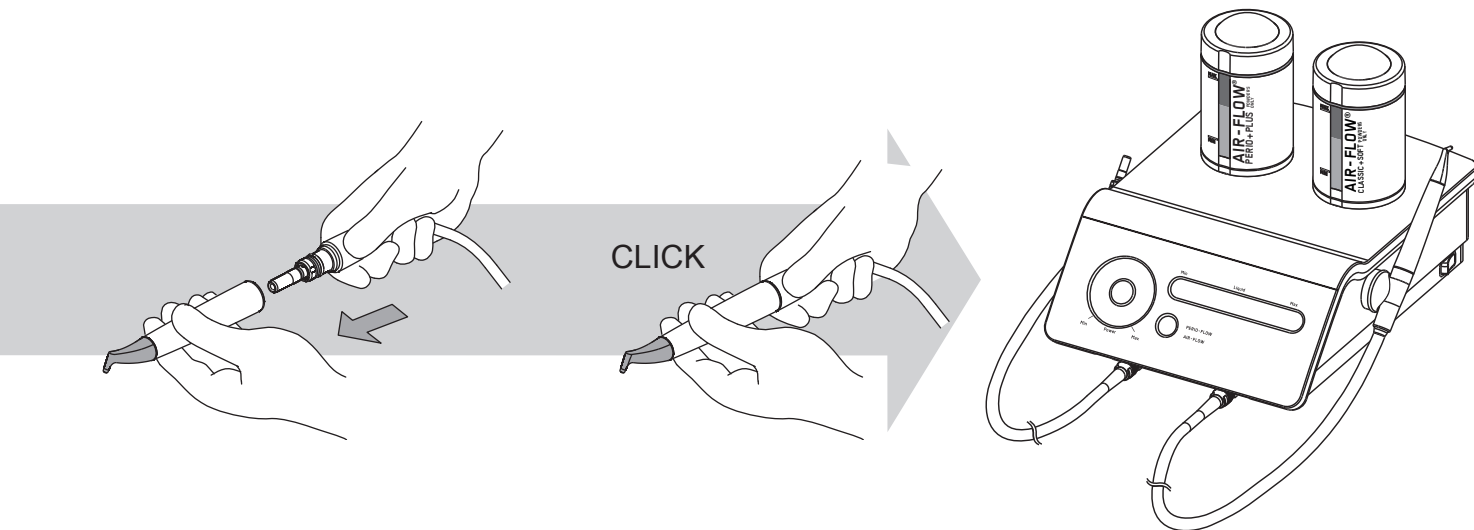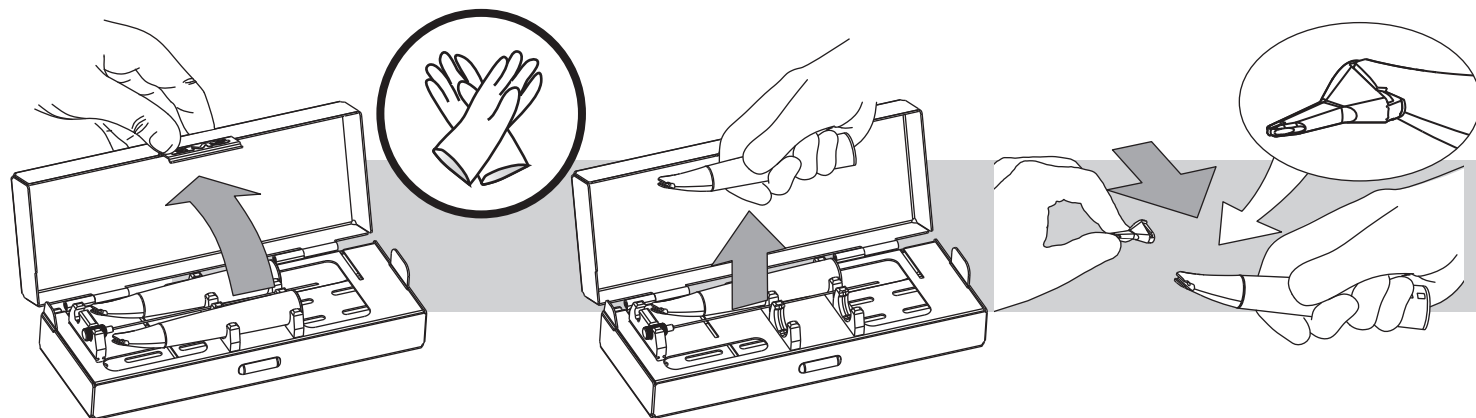
Installing the nozzle and connecting the PERIO-FLOW handpiece


For further details concerning the use of the handpiece please refer to the operating instructions of the PERIO-FLOW system.

The PERIO-FLOW handpieces must be used only with the AIR-FLOW PERIO+PLUS powder chambers.

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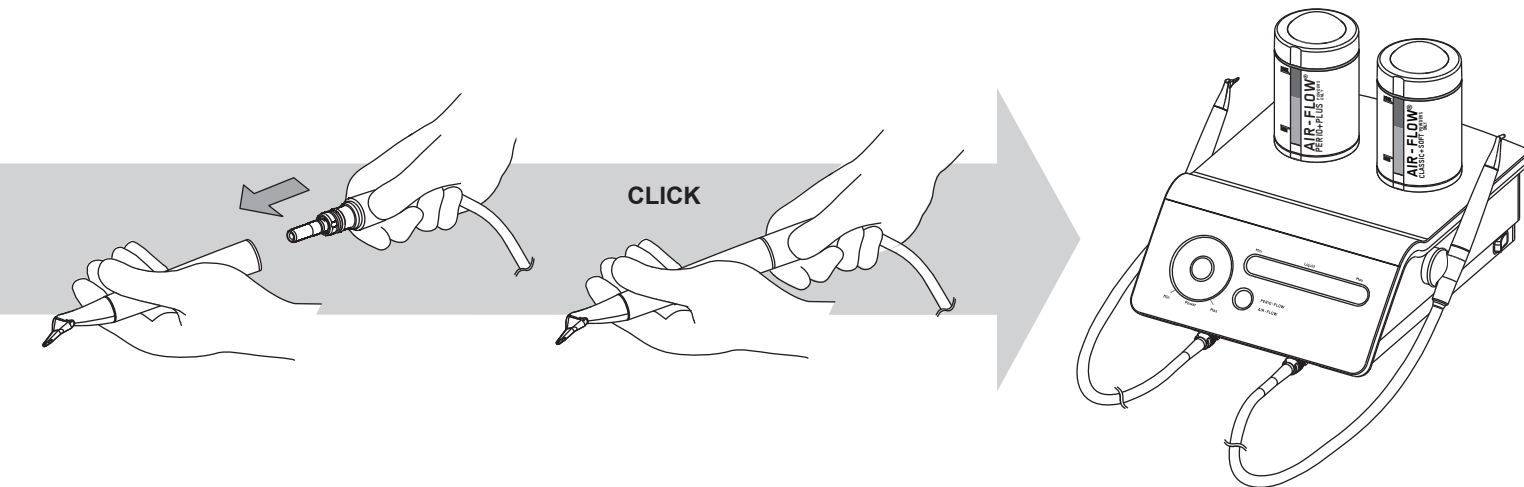
OPERATING THE UNIT

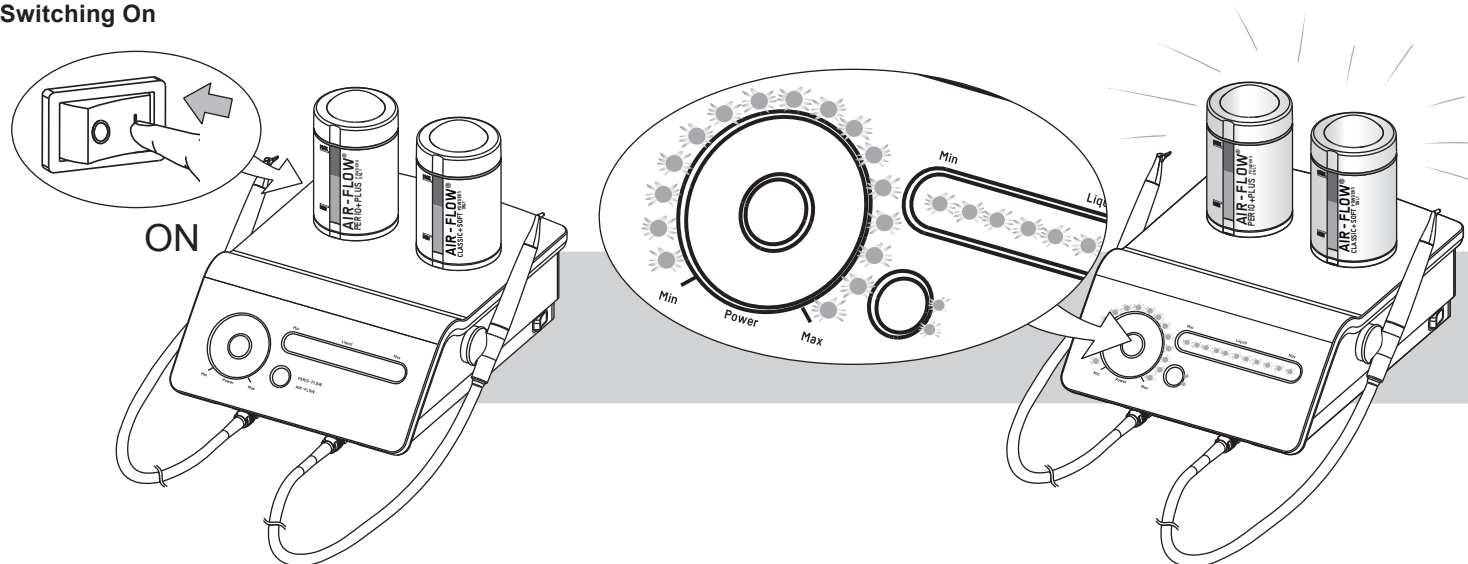Switching On


When the unit is switched on, the control lamps lights up and the powder chambers are under pressure.

 After 15 min. of inactivity, the unit turns itself on standby mode. Placing a finger on the sensitive zone wakes the unit from standby mode. The unit returns to its previous configuration.

The powder chambers light up only when the powder spray is used.

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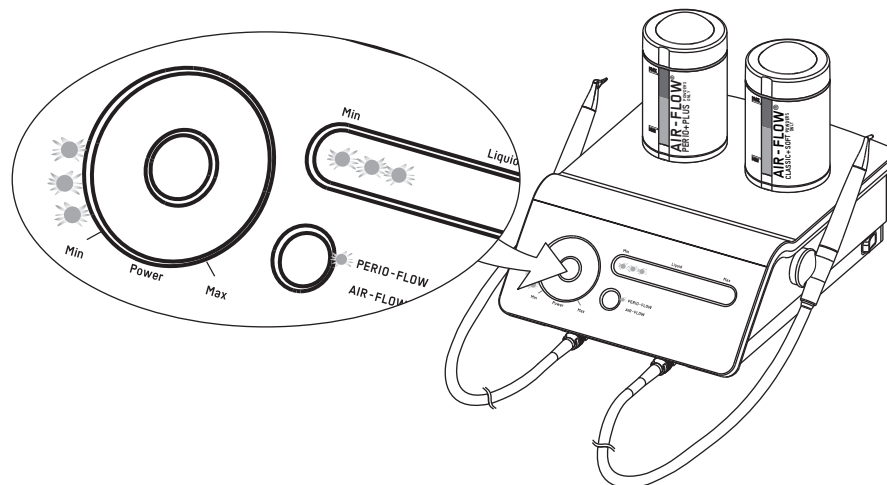
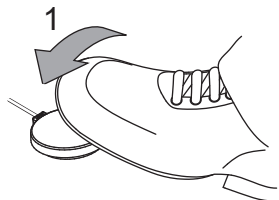
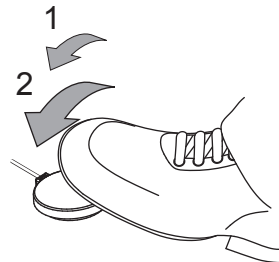
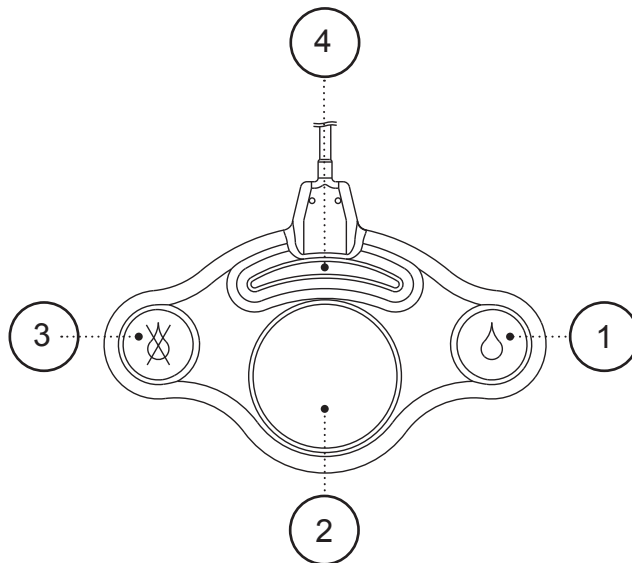
*Temporary air pressure increase of 30% (up to 100%)
The light of the powder chamber blinks rapidly.

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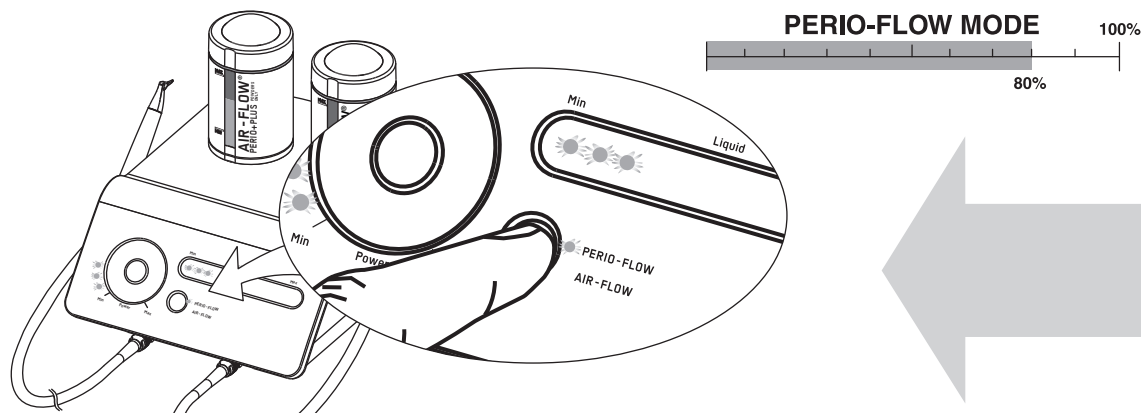Working mode selection


In PERIO-FLOW mode, the available air pressure is reduced compared to AIR-FLOW mode.

In PERIO-FLOW mode, one beep is generated every 5 seconds (recommended duration per single pocket treatment). Counting stops after 15 beeps, and 3 consecutive beeps will be generated. The sequence counting is initiated again when the pedal is released for more than 15 seconds.

The unit adapts the pressure when the mode is changed.

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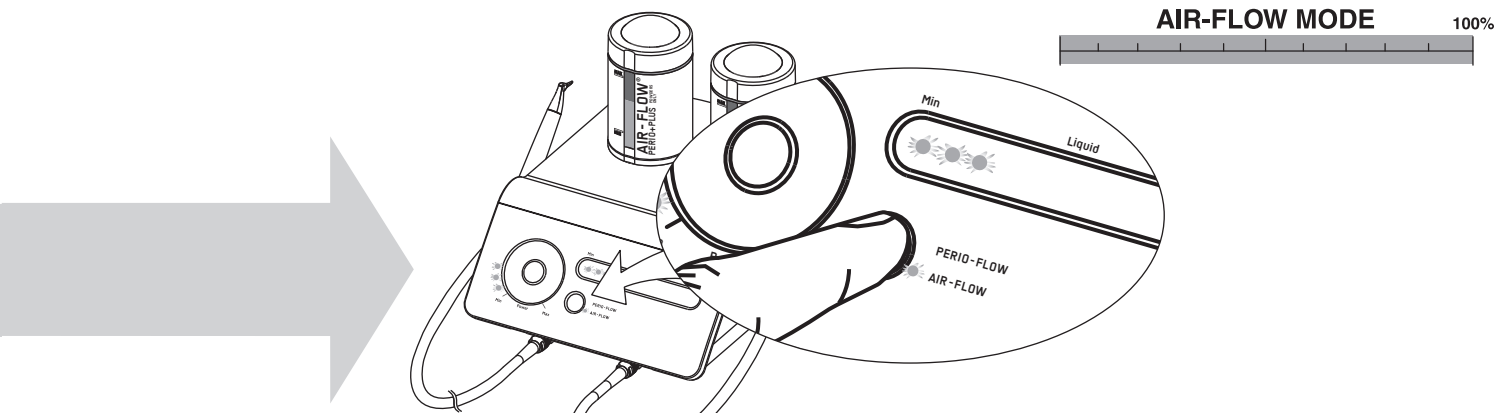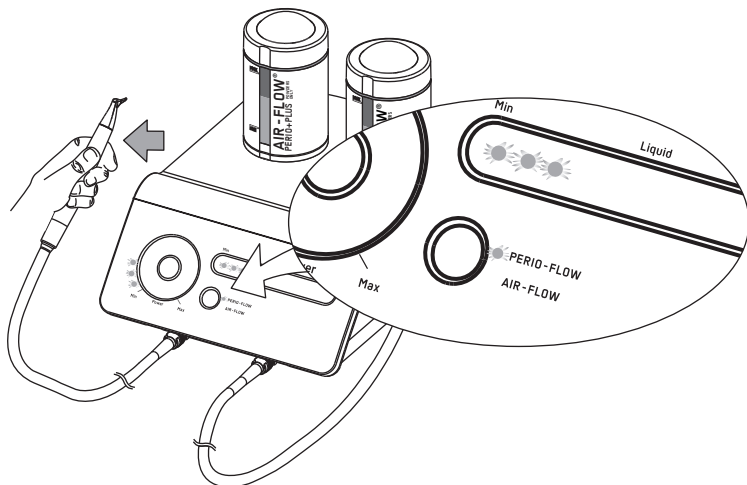
PERIO-FLOW mode detection

The PERIO-FLOW mode is automatically selected when the handpiece used with the AIR-FLOW PERIO+PLUS powder chamber is lifted from its support.

 Press the "working mode selection" switch to return to AIR-FLOW mode when the AIR-FLOW handpiece is used.

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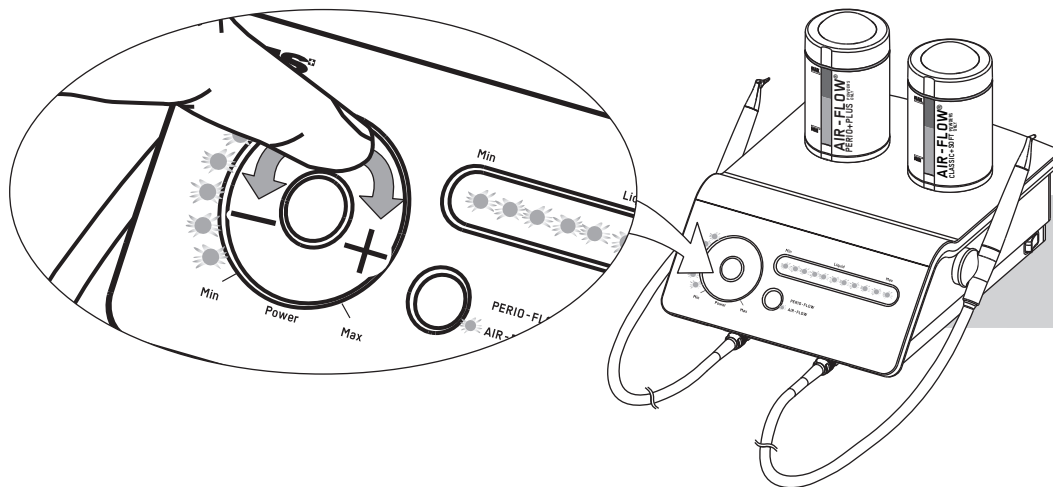
Setting the air/powder flow rate


The adjustments can be obtained either by sliding or by placing a finger on the sensitive zone. To place a finger directly at the desired location is a quick-setting alternative.

The unit has a memory function that allows for recalling the last settings at start up.

 Increasing the operating pressure will improve efficiency but also patient discomfort

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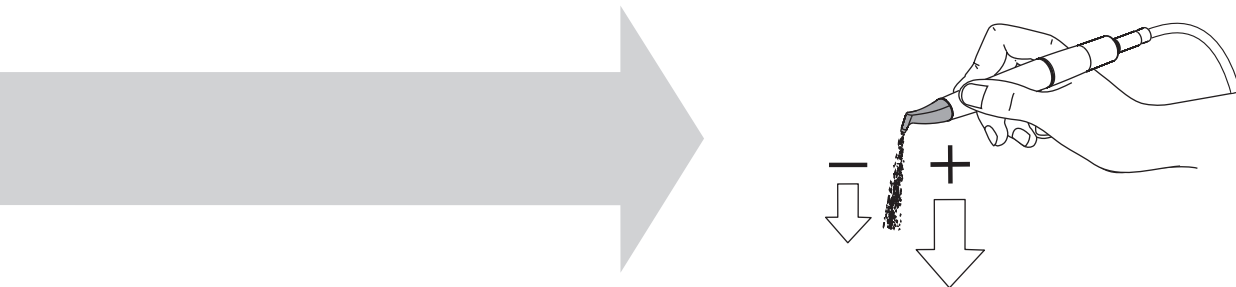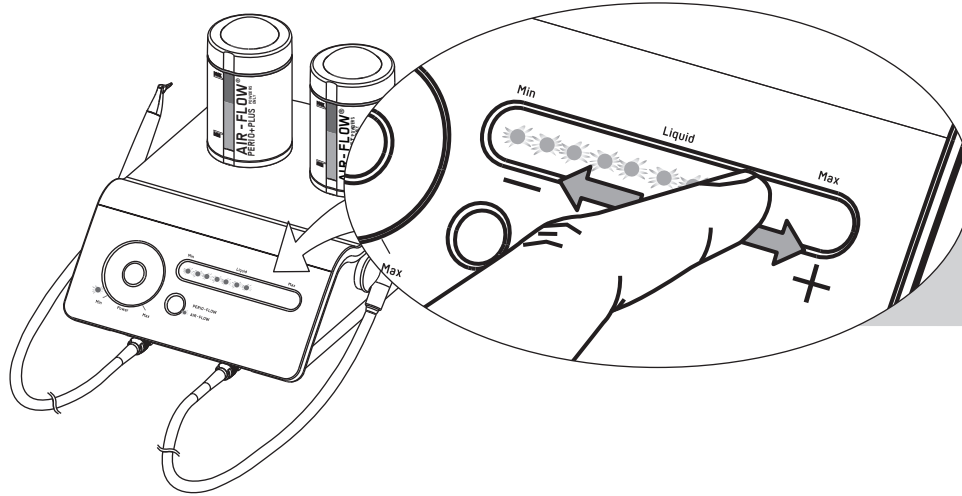
Setting the water flow rate


Information regarding how to use and adjust the water/air setting can be found in the operation instructions of the AIR-FLOW systems.

Use of the unit without water generates an important powder cloud that can make the patient feel uncomfortable.

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
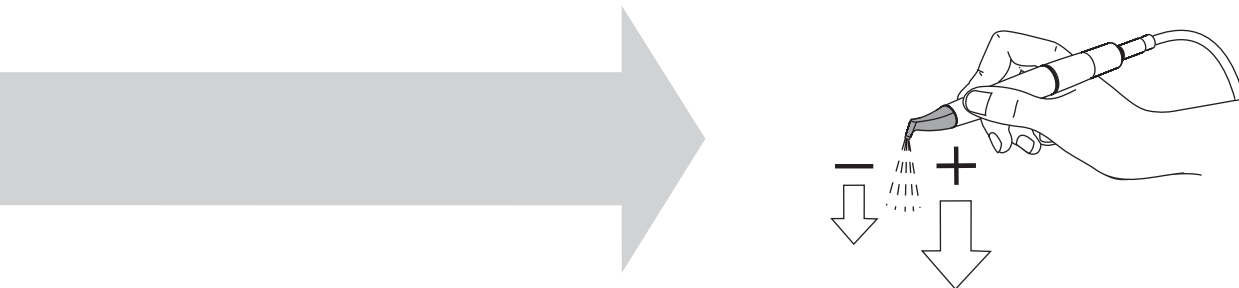
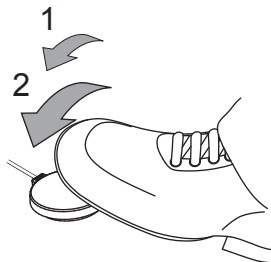
 After refilling the powder chamber it is recommended to ensure that the equipment is functioning. Make a test over a basin.

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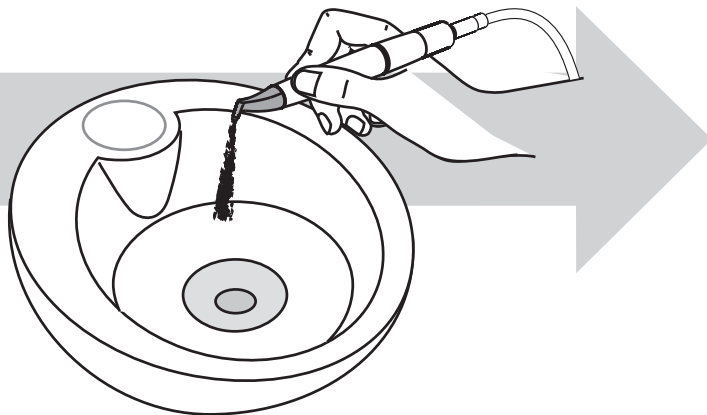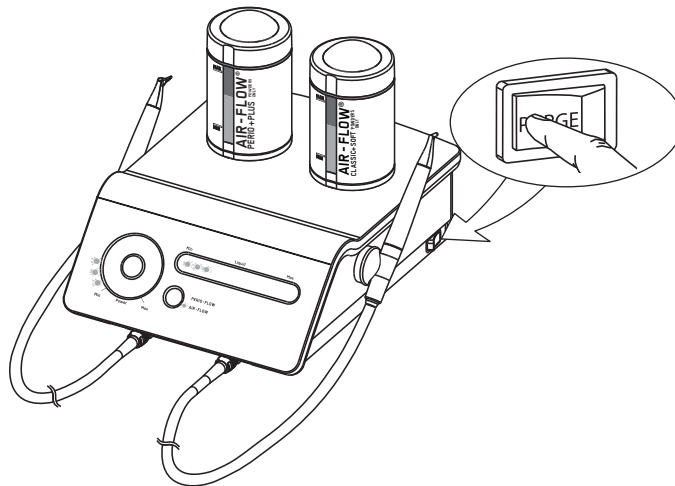Purge switch and mode

The purge switch allows to depressurize the powder chambers for safe removal. It also controls the "purge" function (see following pages).

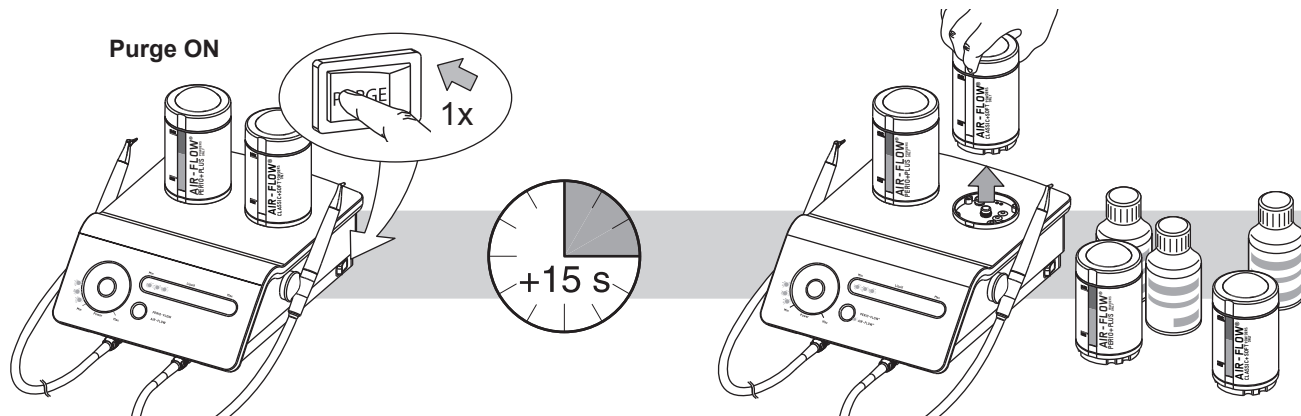
Beeps will be emitted when the purge switch is pressed and the powder chamber lights will blink. After approximately 1 minute the unit will return to its previous configuration (or when pressing the switch a second time).

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
Powder exchange or refill

-
- Do not remove or open the powder chambers when they are under pressure.
 - Press once on the purge switch to release the air pressure from the powder chambers.
 - Fill the chambers as indicated in chapter "Filling the powder chambers".
 - Never unscrew the cap of the powder chambers when they are on the unit.

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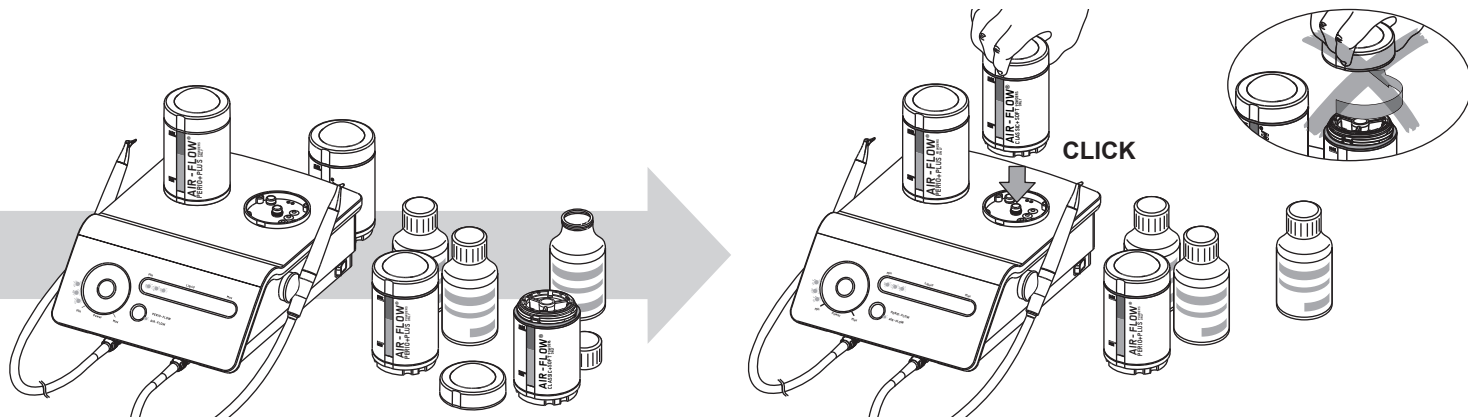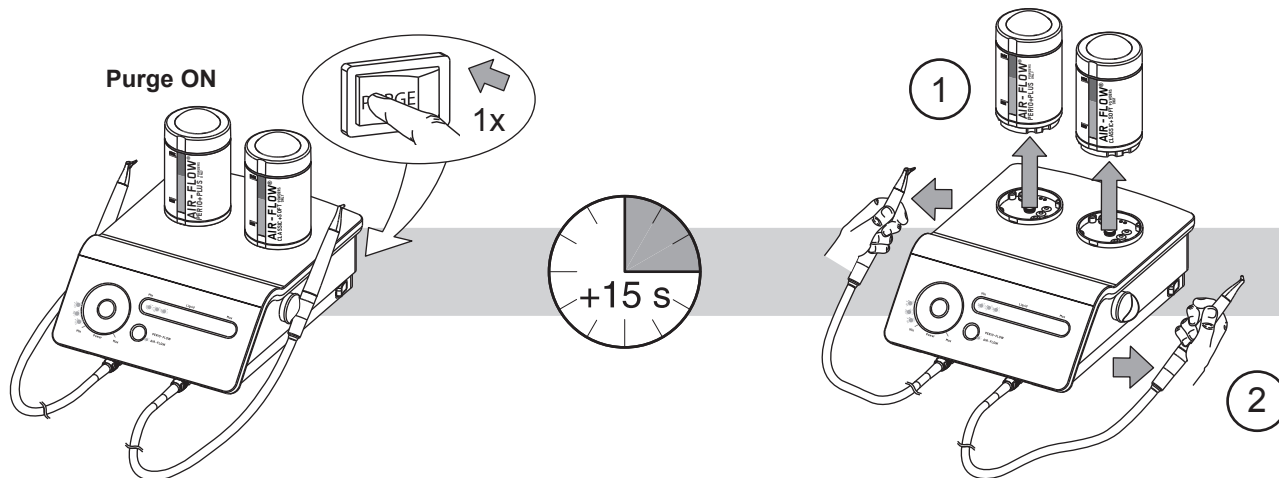
END OF THE TREATMENT

Powder and water purge

Remove the powder chambers before switching the unit off and store them in a dry place.

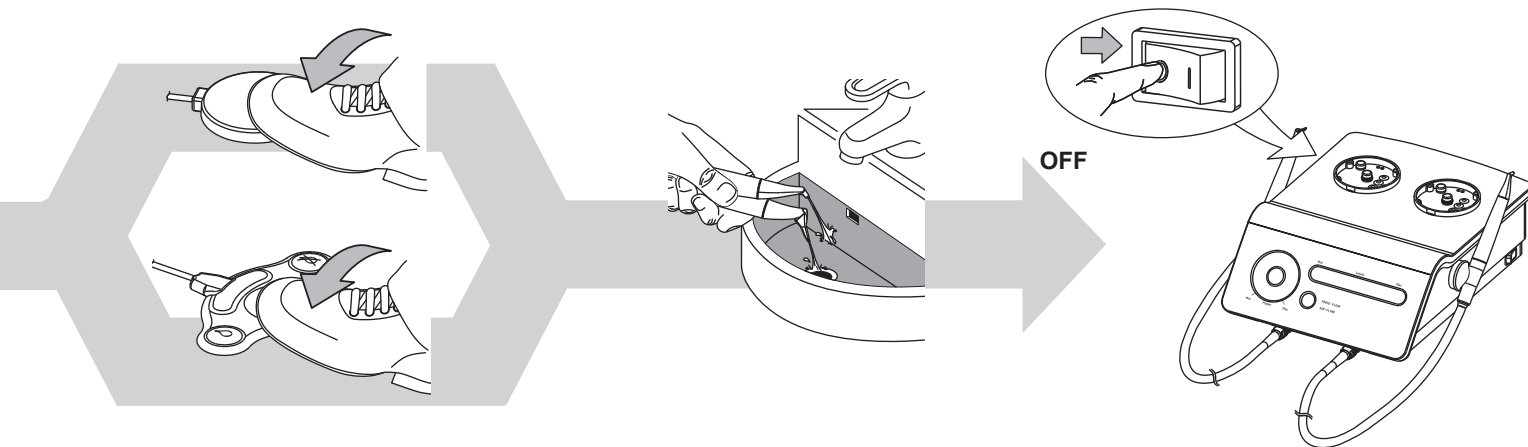
The powder should not remain over a determined time inside the powder chamber. Read the instructions delivered with the powders.

After each treatment a purge is necessary to remove all residues of powder and liquid inside the cords and the handpieces. During a prolonged stop of the unit, it is important to empty the irrigation circuit to avoid the formation of biofilm.

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CLEANING, DISINFECTING AND STERILIZING

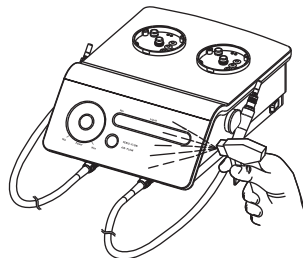
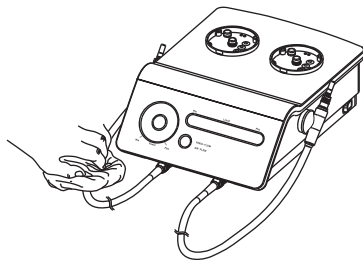
Please conform to the recommendations of the Reprocessing Instructions chapter of this manual.

Cleaning must be carried out immediately, no later than 30 minutes after the end of the treatment.

Follow present-day regulations enforced in the country about reprocessing.

Cleaning the unit

Clean the unit only with 70% isopropyl alcohol or other EPA recognized surface disinfectant, following carefully the instructions provided by the disinfection solution manufacturer. The use of scouring powder or an abrasive sponge will damage its surface.

Never remove the silicone cap from the powder chamber holder when cleaning the unit.

Do not use needles on the silicone cap of the powder chamber holder.

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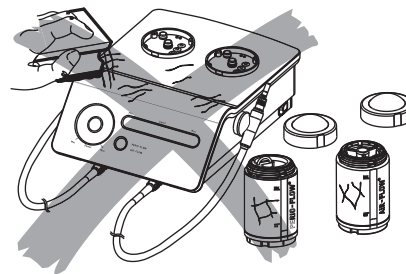
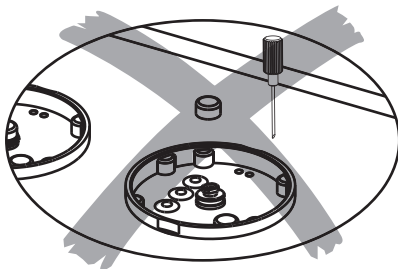
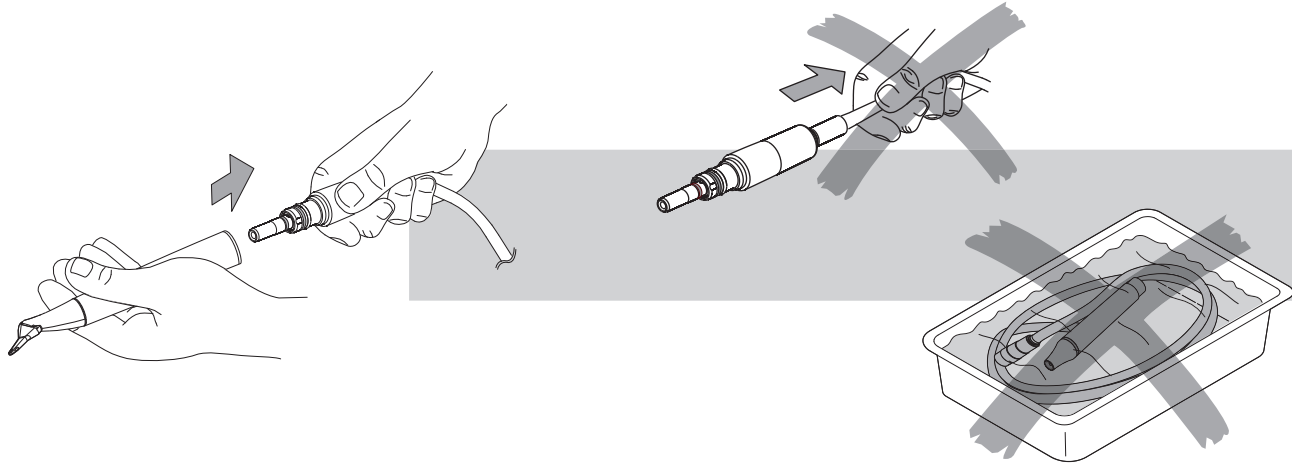
Cleaning the Handpiece cord


Clean the surface of the handpiece cord using a cleaning wipe with proven efficacy (Enzol 2%).

 Disinfect the surface of the cord using 70% isopropyl alcohol or other EPA recognized surface disinfectant, following carefully the instructions provided by the disinfection solution manufacturer.

-  The use of scouring powder or an abrasive sponge will damage its surface.
-  Do not submerge the cord in a cleaning solution bath. It would be likely to be damaged.
-  Make sure the handpiece connection of the cord is dry.

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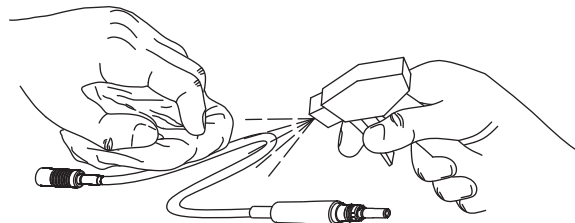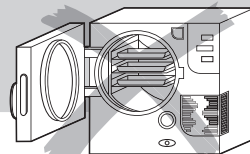
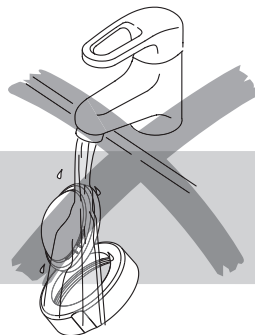
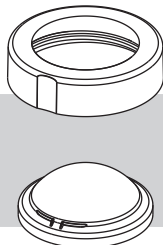
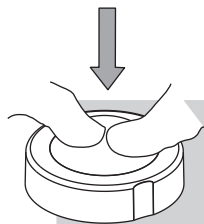
Cleaning the dome

Disassemble the dome to facilitate cleaning.

Regularly clean and disinfect the cap.

Clean the surface of the cap using a cleaning wipe with proven efficacy (Enzol 2%).

Disinfect the surface of the cord using 70% isopropyl alcohol or other EPA recognized surface disinfectant, following carefully the instructions provided by the disinfection solution manufacturer.

The ribs of the dome must be aligned with the lugs of the ring. The two pieces must be correctly aligned and interconnected.

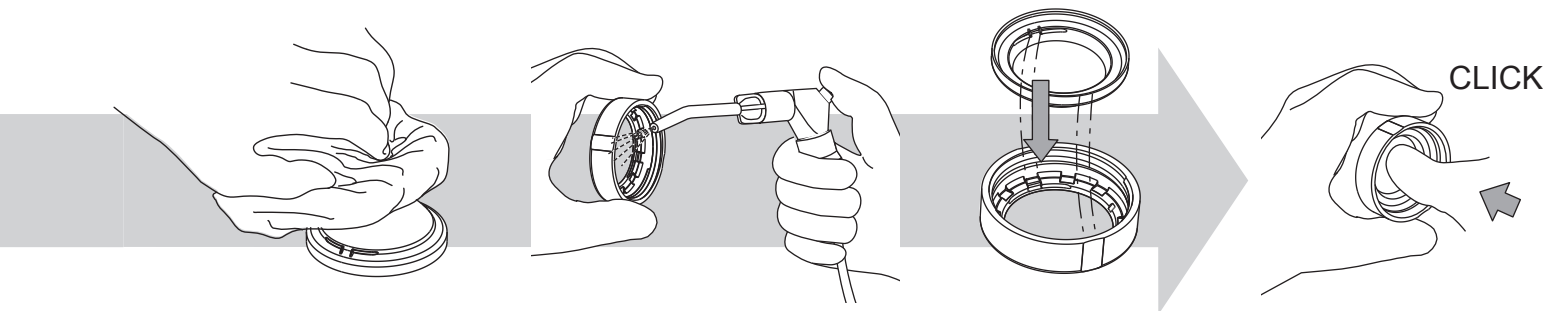
The cap must be absolutely dry before closing.

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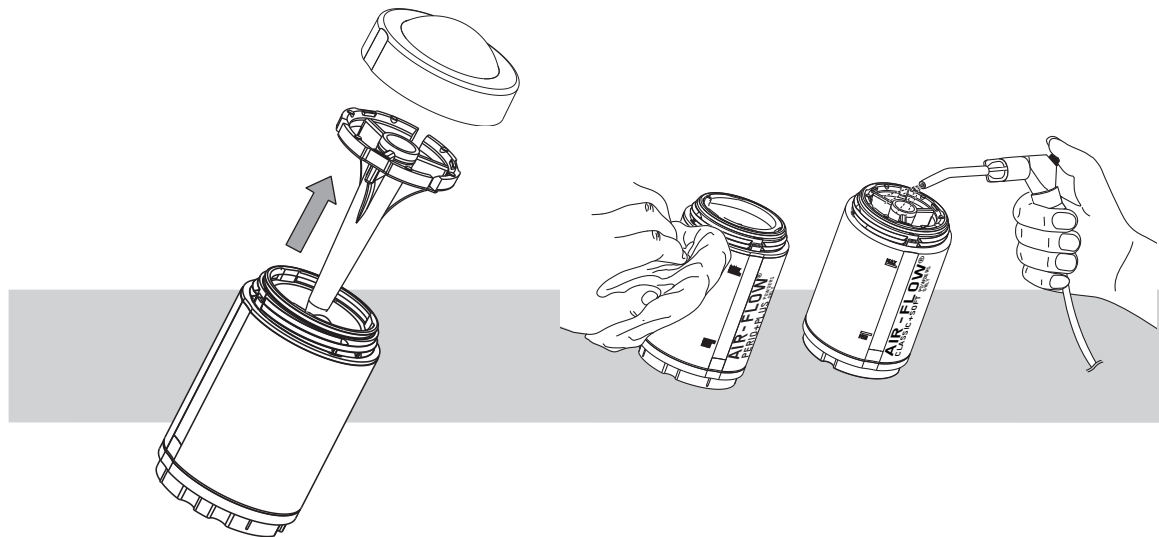
Cleaning the Powder Chamber


Unscrew the dome and pull out the powder diffuser.

Regularly clean this part.

- ⚠ The cap and the powder diffuser must be absolutely dry before reassembling.
- ⚠ Do not switch both powder diffusers during the reassembling.
- ⚠ Do not use needles to clean the powder chamber valve (risk of perforation)

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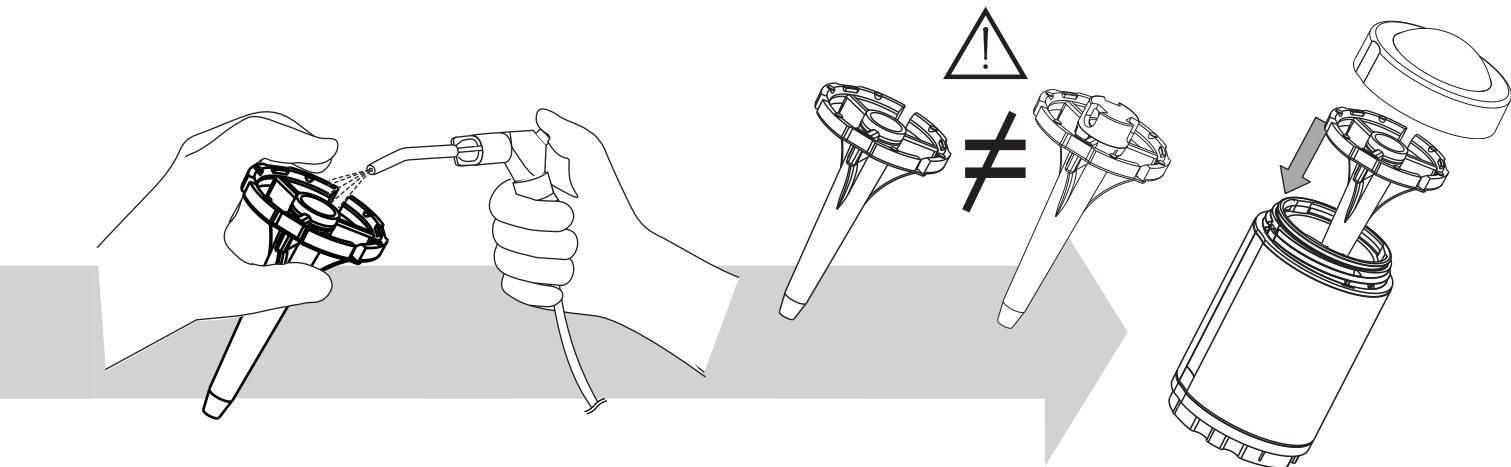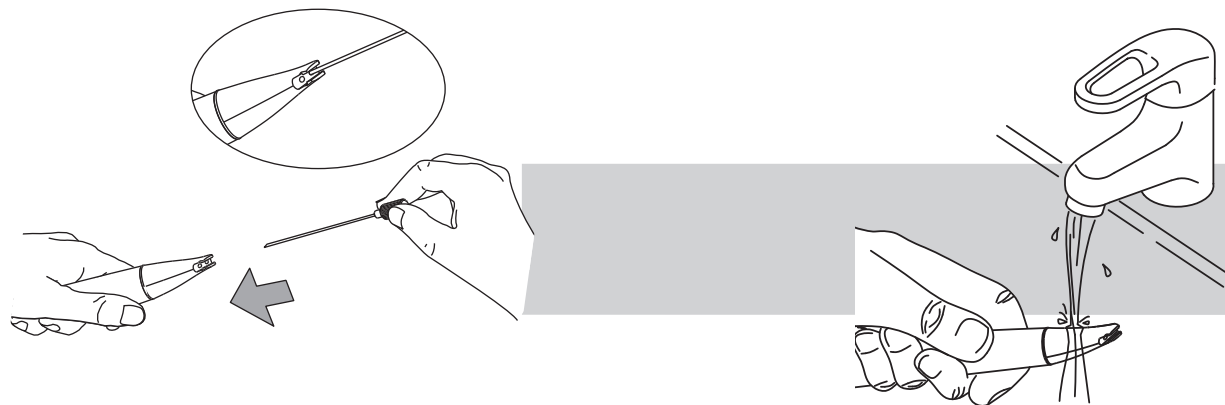
Unclogging the handpiece


Unclog the handpiece with the needle supplied with the unit.

Blow compressed air inside the handpiece to dry it.

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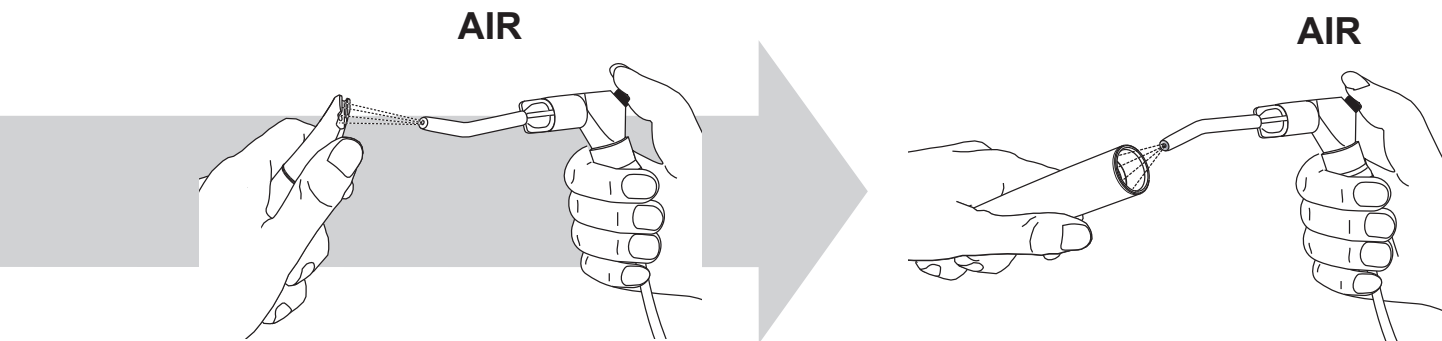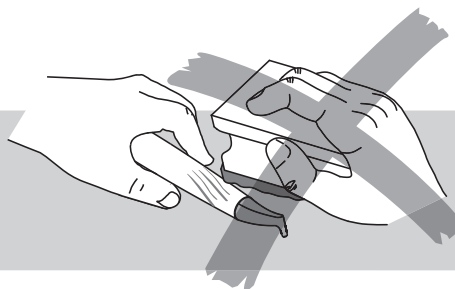
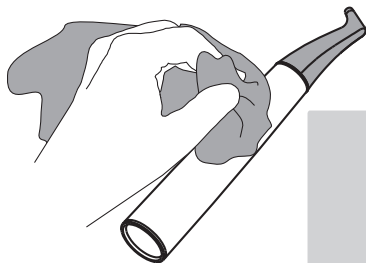
Cleaning the handpiece


Cleaning steps:

- Remove any externally attached soiling by brushing carefully with a soft surface brush or cloth.
- Soak all parts in the cleaning solution for, as a minimum, the time and the concentration specified by the manufacturer of the cleaning agent (ENZOL, 2 %, 10 min).
- Brush all lumens and surfaces with a suitable lumen / surface brush until all visible soiling is removed.
- Rinse through all product lumina (e.g. irrigation and aspiration connection) with the cleaning solution (ENZOL, 2 %), at least 5 times in the flow direction (no back rinsing), using a disposable syringe (min. volume 50 ml).
- Submerge all components completely in deionized water.
- Rinse through all product lumina (e.g. irrigation and aspiration connection) with deionized water at least 3 times in the flow direction (no back rinsing), using a disposable syringe (min. volume 50 ml).
- Repeat the cleaning process if the last rinse does not run clear, or if stains are still visible on the product.
- Dry all components at room temperature.
- Place the handpiece in a sterilization bag.

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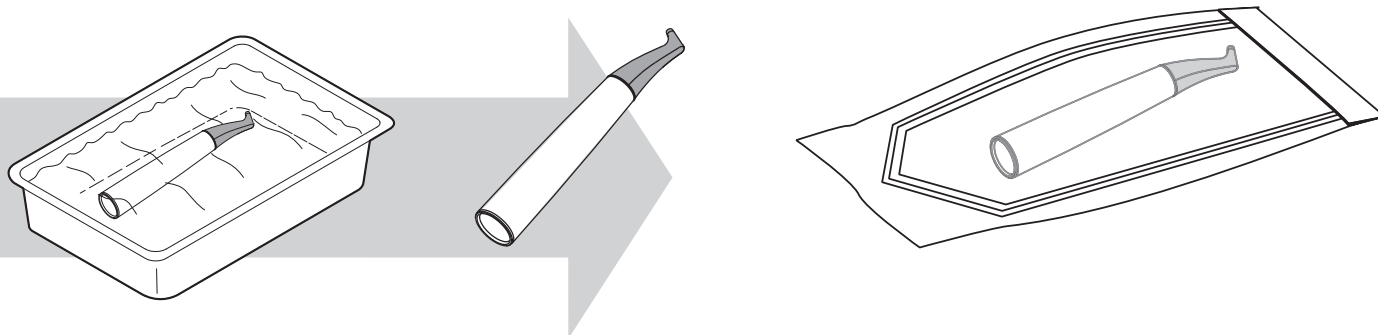
Sterilization of the handpiece and the cleaning needles

Packaging for Sterilization

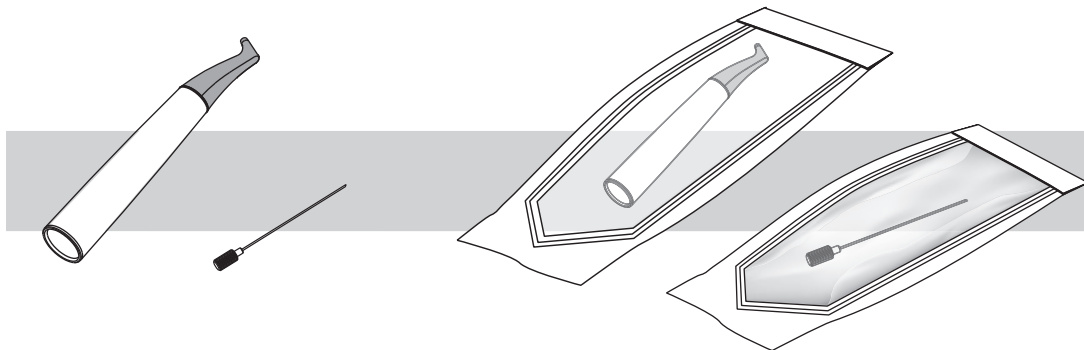Refer to standards and validated cycles

The accessories should be bagged in a sterilization pouch that is validated for use in a 132°C pre-vacuum steam sterilization cycle.

The sterilization pouch should be sealed according to the manufacturer's instructions.

Sterilization

Hu-Friedy & EMS recommend sterilization in a fractional pre vacuum cycle at 132°C for 4 min and a drying time of minimum 20 min. Hu-Friedy & EMS recommend use of sterilization pouches, biological indicators and chemical indicators that are legally marketed and have been validated for use in a 132°C 4 minutes pre-vacuum steam sterilization cycle.

Do not exceed the maximum number of sterilization cycles (more information in the technical data section of this manual).

The instrument may never be heated up >138°C.

Reprocessing procedures have only limited implications to a medical instrument. The limitation of the numbers of reprocessing procedures is therefore determined by the function / wear of the device.

The reprocessing must be carried out immediately, no later than 30 minutes after the completion of the operation. Additional information, where necessary, is provided in the respective product-specific usage instructions.

In case of damage the device should be reprocessed before sending back to the manufacturer for repair.

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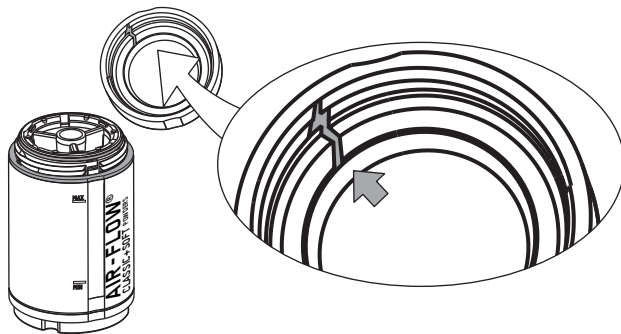
MAINTENANCE

Replacing the dome


 Check the condition of the threads on the powder chamber and cap. The powder chamber is pressurized during use. The condition of the powder chamber and cap (ring and dome) is an important safety factor.

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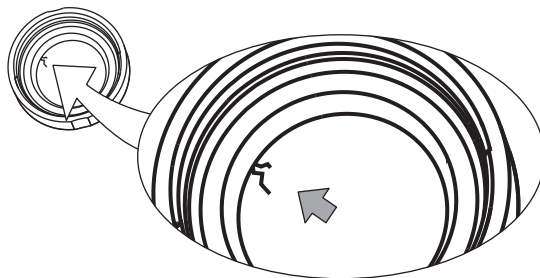
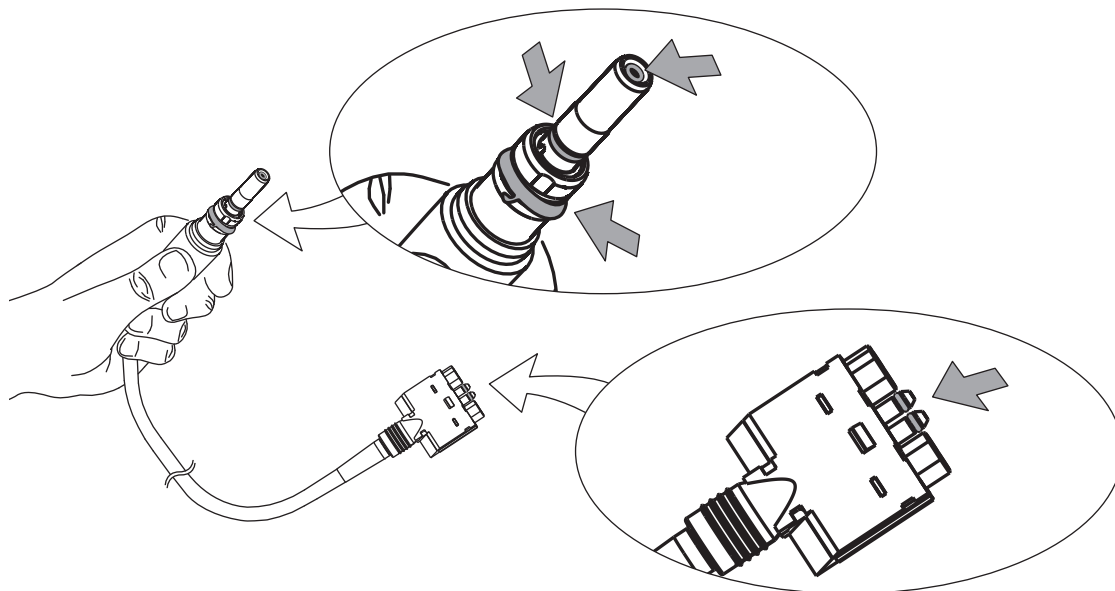
O- ring


 If an O-ring is damaged or worn out, it must be replaced immediately.

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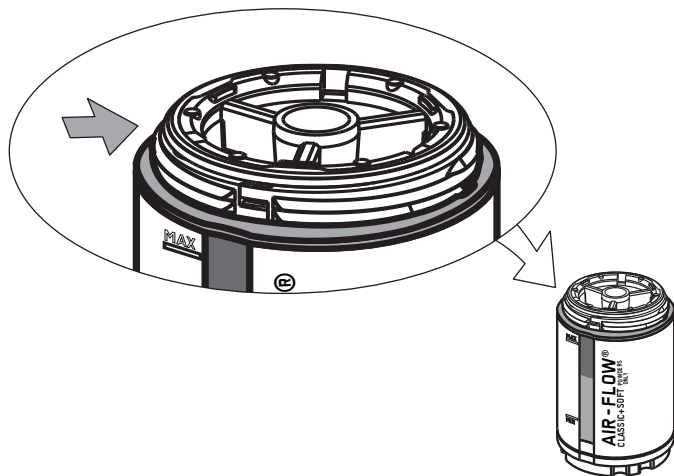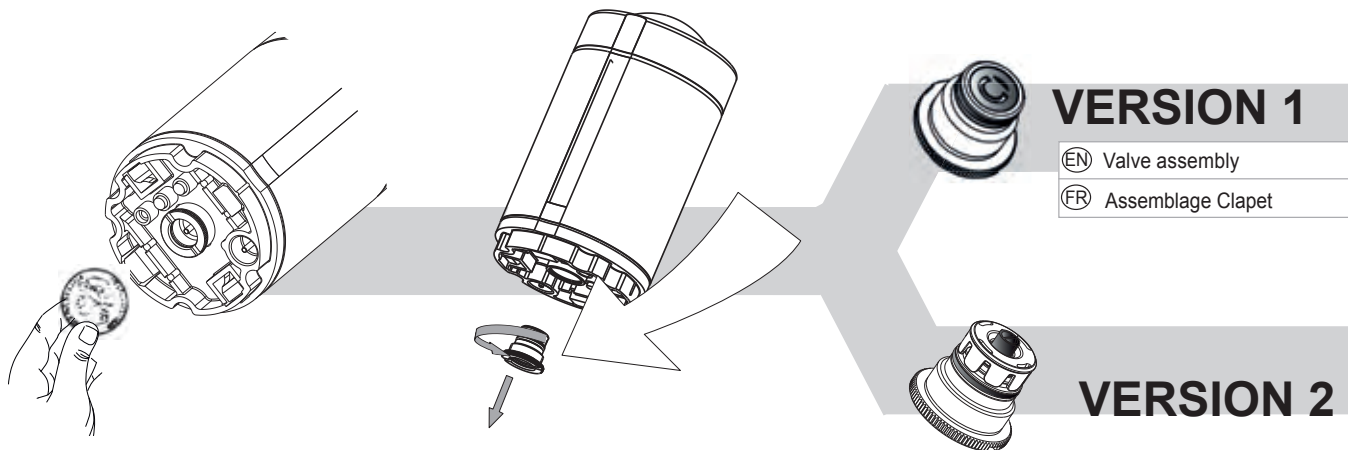Valve assembly


⚠ If the valve is damaged or worn out, it must be replaced immediately.

To replace the powder chamber valve:

- Do not turn the powder chamber upside down.
- Unscrew the valve assembly under the powder chamber with a coin, unclip the cap and extract the valve.
- Place the new valve in the correct position on the support and screw the assembly under the powder chamber.

⚠ If an O-ring is damaged or worn out, it must be replaced immediately.

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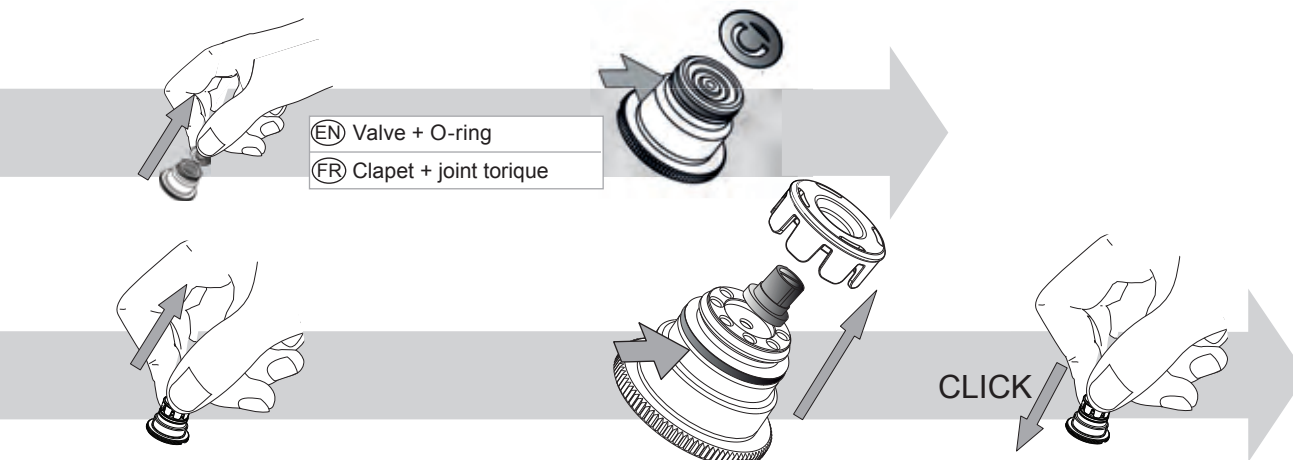
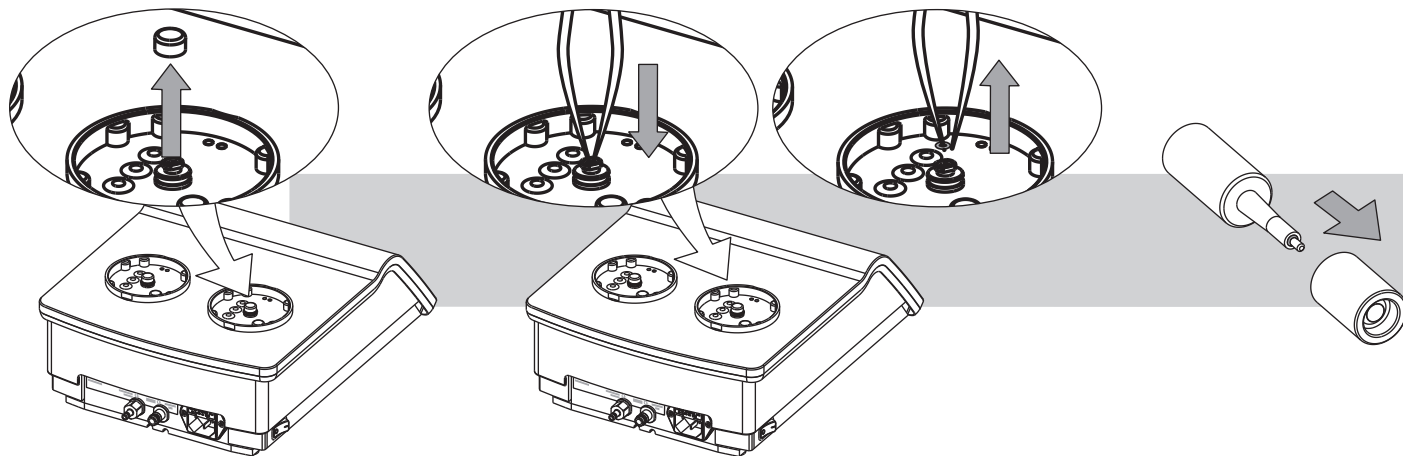
Replacing the O-rings of the connectors


 Use only plastic tweezers to remove the O-ring of the powder chamber holders.

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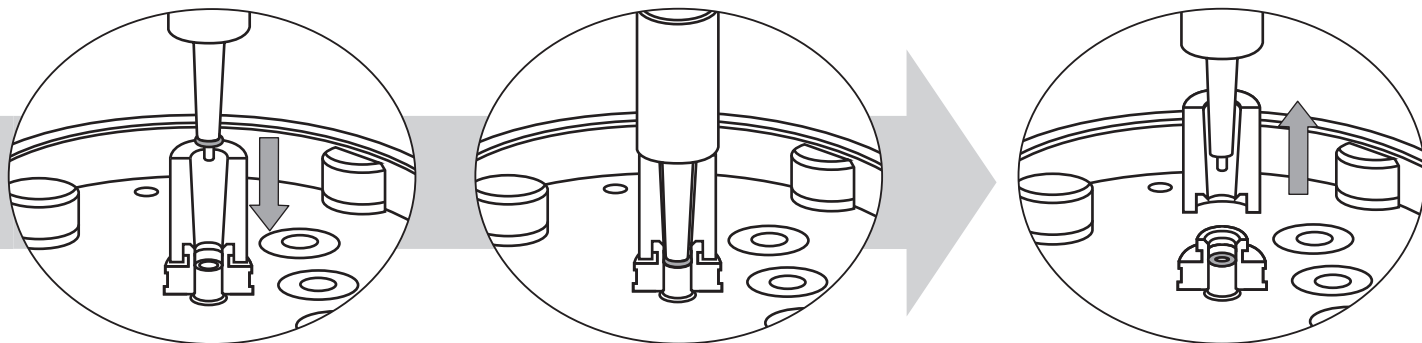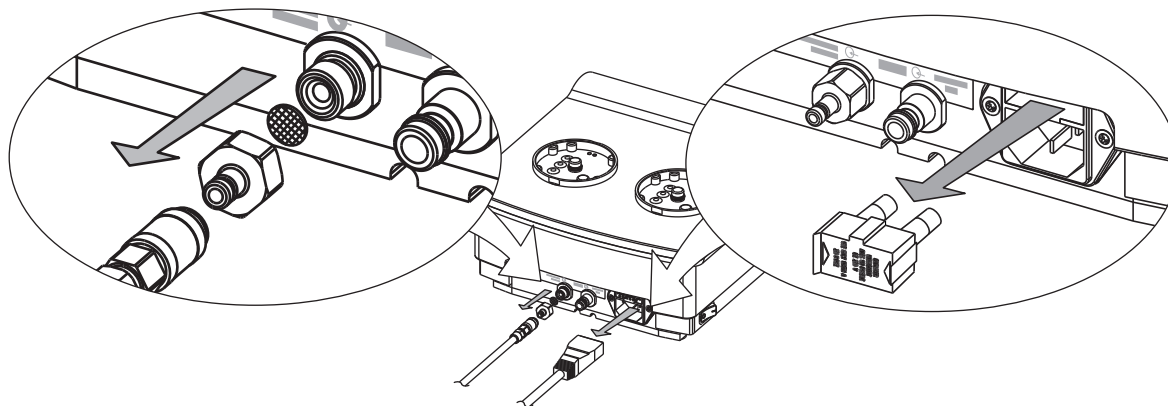
Replacing the water filter and the fuses

Regularly check the filter. Clean or change the filter if necessary.

 Replace exclusively with the type of fuse indicated on the back of the unit.

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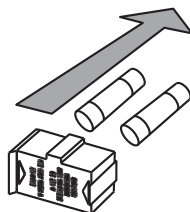
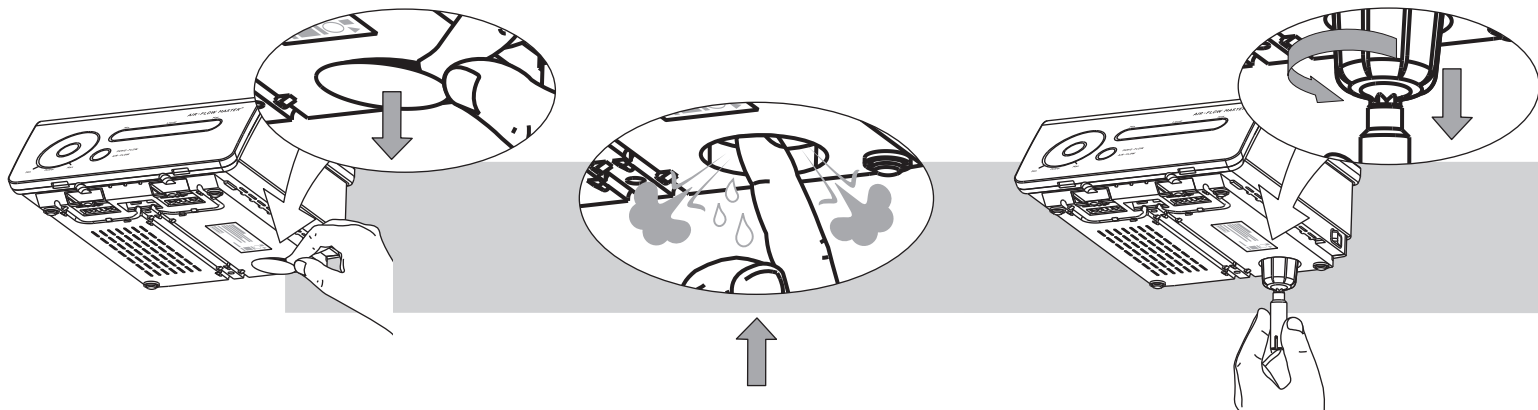
Air filter

Regularly control and purge the air filter.

The air filters are not reusable. Always install a new Hu-Friedy & EMS filter.

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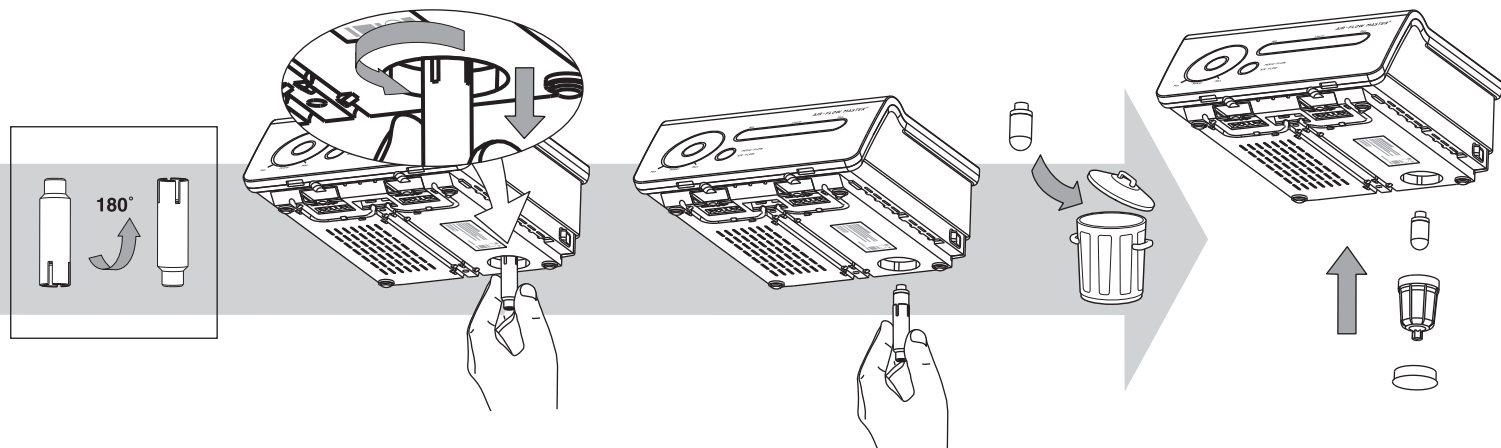SAFETY PRECAUTIONS

Hu-Friedy & EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 Use for the intended purpose only: before using the product, make sure that you have studied the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 Risk of explosion: do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel.

 Before each use, clean, disinfect and sterilize the different pieces and accessories of this product.

 Use only the powder recommended by Hu-Friedy & EMS for the subgingival application when the PERIO-FLOW handpiece is connected to the unit. Any other powder could cause premature wear of the nozzle and would consequently increase the risk of emphysema and damage to the unenameled dentine.

 This product has been investigated with regard to safety from electrical shock and fire hazard. CSA (Canadian Standard Association) has not investigated physiological effects.

 The powders provided by Hu-Friedy & EMS are specially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

 Never use an abrasive powder i.e. based on silica or alumina. This would damage the unit.

 Close the powder chamber after use, risk of device obstruction

 Always examine the product for damage before starting treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original Hu-Friedy & EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved Hu-Friedy & EMS repair center.

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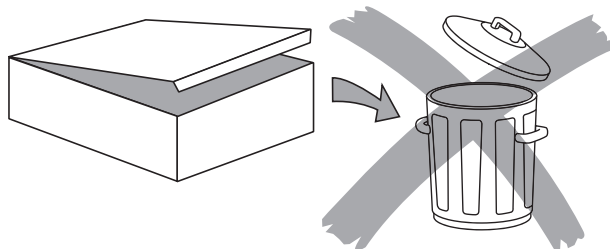
STORING THE PRODUCT

 Keep the original packaging until the product is to be disposed of permanently. It can be used for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the chapter "Cleaning, disinfecting and sterilizing",
- Remove the screw of the water filter to empty the unit,
- Pack the product and all accessories in the original packaging.

 The powder should not be stored near acids or heat sources.

Please refer to the bottles of powders for the transport and storage conditions.

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PRODUCT DISPOSAL

The product must not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

Waste Electrical and Electronic Equipment belonging to customers located in the European Union may be shipped to Hu-Friedy & EMS for recycling in accordance to the WEEE regulations. The costs of recycling, exclusive of shipping fees, are covered by Hu-Friedy & EMS.

WARRANTY

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from Hu-Friedy & EMS or any authorized dealers. Please contact your customer service directly.

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HU-FRIEDY & EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or to an authorized Hu-Friedy & EMS Repair Center.

In the case of non-authorized repairs or damages due to non-adherence to the operating instructions, Hu-Friedy & EMS accept no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operating instructions.

When sending your product directly to the approved Hu-Friedy & EMS repair center, please include the name of your dealer. This simplifies processing for us.

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TECHNICAL DATA

Description

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	AIR-FLOW MASTER®
Classification EN 60601-1	- Class I - Applied part, Type BF - IP 20 unit - IP X1, foot pedal
Classification 93/42/EEC	Class IIa
Operating mode	Continuous operation
Supply voltage	100-240 VAC 50-60Hz
Power consumption	max. 100 VA
Fuses	T 1.6 A 250 VAC (Ø5 X 20mm, time lag, low breaking capacity)
Water supply	1 to 5 Bar (100 to 500 kPa)
Compressed air supply	5.5 to 7.5 Bar (550 to 750 kPa)
Weight (approx.)	5.0 kg
Dimensions in mm (HxWxD)	225 x 225 x 295
Operating conditions	+10°C to +40°C 30% to 75% relative humidity Altitude max. 3000 m
Validated life time of the handpiece	500 sterilization cycles
Storage and transport conditions	-10°C to +40°C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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ELECTROMAGNETIC COMPATIBILITY

 This product needs special precautions regarding electromagnetic compatibility and needs to be installed and put into service according to the note "Electromagnetic compatibility".

 Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the note "Electromagnetic compatibility" must be taken into account.

 This product should not be used adjacent to or stacked with another unit. If adjacent or stacked use is necessary, this product should be observed to verify normal operation in the configuration in which it will be used.

 The use of accessories and cables other than those specified or sold by Hu-Friedy & EMS as replacement parts may result in increased emissions or decreased immunity of this product.

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TROUBLESHOOTING

Symptom	Solution
The unit does not start or stop	• Check the electrical connections and power supply at the mains • Check the fuses on back panel • Check main switch function: at least one LED must be visible on adjustments and working mode areas • The unit is in "standby" mode, tap on any sensitive zone to exit standby • Send the unit to an authorized Hu-Friedy & EMS Repair Center
The foot pedal does not activate the unit	• Check that the foot pedal is properly connected • Replace the foot pedal • Send the unit to an authorized Hu-Friedy & EMS Repair Center
The unit is still activated when the foot pedal is released	• The foot pedal is active, check for jam at foot pedal • The switch in the pedal is blocked. Turn the unit off at the power switch • Replace the foot pedal • Send the unit to an authorized Hu-Friedy & EMS Repair Center
User interface displays an error: there is one acoustic signal and all LEDs are blinking	• Powder chamber pressure is too low or too high • The air or liquid regulator is switched off • Foot pedal and purge commands are not recognized by system. Check connections
The handpiece does not remain on the magnetic support	• Install the magnetic support on a metallic part of the unit • The support is demagnetized • Replace magnetic support
The handpiece does not hold on the handpiece cord	• Check for presence of the O-ring seal and replace if necessary • Replace the handpiece • Replace the handpiece cord • Send the handpiece and the handpiece cord to an authorized Hu-Friedy & EMS Repair Center
No water jet coming from the handpiece	• Check the water supply and water filter • Check the water adjustment of your unit • Check the connections between the unit, the cord and the handpiece • Check for presence of water at the cord outlet port • Send the handpiece to an authorized Hu-Friedy & EMS Repair Center
No powder/air jet coming from the handpiece	• Check the quantity of powder in your unit • Check the compressed air supply and air filter • Check for presence of powder/air at cord outlet port • Eliminate residual powder present in the channels using the cleaning needles provided • AIR-FLOW PERIO+ PLUS powder chamber only: remove the diffuser and empty the powder. Blow the diffuser with compressed air before installing it again and refilling. • Send the system to an authorized Hu-Friedy & EMS Repair Center

Water leakage between the handpiece cord and the handpiece	• Check the condition of the handpiece connecting O-rings • Send the system and the handpiece cord to an authorized Hu-Friedy & EMS Repair Center
Water leakage between the handpiece and the nozzle	• Verify the position of the nozzle and its position on the handpiece • Verify that no residual remains between the nozzle and the handpiece • Verify that nothing affects the alignment or prevents fixing the nozzle on its holder. • Replace the nozzle • Replace the handpiece • Send the system and the handpiece cord to an certified Hu-Friedy & EMS Repair Center
Air and powder leakage on the main unit	• Verify the positioning of the bottles • Check the O-rings under the silicone caps of the powder chamber holders • Verify the cap of the powder chamber and the O-ring positioning • Verify the check valve and the O-ring under the powder chamber • Remove all powder dust between the bottle holder and the powder chamber • Send the unit to an authorized Hu-Friedy & EMS Repair Center
Air and powder is leaking at powder chamber	• Check the seal condition and the cleanness of the thread of the powder chamber cap • Replace the seals if necessary • Clean the sealing surfaces carefully • Replace the powder chambers • Send the unit to an authorized Hu-Friedy & EMS Repair Center
Handpiece efficiency becomes weak or decreases during use	• Verify the quantity of powder in your unit • Eliminate residual powder present in the channels using a cleaning needle • Send the system and the handpiece cord to an certified Hu-Friedy & EMS Repair Center
Leakage on the unit water supply	• Check the water supply connection and inlet pressure • Send the unit to an authorized Hu-Friedy & EMS Repair Center

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Hu-Friedy Mfg. Co., LLC
EMS Electro Medical Systems S.A.