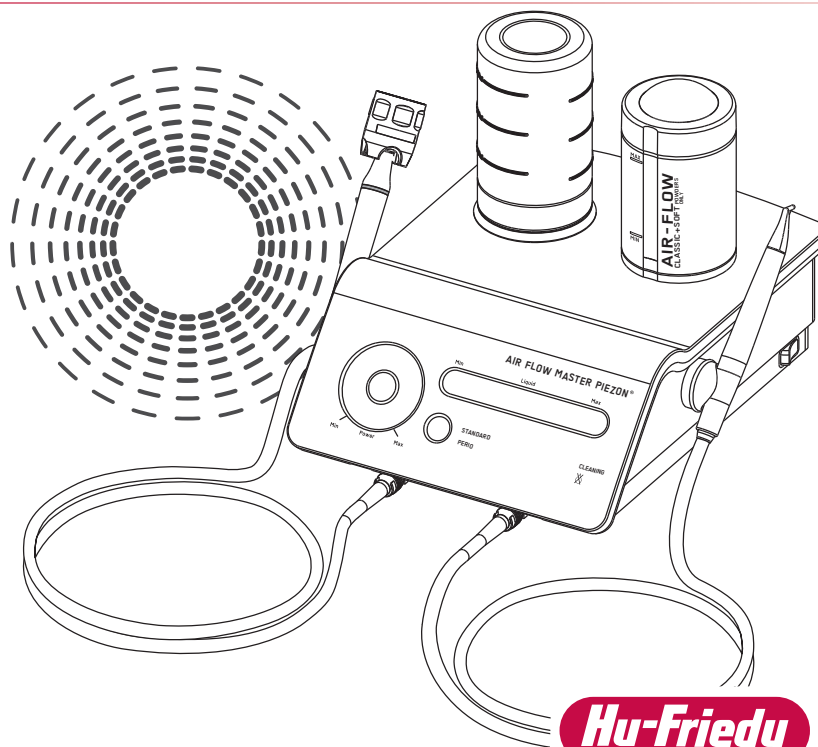
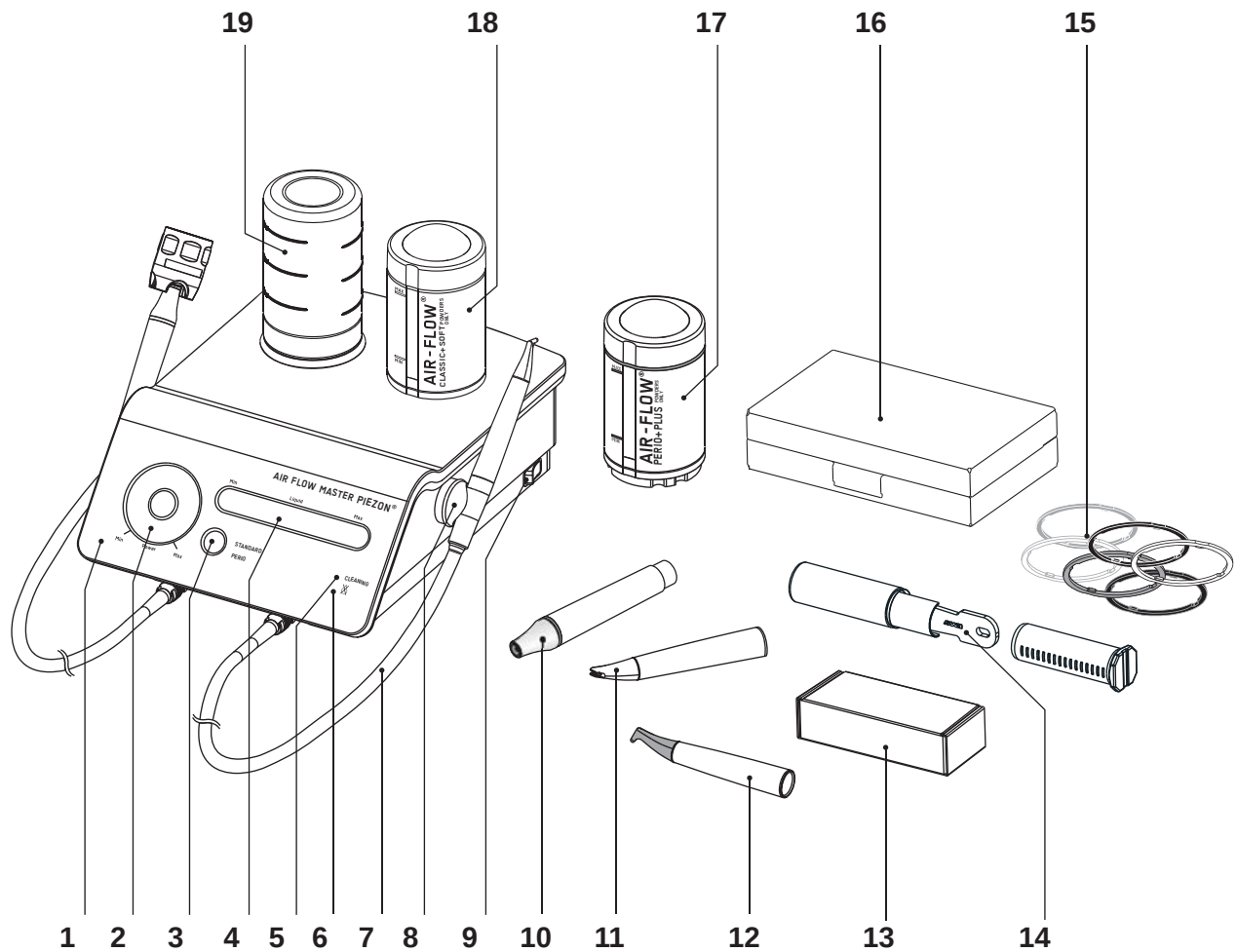
AIR-FLOW MASTER PIEZON®

OPERATION INSTRUCTIONS

MODE D'EMPLOI

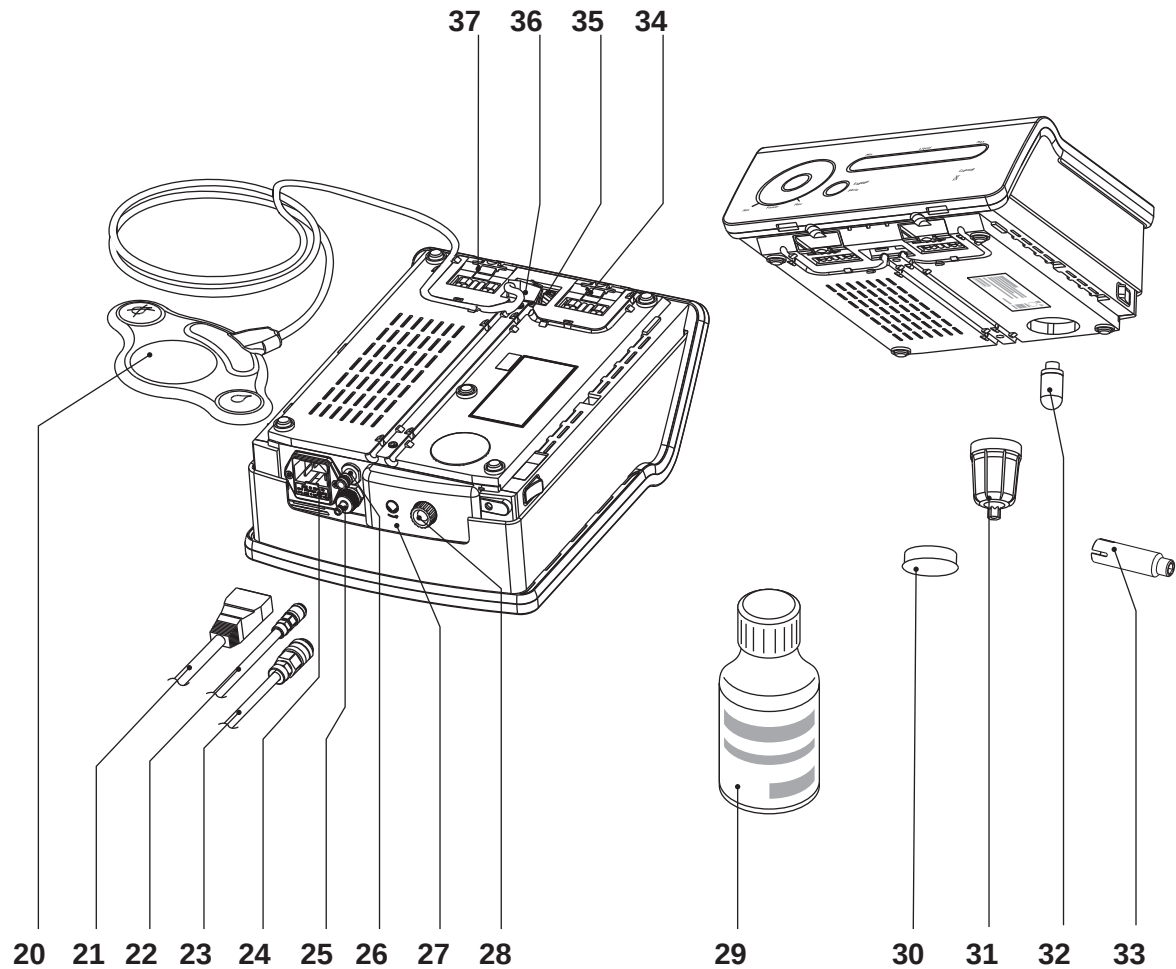
COMPONENTS

- 1 Unit
- 2 Power setting
- 3 Working mode selection
- 4 Flow rate adjustment
- 5 "Cleaning" mode control lamp
- 6 "Dry Work" control lamp
- 7 Handpiece cord
- 8 Magnetic handpiece support
- 9 Purge switch
- 10 Piezon® handpiece
- 11 PERIO-FLOW® handpiece
- 12 AIR-FLOW® handpiece
- 13 Box of PERIO-FLOW® nozzles
- 14 PERIO-FLOW® Nozzle extractor
- 15 Powder chamber rings set
- 16 SystemBox
- 17 AIR-FLOW® PERIO+PLUS Powder chamber
- 18 AIR-FLOW® CLASSIC+SOFT Powder chamber
- 19 Bottle of irrigation liquid

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- 20 Foot pedal
- 21 Power cable
- 22 Compressed air supply hose
- 23 Water supply hose
- 24 Fuses
- 25 Water hose connector
- 26 Air hose connector
- 27 Unit pump cover
- 28 Screw of the unit pump cover
- 29 AIR-FLOW® Powder bottle
- 30 Air filter housing cap
- 31 Air filter housing
- 32 Replacement air filter wrench
- 33 Filter cartridge
- 34 Connector for AIR-FLOW® or PERIO-FLOW® handpiece cord
- 35 Connector for connecting a second Hu-Friedy & EMS unit
- 36 Connector for the foot pedal
- 37 Connector for Piezon® handpiece cord

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Hu-Friedy & EMS supply this unit with various accessories. The "Packing list" shows exactly what your unit includes.

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TECHNICAL MODIFICATIONS

Hu-Friedy & EMS reserve the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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DEAR CUSTOMERS,

Thank you for buying a new Hu-Friedy & EMS product. It meets the highest quality and safety standards.

Your AIR-FLOW MASTER PIEZON is a combination of one air polishing unit and one Piezon system.

It is for use in dentistry with the Hu-Friedy & EMS Piezon ultrasound instruments and the Hu-Friedy & EMS AIR-FLOW systems.

This product works exclusively with the powders and the systems supplied by Hu-Friedy & EMS.

The instructions in this manual cover the combined use of an AIR-FLOW handpiece or a PERIO-FLOW handpiece and Piezon handpiece. Please refer only to instructions related to AIR-FLOW handpiece when a PERIO-FLOW handpiece is not present.

For further details concerning the treatments, please refer to the instruction which are supplied with your AIR-FLOW or PERIO-FLOW or Piezon systems. Please do not hesitate to contact your local Hu-Friedy & EMS dealer for further information.

Contra-indications:

Please refer to operation instructions of the related system.

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Piezon Systems

For further details on the various instruments and their scopes of application, please refer to the operating instructions supplied with your Piezon system or available from the distributor in your country.

Applications

This product is designed for use in dentistry with Hu-Friedy & EMS Piezon ultrasound instruments for the following indications:

- Removing supra and subgingival calculus deposits and stains from teeth
- Periodontal pocket lavage with simultaneous ultrasonic tip movement
- Scaling and root planing
- Releasing crowns, bridges, inlays, and posts as well as condensing gutta percha
- Plugging for amalgam condensation
- Amalgam burnishing
- Preparing, cleaning and irrigating root canals
- Cavity preparation
- Cementing inlays and onlays
- Retrograde preparation of root canals

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AIR-FLOW systems

The confinement of the powder by a water jet makes it possible to steer the water jet with great precision and thus ensures a pleasant treatment for your patient.

This product removes dental plaque, soft deposits and surface stains from pits, grooves, interproximal spaces or smooth surfaces of the teeth.

Applications

- Plaque removal for placement of sealants,
- Surface preparation prior to bonding/cementation of inlays, onlays, crowns and veneers,
- Surface preparation prior to placing composite restorations,
- Cleaning prior to bonding of orthodontic brackets,
- Effective plaque and stain removal for orthodontic patients,
- Clean implant fixture prior to loading,
- Stain removal for shade determination,
- Plaque removal prior to fluoride treatment,
- Plaque and stain removal prior to whitening procedure.

The AIR-FLOW MASTER PIEZON is also intended for use as an air-polisher in patients suffering from periodontal disease. The AIR-FLOW MASTER PIEZON is indicated for the non-surgical removal of subgingival plaque in pockets up to 5 mm after initial periodontal treatment.

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PERIO-FLOW system

Applications

This product must only be used by a qualified and trained personnel (dentist or dental hygienist).

This product should only be used with powders supplied by Hu-Friedy & EMS for a subgingival application.

This product removes the subgingival biofilm during periodic periodontal treatments. Its long fine nozzle was specifically developed for easy access inside the periodontal pocket.

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ABOUT THIS MANUAL

These operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this document close at hand.

Instructions concerning the components of the system are provided separately in the respective instructions for use. Read the instruction manuals originally delivered with each of the components before using this product. Do not hesitate to contact your local Hu-Friedy & EMS dealer for further information.

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 To prevent possible patient or user injury or damage to the AIR-FLOW MASTER PIEZON, please note the corresponding directives:

Danger

Risk of patient or user injury

Caution

Risk of damage to the device or environmental harm

Please note

Useful additional information and hints

Prohibited

Authorized

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please contact your approved Hu-Friedy & EMS repair center or your dealer.

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Hu-Friedy & EMS

CONTENTS


Installation	2	Switching On	64
Foot pedal	2	Working mode selection.....	66
Handpiece cords and supports.....	4	Multifunction foot pedal.....	68
Connecting to the water supply and to the compressed air supply	6	Setting the air/powder flow rate	70
Connecting to the mains power	8	Setting the water flow rate	72
Connecting to another Hu-Friedy & EMS unit (option)	12	Purge switch and mode.....	74
Piezon System	14	Powder exchange or refill.....	76
Preparing for the Piezon treatment.....	14	End of the treatment.....	78
Solutions compatible with the circuit	16	PERIO-FLOW System	80
Connecting the handpieces	18	Installing the nozzle and connecting the PERIO-FLOW handpiece.....	80
Attaching the instruments	20	PERIO-FLOW mode	82
Switching On	22	Cleaning, disinfecting and sterilizing	84
Working mode selection.....	26	Cleaning the accessories.....	90
"Dry Work" mode selection.....	28	Cleaning the dome.....	92
Multifunction foot pedal.....	30	Cleaning the Powder Chamber.....	94
Piezon handpiece LED.....	32	Maintenance.....	96
Setting the ultrasound power.....	34	Replacing the dome	96
Setting the irrigation flow rate	36	O-rings	98
End of the treatment.....	38	Replacing the water filter and the fuses ..	102
Cleaning and disinfecting the circuit.....	40	Air filter	104
Illustration of a "Cleaning" mode cycle	44	Replacing the pump of the unit	106
Removing the bottle	46	Safety precautions	108
Powder chambers	48	Storing the product	112
Chambres à poudre	49	Product disposal	114
Supragingival treatment using the AIR-FLOW CLASSIC+SOFT powder chamber.....	50	Warranty.....	114
Supragingival treatment using the AIR-FLOW PERIO+PLUS powder chamber	52	Accessories.....	114
Subgingival treatment using the AIR-FLOW PERIO+PLUS powder chamber	54	Hu-Friedy & EMS-Service	116
Connecting the AIR-FLOW handpiece	62	Symbols	118
Operating the unit.....	64	Technical data	126
		Electromagnetic compatibility.....	128
		Troubleshooting	130

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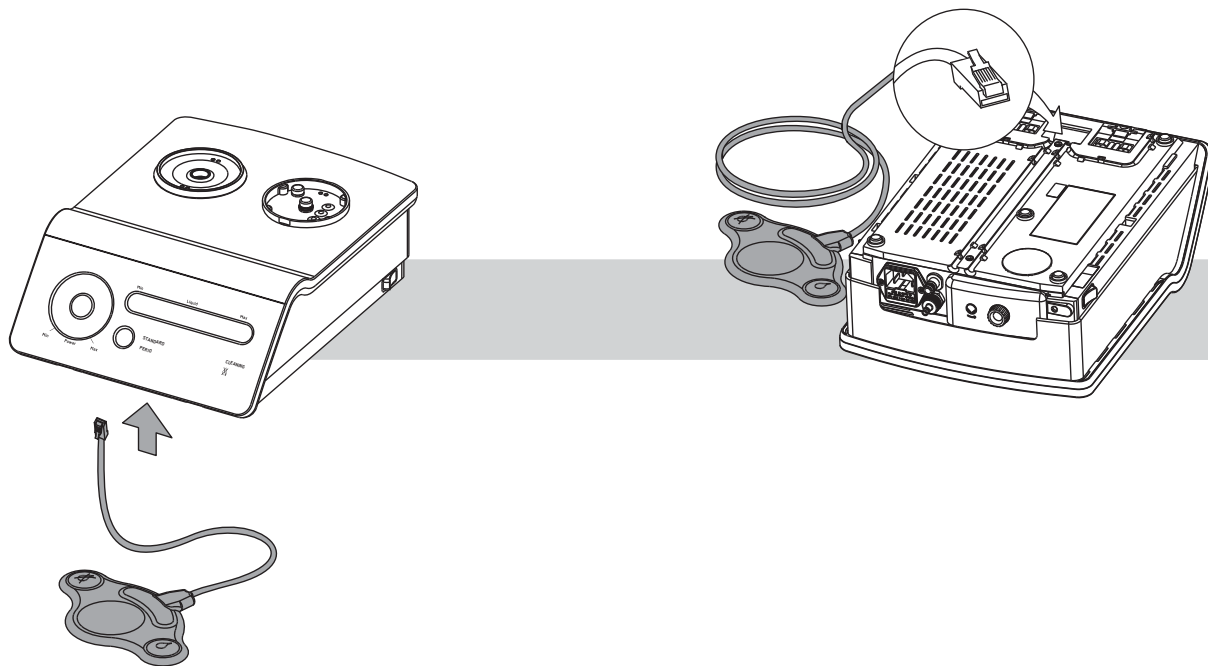
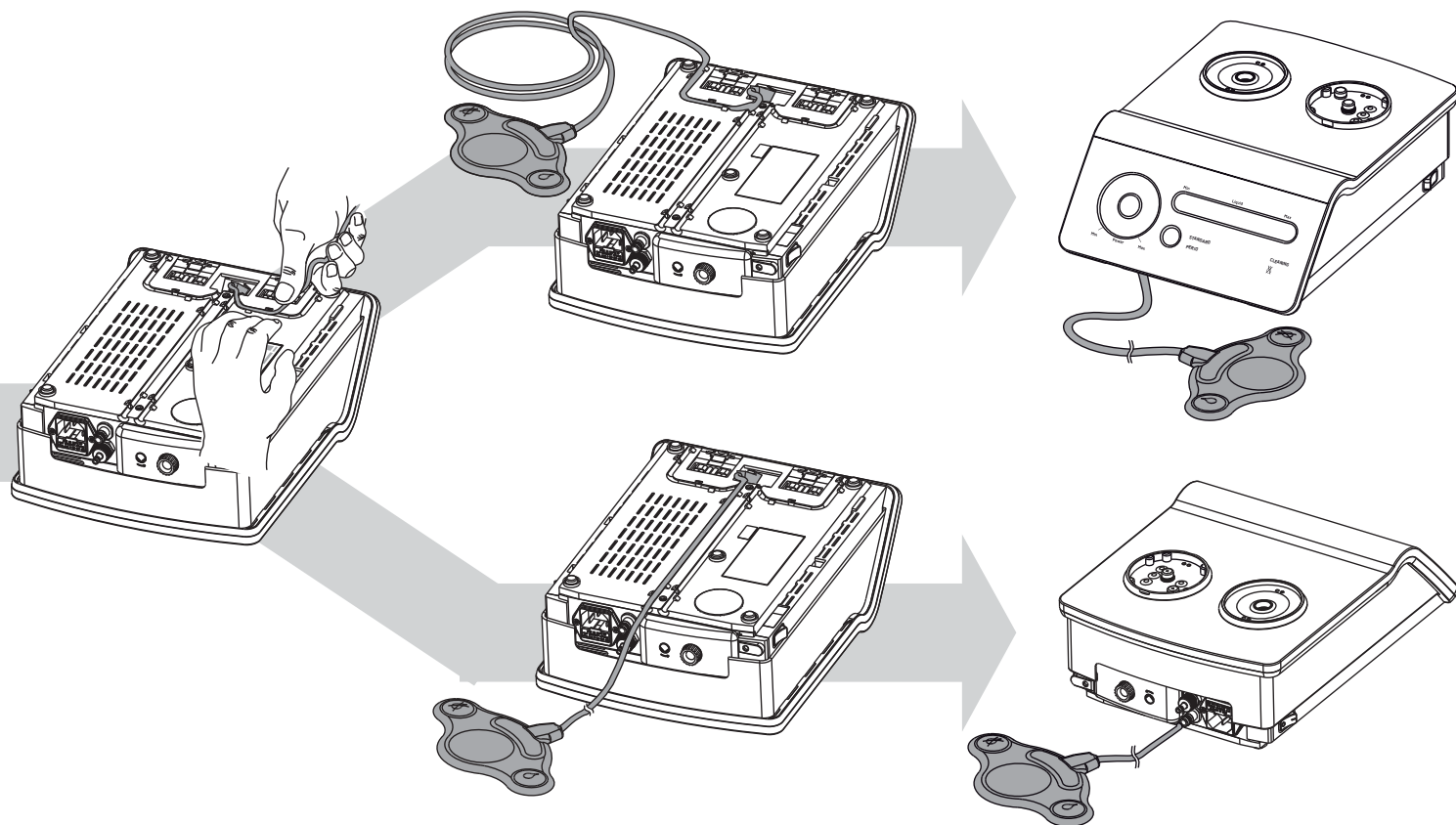
Pédale de commande

Handpiece cords and supports

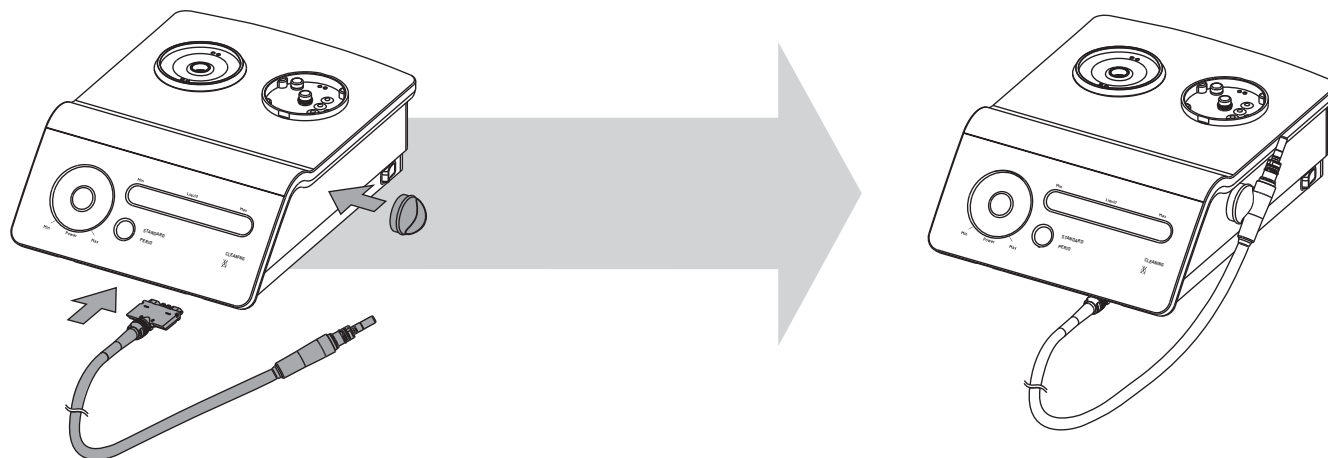
Check that the cords are fully dry before connecting to the unit.

Only place the magnetic handpiece holder on the metallic part of the unit or the furniture.

 Always check that the cords will not rub against the front face during the treatment because this could eventually modify the selected settings.

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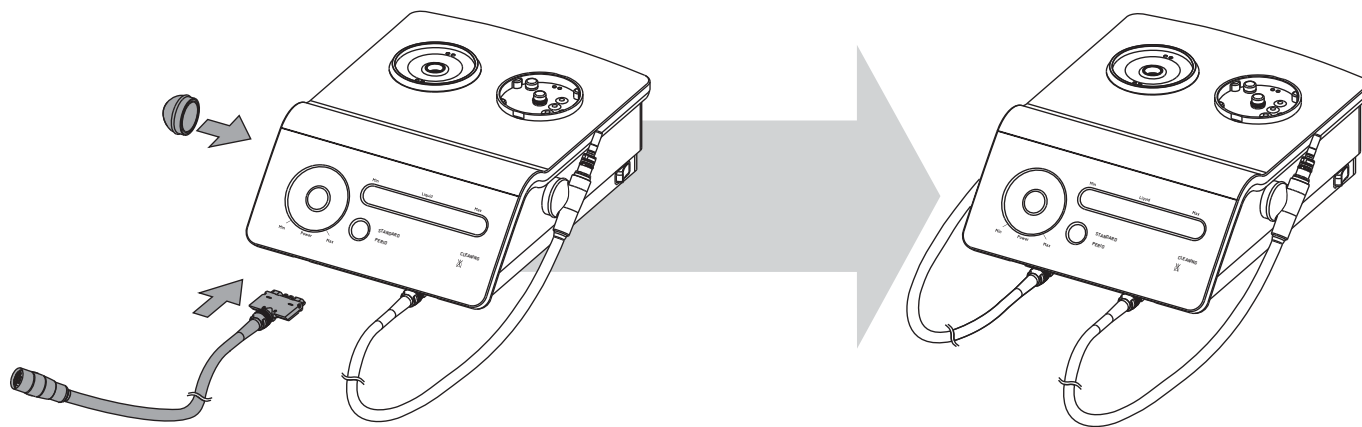
Connecting to the water supply and to the compressed air supply

H₂O

bar	1.0 - 5.0
psi	15 - 72
kPa	100 - 500

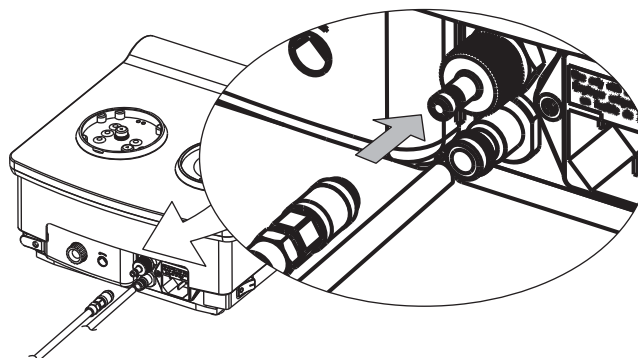Connect the unit to a water supply using the original Hu-Friedy & EMS water hose supplied with your product.


Use only dry and filtered compressed air.

 Check that the supply pressure of compressed air does not exceed the specified value (mentioned on the unit).

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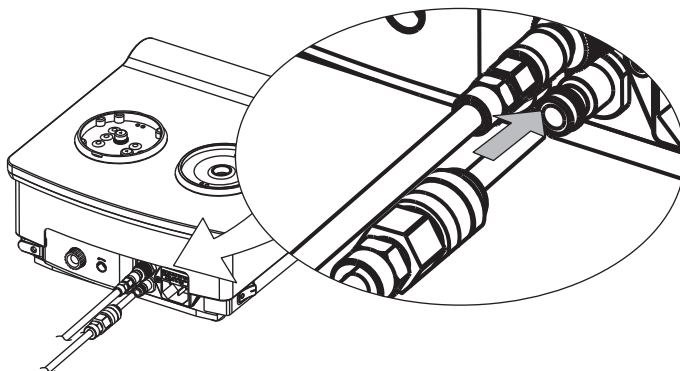
Connecting to the mains power

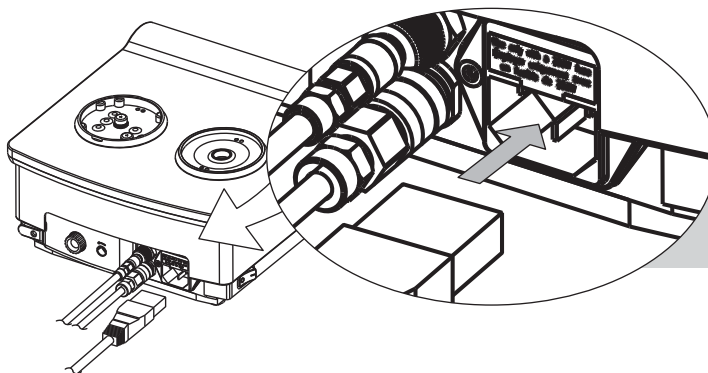100-240 VAC 50-60Hz


⚠ To prevent damage to the unit, check that its rated voltage corresponds to the local line voltage.

Connect the unit only to a FI protected mains supply (FI = Residual current protection).

👉 For USA or Canada: connect only to Hospital Grade receptacle.

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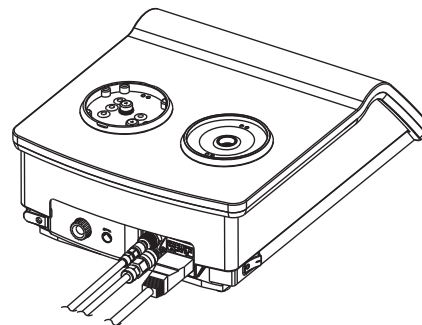Electrical Safety precautions

 To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

 No modification shall be made on this product.

 The main switch of the unit must be accessible at any time.

 When the main switch is on the "0" position, the unit is disconnected from the supply network.

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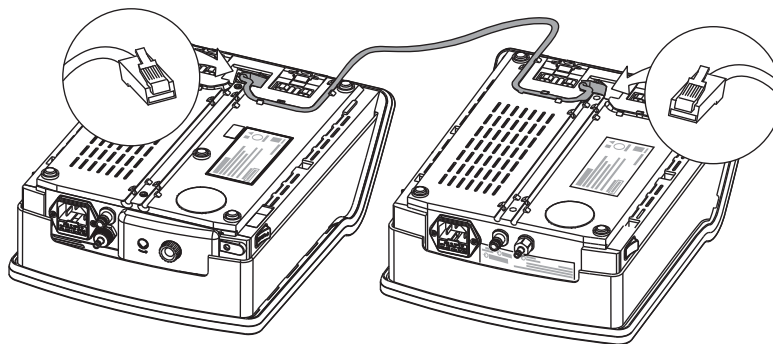
Connecting to another Hu-Friedy & EMS unit (option)

The AIR-FLOW Master Piezon can be connected to one other compatible Hu-Friedy & EMS device, so both devices can be operated using a single foot pedal. An optional link cable allows this connection. This option is only available with the "multi-function" pedal.

The pedal will trigger only the handpiece which is lifted from its support whatever the unit.

For additional details on device compatibility, please contact the distributor in your country.

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PIEZON SYSTEM

Preparing for the Piezon treatment

Installing the bottles

 Always check the condition of the bottle cap O-ring before installing the bottle. Please refer to the chapter "Maintenance".

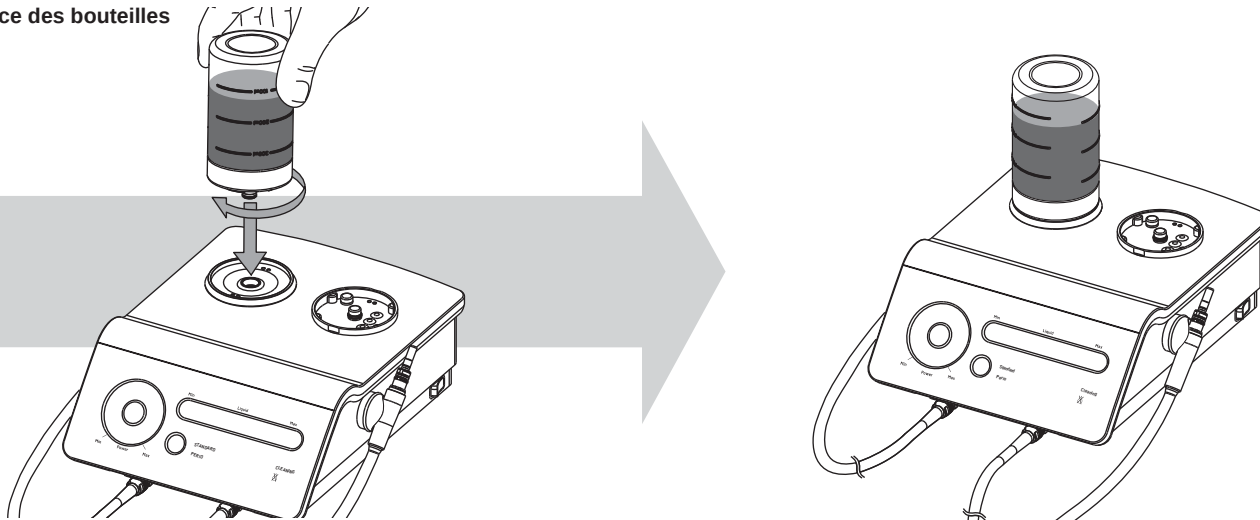
 Use the graduations provided on the bottle to adjust the quantity of the solution for the treatment.

Install the bottle using a rotating movement until it reaches the stop point.

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Solutions compatible with the circuit

Please read the instructions for use of the disinfecting products.

Please refer to standards applicable in your country regarding the purchase and use of these products.

Disinfecting solutions for the treatment

- Sodium hypochlorite ($\text{NaOCl} \leq 3\%$)
- Chlorhexidine ($\leq 0.3\%$)
- Physiological water ($\text{NaCl} \leq 0.9\%$)
- Hydrogen peroxide ($\text{H}_2\text{O}_2 \leq 3\%$)
- Carbanilides
- Povidone iodine
- Quaternary ammonium compounds
- Cetylpyridin chloride
- Citric acid
- Aromatic essence
- Ethanol

Examples of common products

BacterX® pro / Triclosan / Betadine® / Listerine® / Alodont® / Meridol® / BreathRx® / CloSYS™ / Dakin's solution / Scope® / Cepacol® / .

Disinfecting solutions for washing the circuit

- Hu-Friedy & EMS Ultra Clean (H_2O_2 6%)
- Hydrogen peroxide (H_2O_2) 5%
- Sterilox (hypochlorous acid)

Incompatible mixing

Different disinfecting solutions can be used during treatment. Chemical reactions might occur between these solutions and damage the unit. Wash the liquid circuit before using a different solution (please refer to the "Cleaning mode" in the "End of treatment" chapter).

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

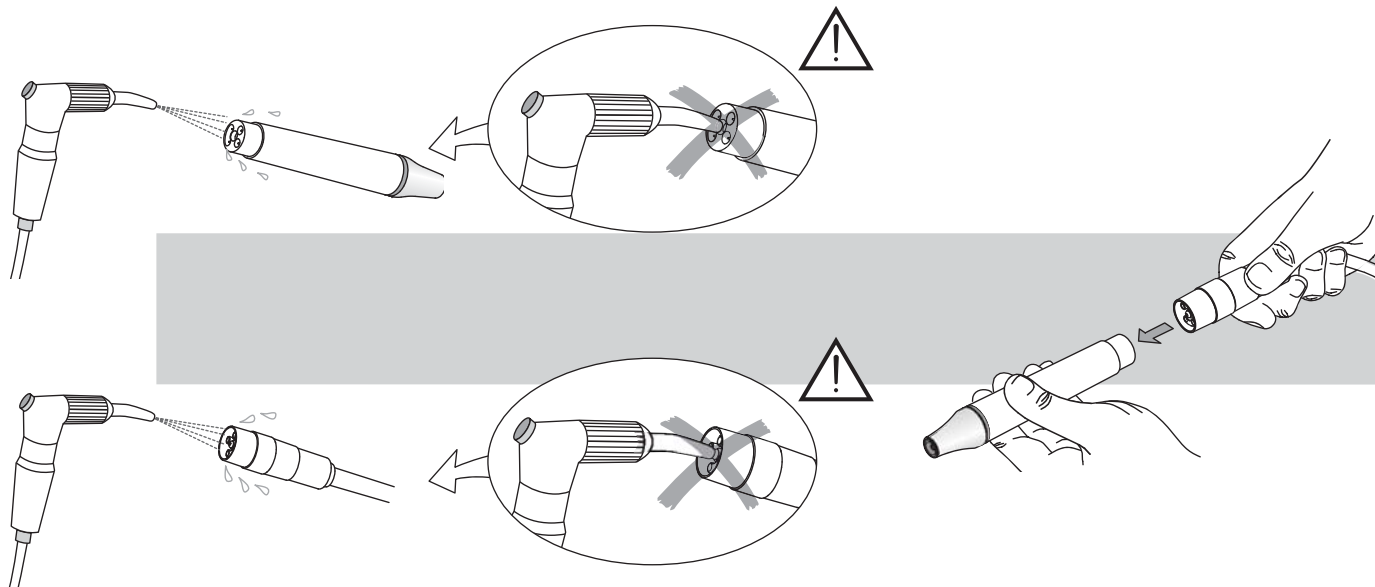
Connecting the handpieces


 Blow dry the connections to remove any presence of liquid and to ensure a proper electrical contact. Never blow compressed air in the irrigation connections as this will irretrievably damage internal parts.

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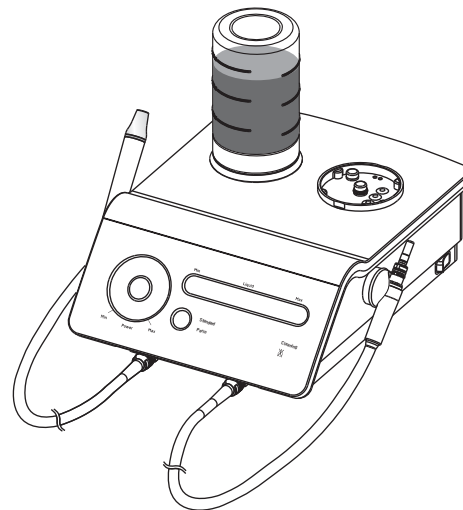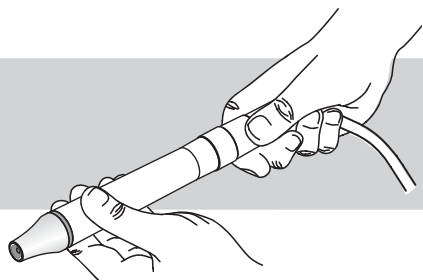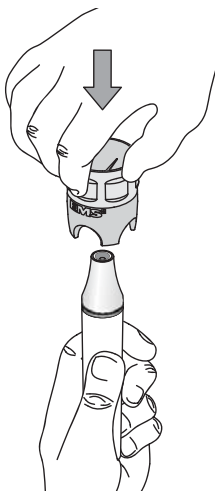Attaching the instruments

 Use only the CombiTorque to tighten the Hu-Friedy & EMS instrument on the handpiece to the correct torque. Once the instrument is screwed all the way in, give an extra quarter of a turn to obtain the required torque.

 For further details on the use of the instruments, please refer to the operating instructions of the Piezon systems.

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
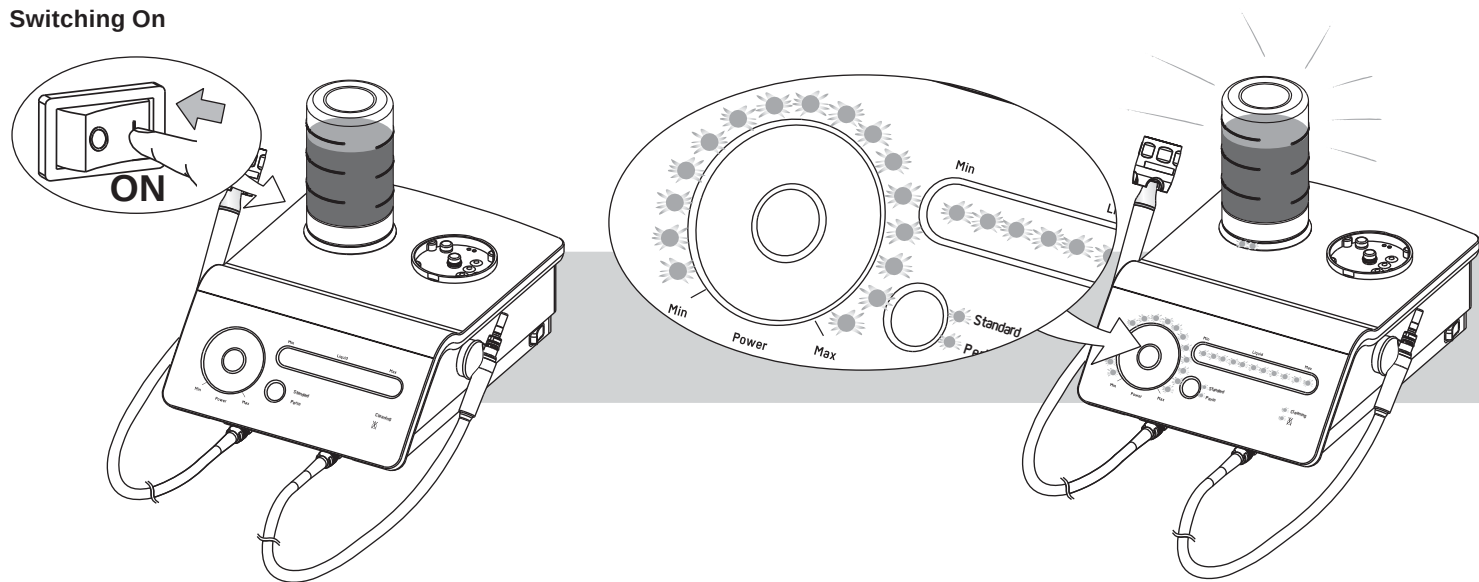
Switching On

When the unit is turned on, the indicator lamps light up and then the unit returns to its previous configuration and the powder chamber is under pressure.

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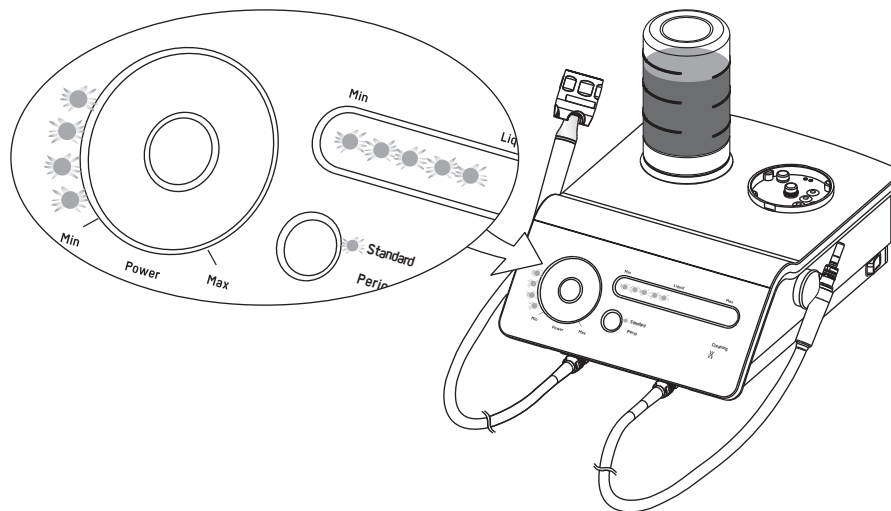
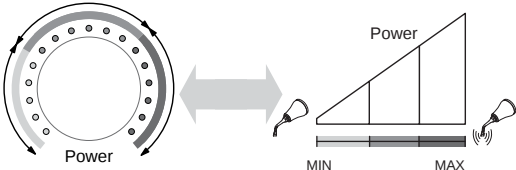
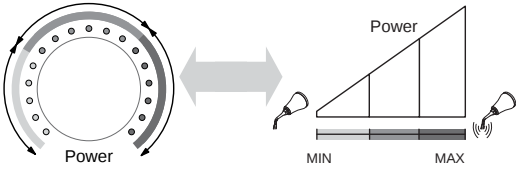
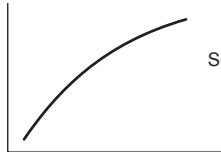
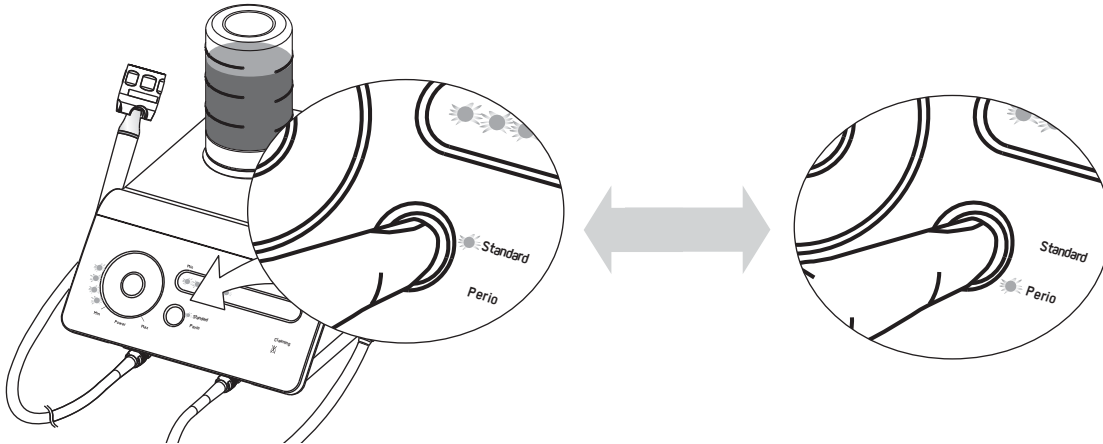
Working mode selection

The unit includes two modes to optimize the ultrasound power range depending on the application. The power setting allows adjustment of the power within the selected range.

For each system (AIR-FLOW and Piezon), the unit remembers the last mode setting with the last power setting and the flow setting.

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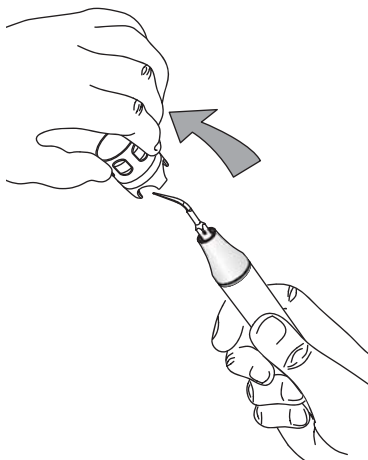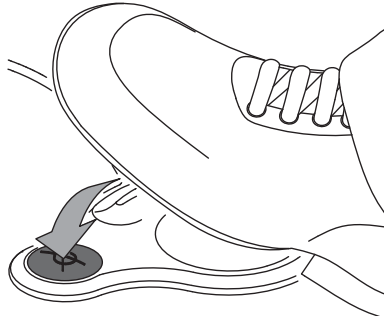
"Dry Work" mode selection


Press the foot pedal on the corresponding symbol to activate the "Dry work" mode

 Never work dry to avoid heat damage to the tissue unless the instrument is intended specifically for this purpose. The tip of the instrument immediately heats up when used without irrigation.

 Please refer to the operating instructions of the Piezon systems for further information concerning instruments usable in "Dry work" mode.

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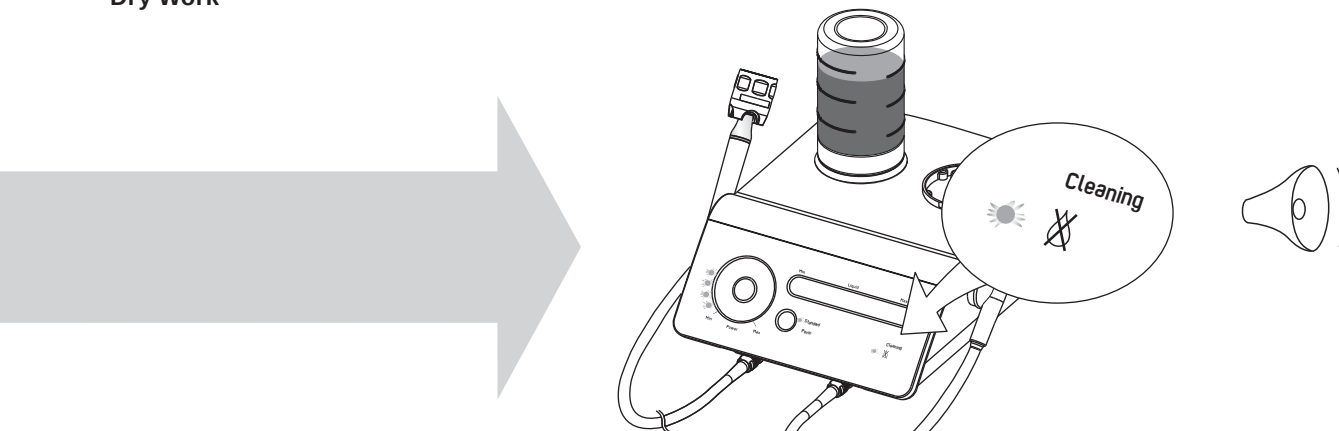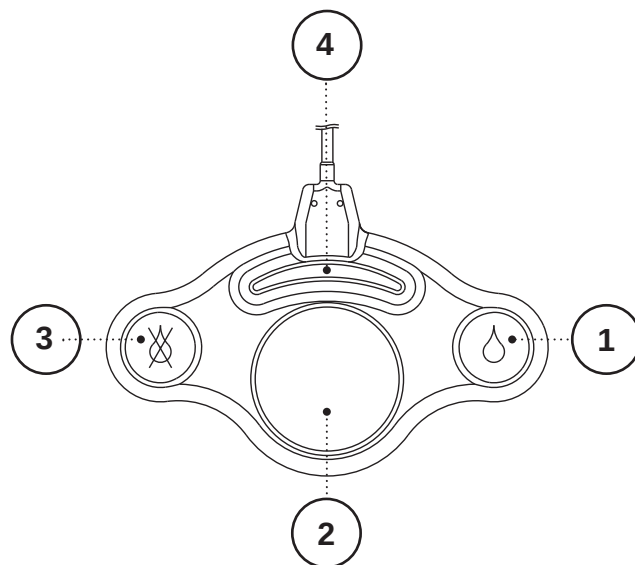
*Temporary ultrasound power increase of 30% (up to 100%)

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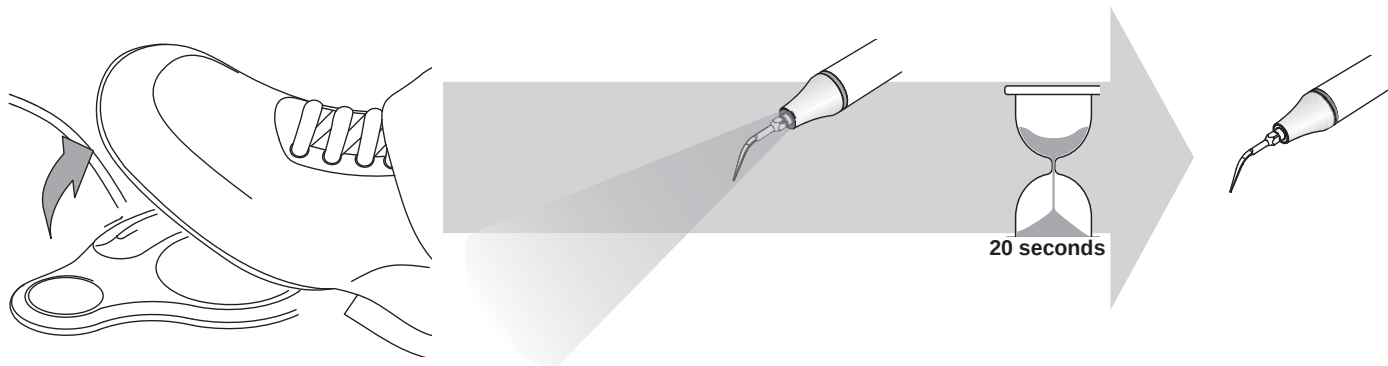
Piezon handpiece LED


 The Piezon handpiece LED lights up when the foot pedal is activated. The light stays on for about 20 seconds after the foot pedal is released.

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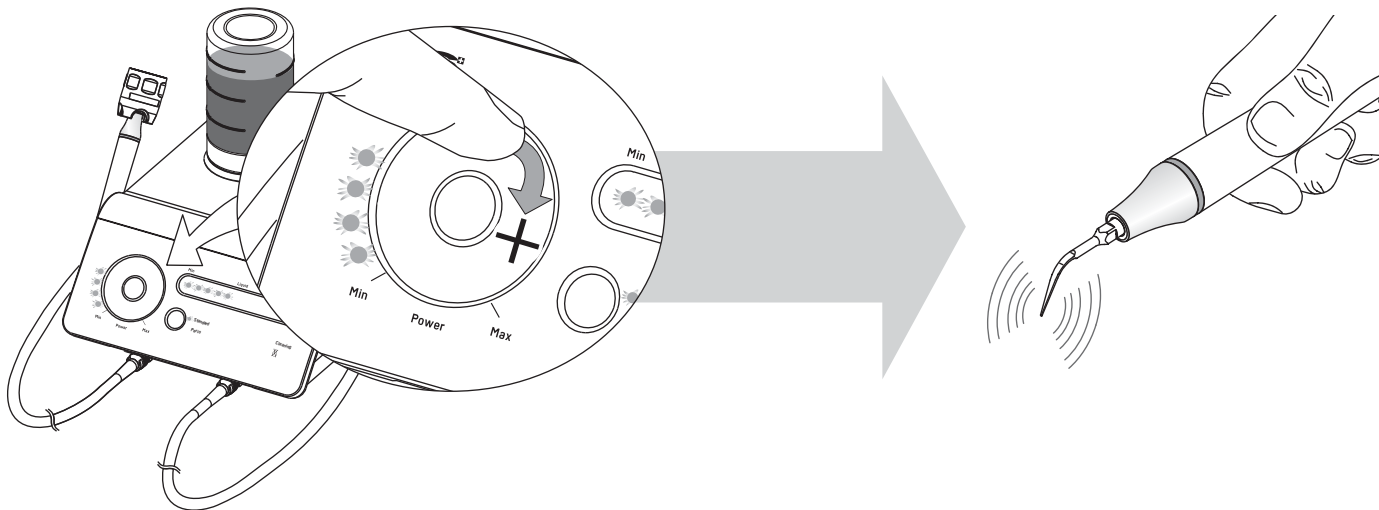Setting the ultrasound power

The settings can be obtained either by sliding or by placing the finger on the touch sensitive zone. To place the finger directly at the desired location is a quick setting alternative.

Optimal setting of the ultrasound power for each instrument are indicated in the operating instructions for the Piezon systems.

The use of a sterile cover on the unit may affect the settings.

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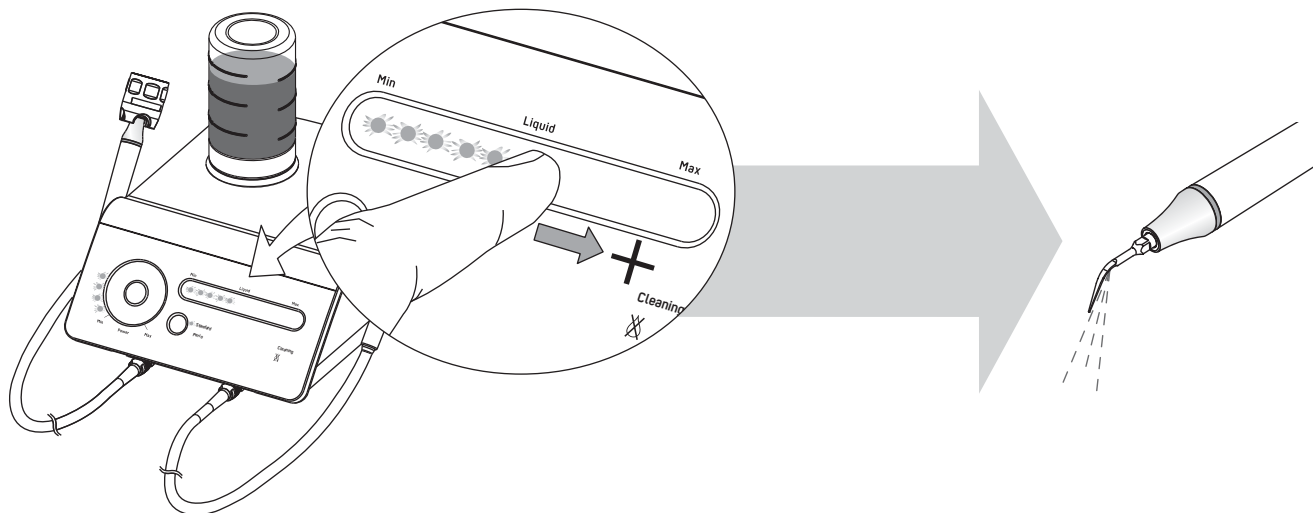
Setting the irrigation flow rate

 Optimal settings for the liquid flow rate for each instrument are indicated in the operating instructions of the Piezon systems

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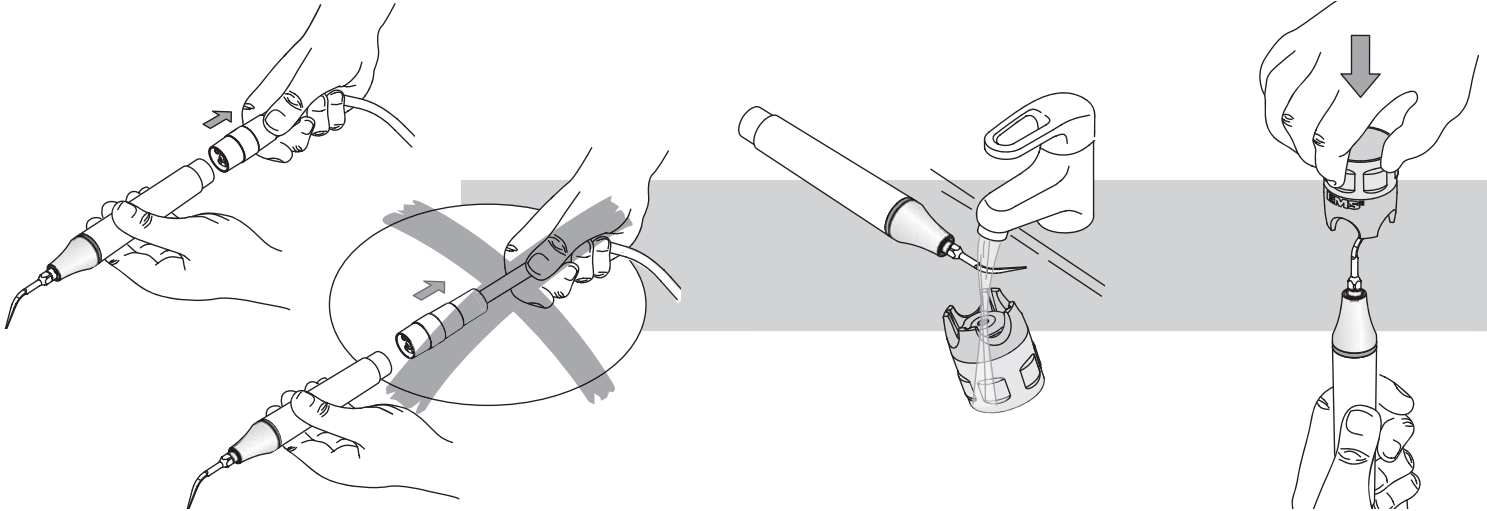
End of the treatment


Instrument disassembly and cleaning

-
-  Clean the instrument with running water before disinfection.
 -  Please conform to the recommendations of the "Reprocessing Instructions" chapter of this manual.

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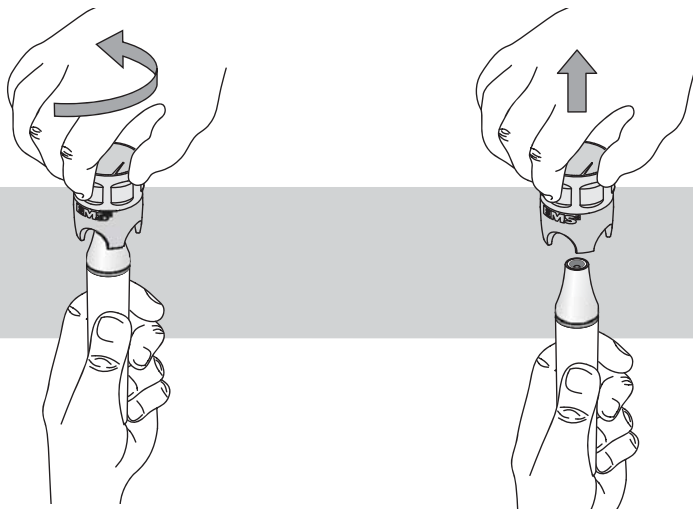Cleaning and disinfecting the circuit

"Cleaning" mode

The liquid circuit of the unit should be washed and disinfected daily.

After 13x on/off switches with no cleaning in between a blinking LED appears in order to remind that a cleaning of the unit is needed.

The 'Cleaning' mode allows the circuit to be washed and disinfected in order to reduce the crystalline deposits and the number of bacteria in the liquid circuit. A rinsing of the circuit is then required to avoid a possible reaction caused by the ageing of the solution or by an unwanted mixture of liquids.

The bottle and the handpiece cords used during treatment must be present during the washing and the disinfecting of the liquid circuit because the process is simultaneous for both elements. Remove the handpiece.

Circuit washing cycle:

- Prepare the bottle with distilled or demineralized water,
- Press the "Purge" mode selector,
- Lift the connector and place it above a sink,
- Press the foot pedal to start the program. Three beep sounds and the blinking of the indicator lamp of the "Cleaning" mode confirms the end of the cycle.

Disinfecting cycle:

- Prepare the bottle with 100 ml of a compatible disinfecting solution (refer to the list of compatible disinfectants at the beginning of this manual),
- Lift the connector and place it above a sink,
- Press the foot pedal to start the program. Three beep sounds and the blinking of the indicator lamp of the "Cleaning" mode confirms the end of the cycle,
- Then allow the disinfecting solution to act for 5 minutes without interrupting the cycle or touching the unit.
- Using the disinfecting solution, press the foot pedal to start a second cycle of the program and allow to act 5 minutes without interrupting the cycle or touching the unit.

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Rinsing cycle:

- Prepare the bottle with sterile distilled water,
- Lift the connector and put it above a sink,
- Press the foot pedal to start the program. Three beep sounds and the blinking of the indicator lamp of the "Cleaning" mode confirms the end of the cycle,

Press the "Purge" mode selector to exit this mode (this cycle must be then restarted).

(Note: it is possible at any time to press the foot pedal to stop the program in progress.)

For information, the duration of the 'Cleaning 'mode cycle is about 30 seconds and the quantity of liquid necessary in the bottle is 50 ml (note that one LED corresponds to 10 seconds).

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Illustration of a "Cleaning" mode cycle



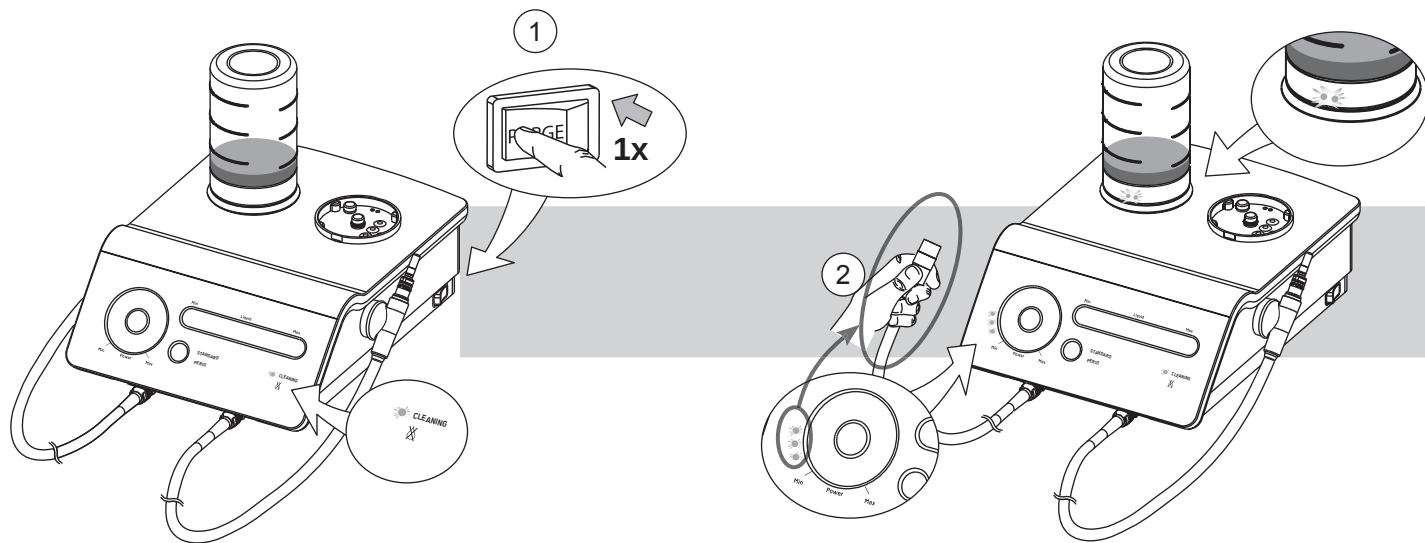
Illustration d'un cycle du mode "Cleaning"

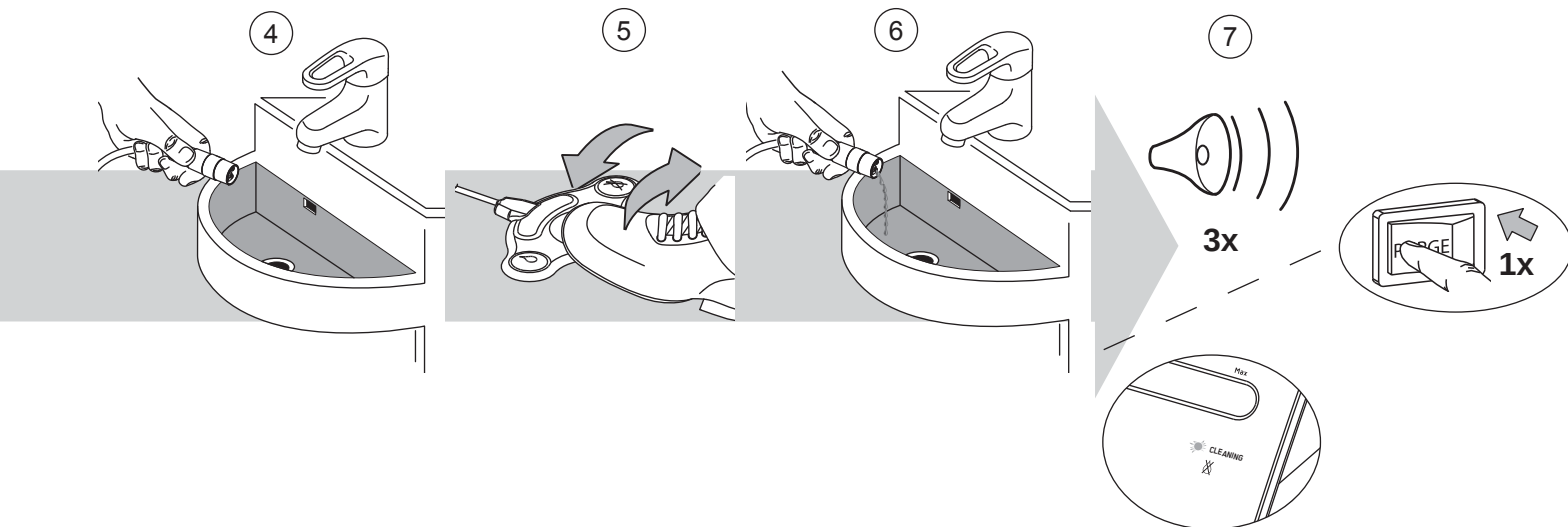Removing the bottle

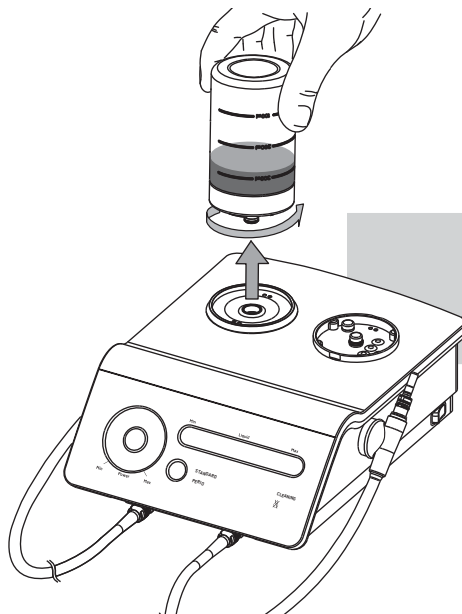
 At the end of each treatment a purge is necessary to remove the liquid inside the cords. If the unit is not going to be used for an extended period, it is essential to completely clean and disinfect the circuit.

Remove the bottle with a rotating movement.

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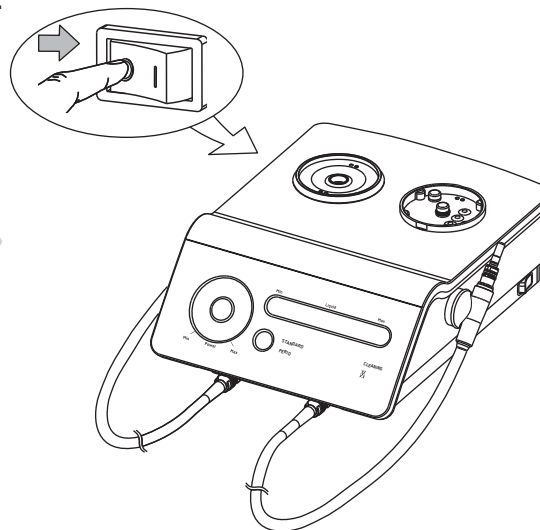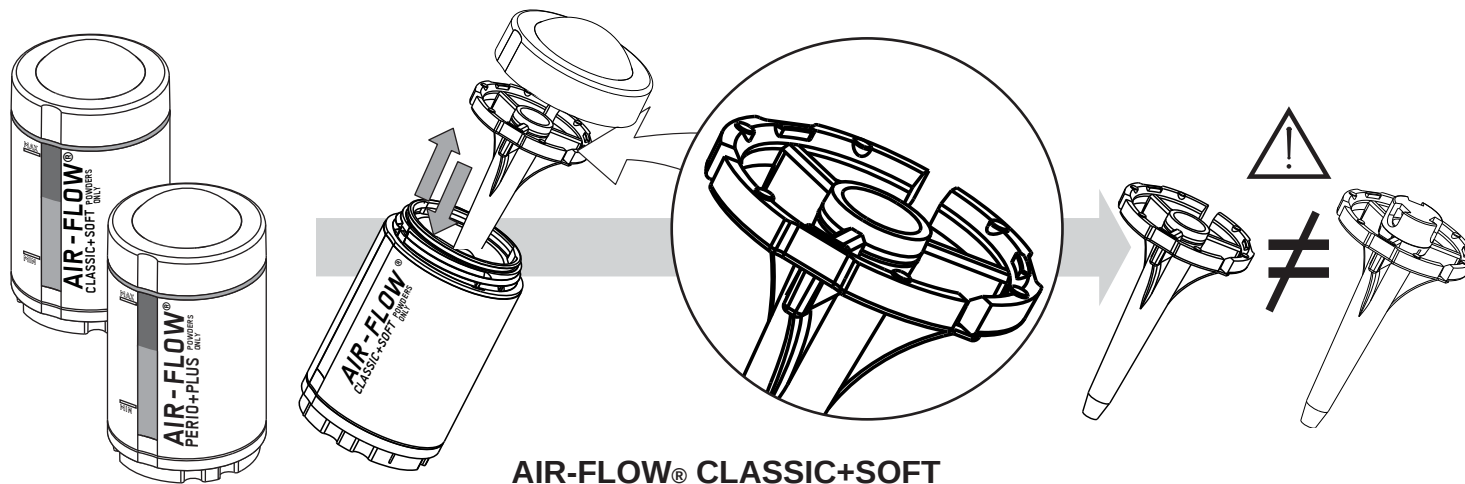
POWDER CHAMBERS

AIR-FLOW® CLASSIC+SOFT

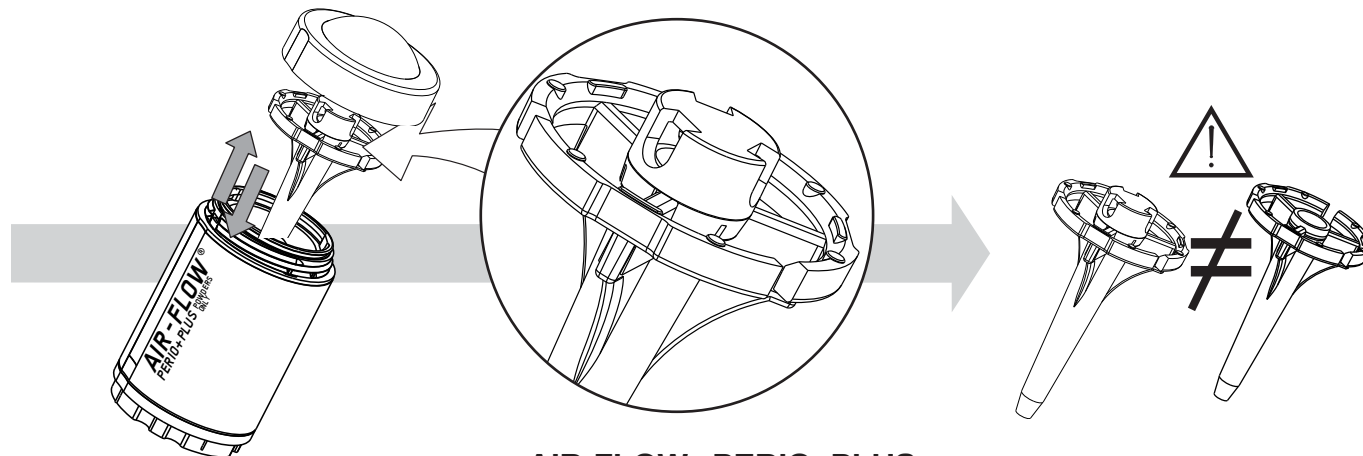
There are two powder chambers having each a specific diffuser.

 The AIR-FLOW CLASSIC+SOFT powder chamber can be used exclusively for supragingival treatment together with the CLASSIC, CLASSIC COMFORT and SOFT powders.

 The AIR-FLOW PERIO+PLUS powder chamber can be used for both supragingival and subgingival treatments using the specific powders PLUS and PERIO.

 Do not invert the diffusers, they are not compatible.

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
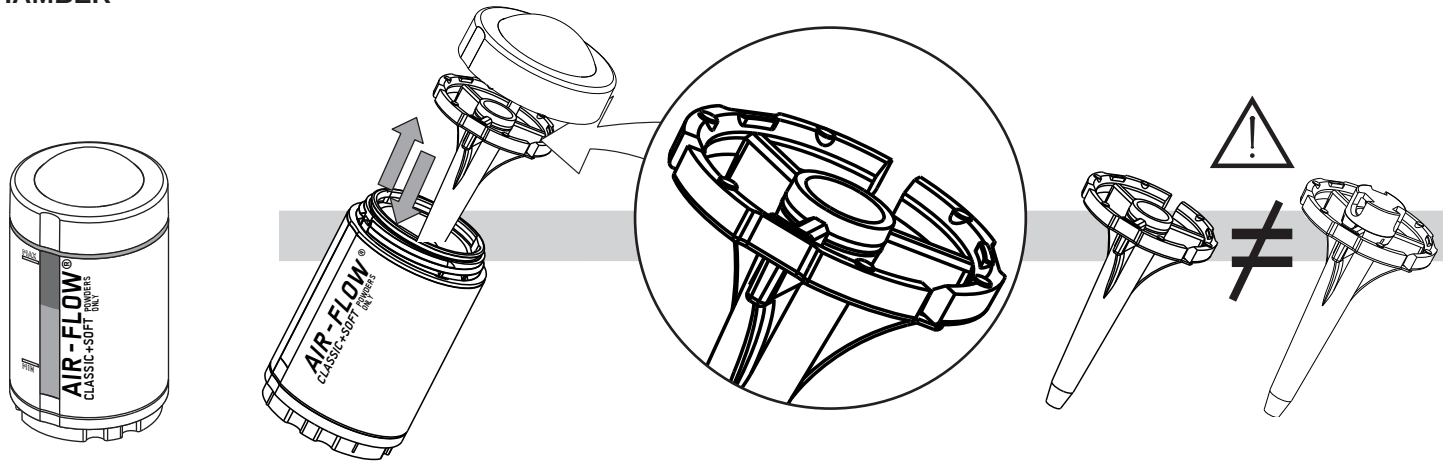
SUPRAGINGIVAL TREATMENT USING THE AIR-FLOW CLASSIC+SOFT POWDER CHAMBER

AIR-FLOW® CLASSIC+SOFT

-  The AIR-FLOW CLASSIC+SOFT powder chamber must be used only for supragingival treatments.
-  The AIR-FLOW CLASSIC+SOFT powder chamber can only be used with CLASSIC, CLASSIC COMFORT and SOFT supragingival powders.
-  Do not invert the diffusers, they are not compatible.
-  Do not use this powder chamber with the PERIO or with the PLUS powders.
-  Use the AIR-FLOW handpiece.

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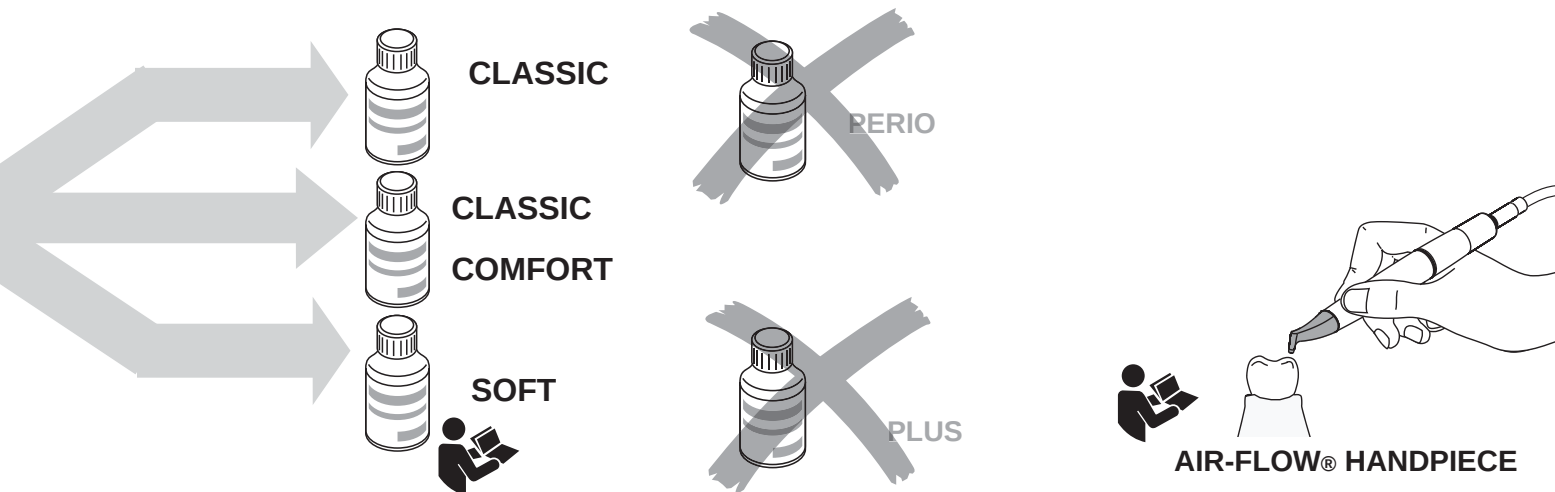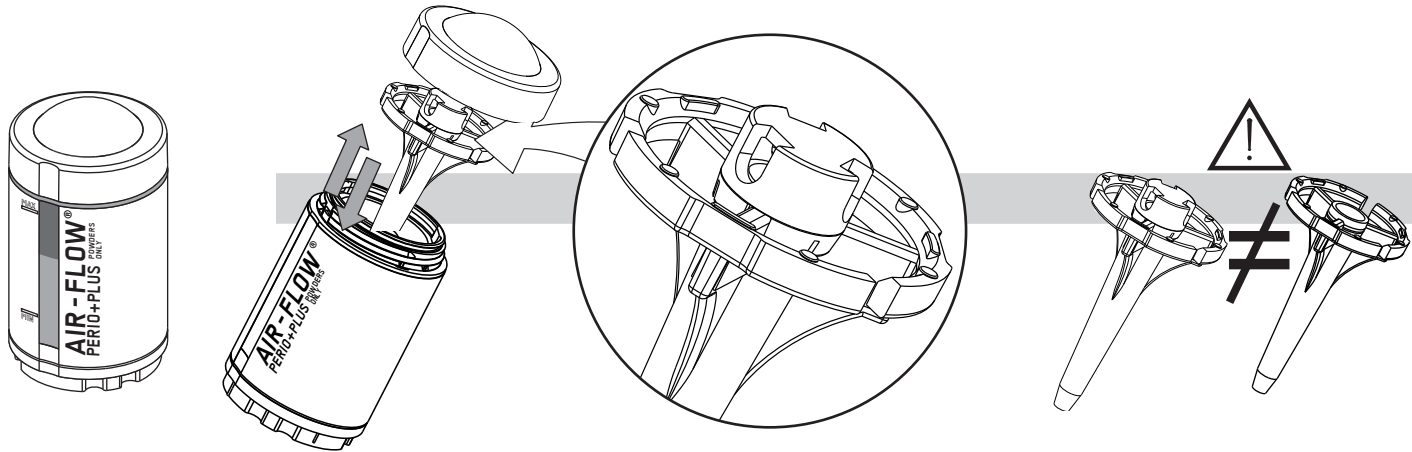
SUPRAGINGIVAL TREATMENT USING THE AIR-FLOW PERIO+PLUS POWDER CHAMBER

AIR-FLOW® PERIO+PLUS

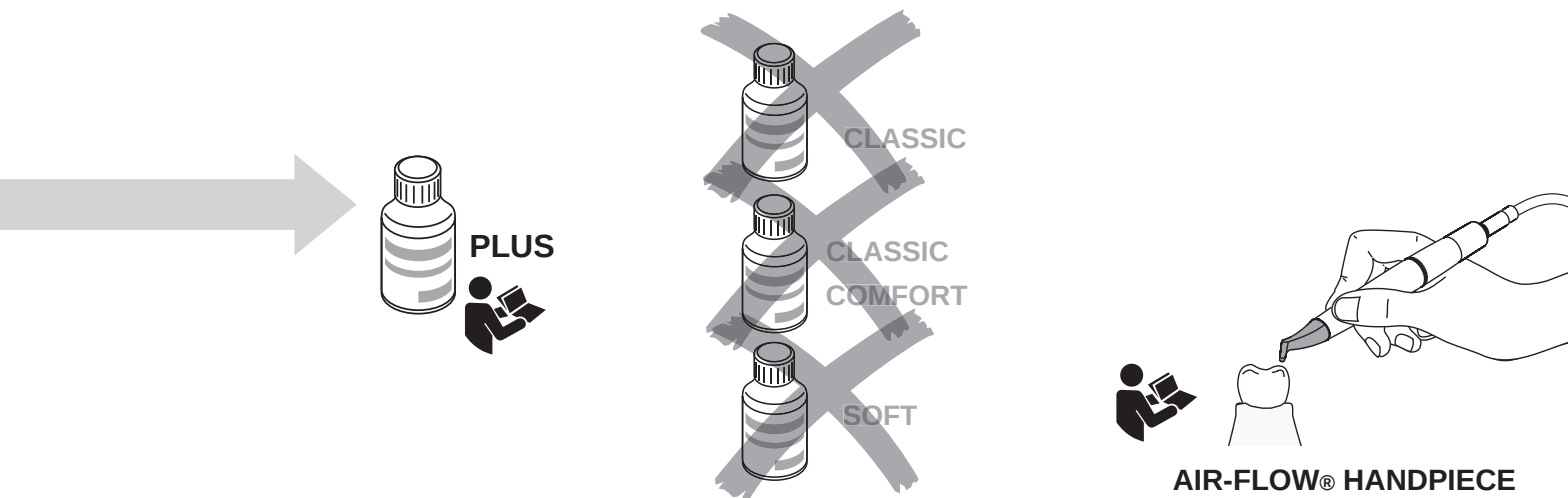
 The AIR-FLOW PERIO+PLUS powder chamber can be used for supragingival treatments using PLUS powder together with the AIR-FLOW handpiece.

 Do not invert the diffusers, they are not compatible.

 Do not use CLASSIC, CLASSIC COMFORT or SOFT powders with this powder chamber.

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

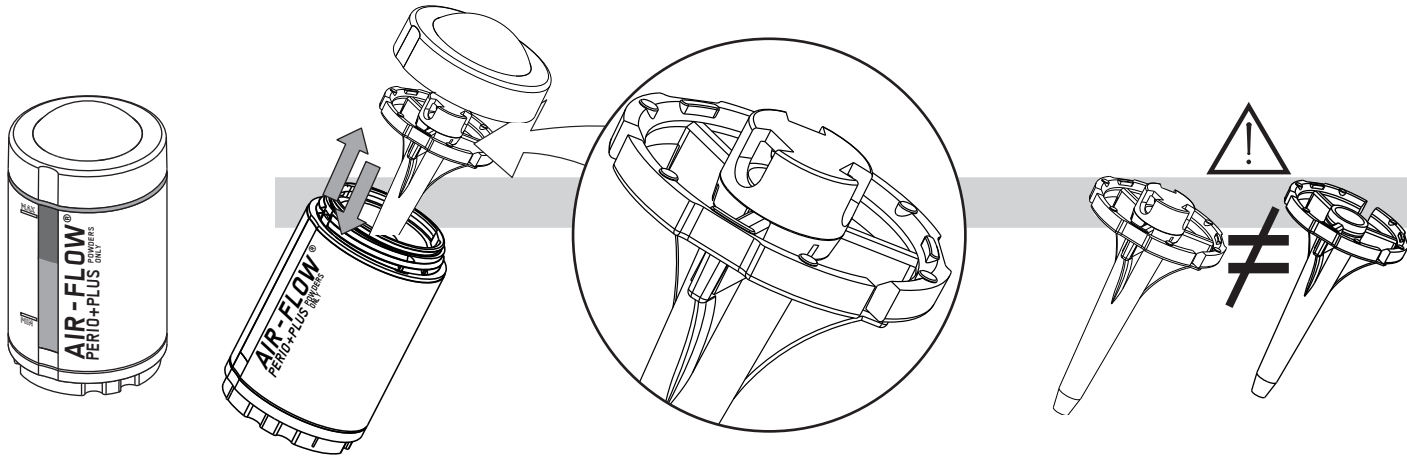
SUBGINGIVAL TREATMENT USING THE AIR-FLOW PERIO+PLUS POWDER CHAMBER

AIR-FLOW® PERIO+PLUS

-
-  For a subgingival treatment with the AIR-FLOW PERIO+PLUS powder chamber use only the PERIO and PLUS powders.
 -  Do not use CLASSIC, CLASSIC COMFORT or SOFT powders for subgingival treatments.
 -  Do not invert the diffusers, they are not compatible.
 -  Read the instructions delivered with the handpiece.

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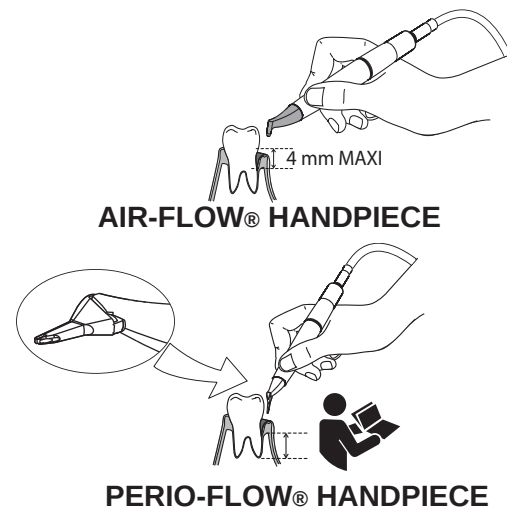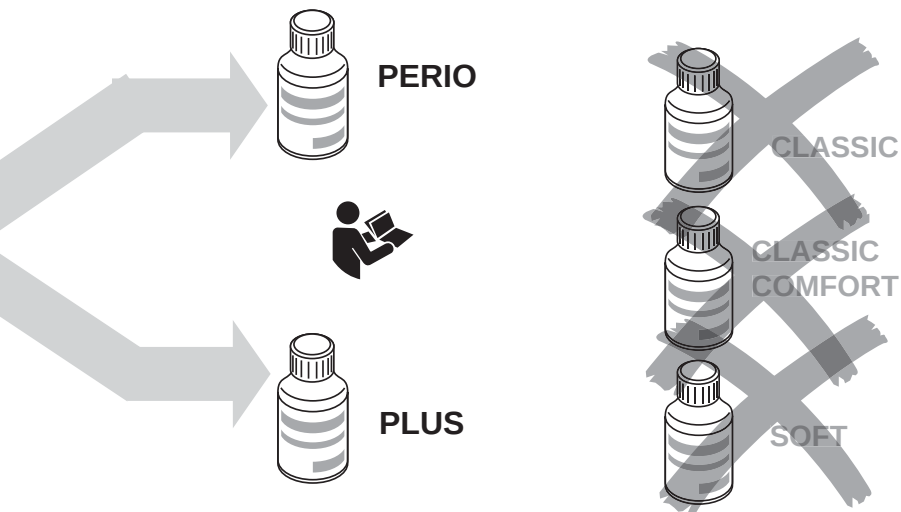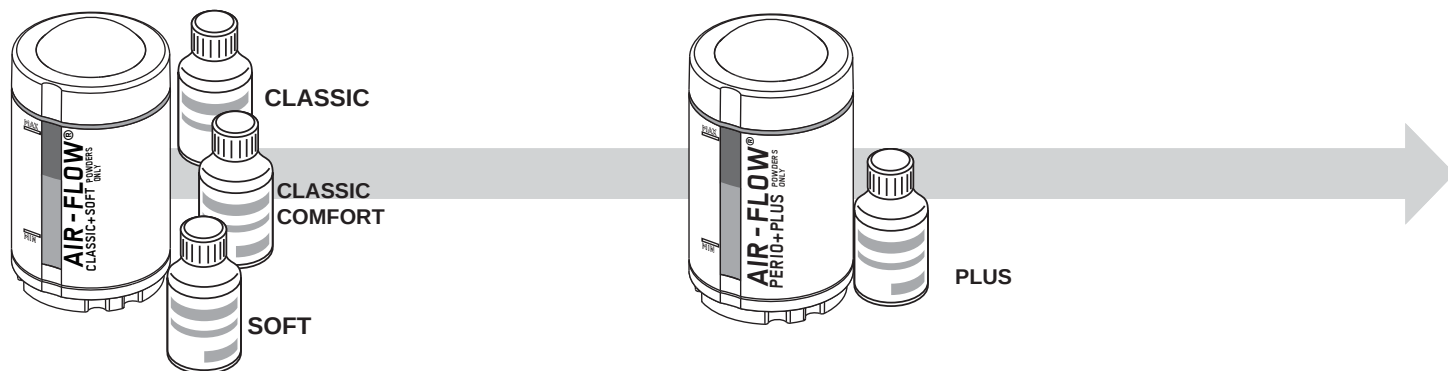
Powder choice

AIR-FLOW® CLASSIC+SOFT POWDER CHAMBER

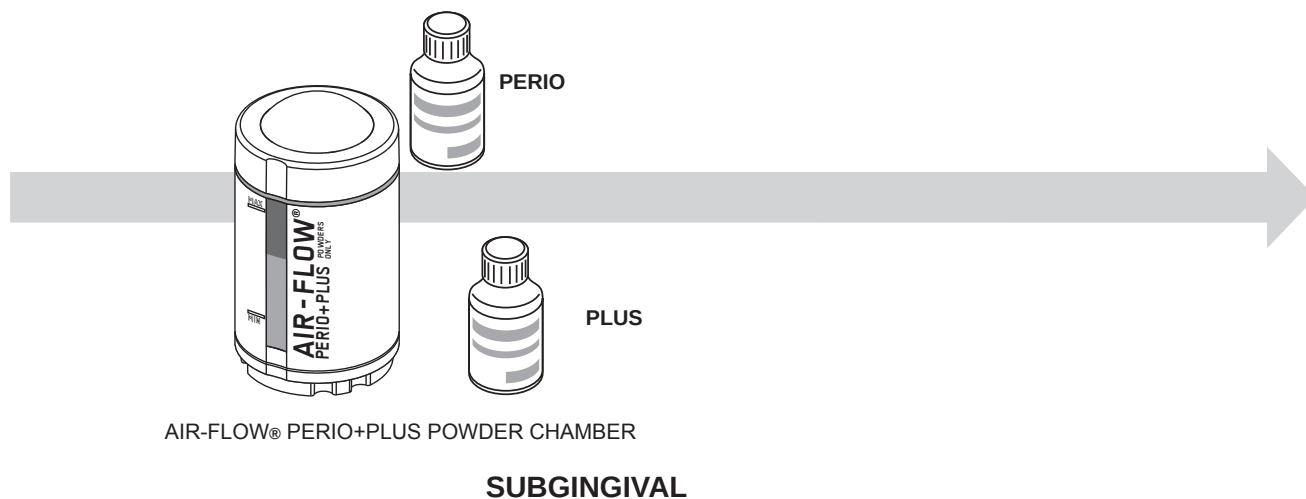
AIR-FLOW® PERIO+PLUS POWDER CHAMBER

SUPRAGINGIVAL

 The powders compatible with the AIR-FLOW CLASSIC+SOFT powder chamber can only be used for supragingival treatments.

 Read carefully the instructions given with the powder before using it. Specific utilisation of each powder is described in the instruction for use delivered with the powders.

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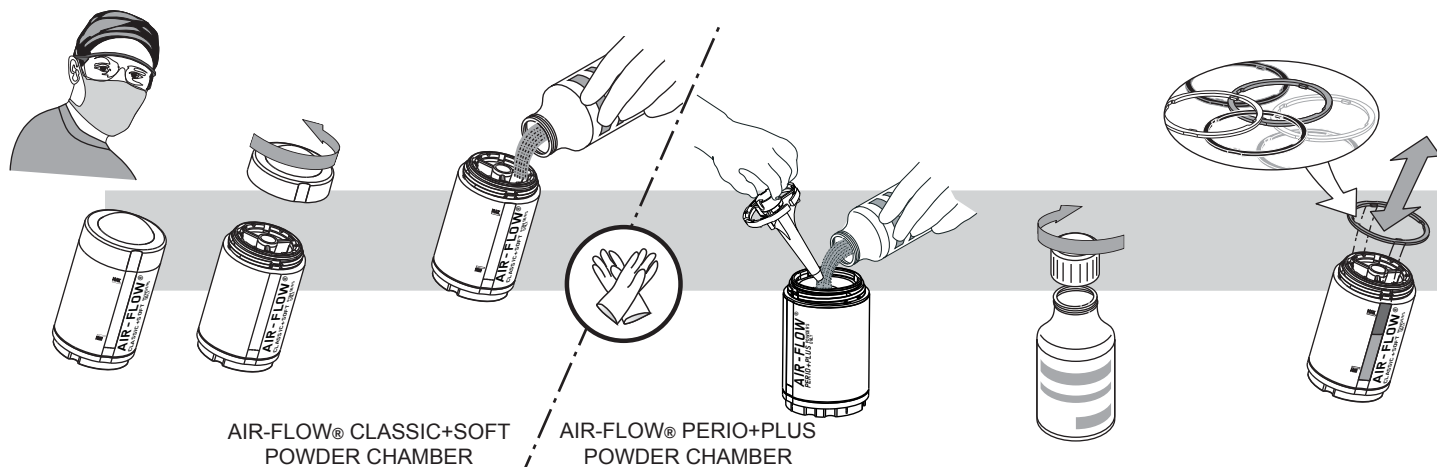
Filling the powder chambers


⚠ Make sure that the powder chambers are absolutely dry. Moisture can cause the powder to cake and provoke a blocking of the circuit.

⚠ To avoid clogging the unit do not fill the powder chambers above the line "Max". Pour the powder into the AIR-FLOW CLASSIC+SOFT powder chamber. The AIR-FLOW PERIO+PLUS powder chamber having a different design follow the instructions as shown above.

⚠ Put the powder diffuser back to its position as soon as you finish to fill the powder chamber.

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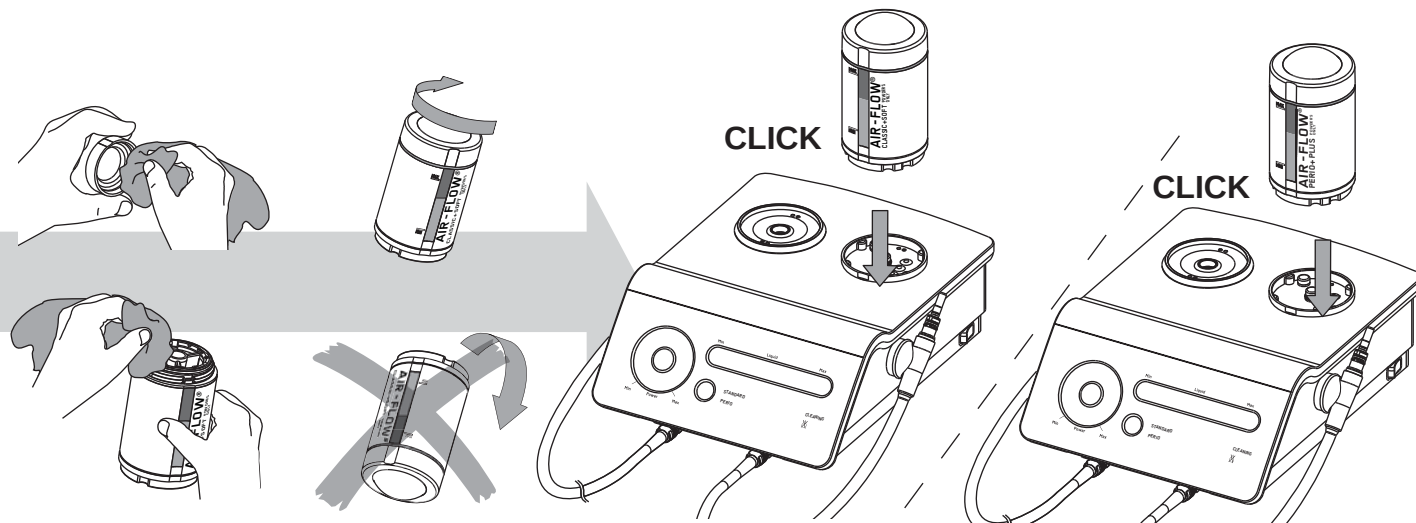-  Use only powder provided by Hu-Friedy & EMS and corresponding to the application.
-  Close correctly the bottle after use. The conditions of storage are described in the chapters "Storing the product" and "Technical data".

A set of interchangeable colored rings is provided to help recognize the chosen powder flavors. Please refer to the information provided in the chapter "Maintenance".

-  Check the condition of the threads on the powder chamber, on the cap and on the O-ring seals. Please refer to the information provided in the chapter "Maintenance".

Clean the threads of the powder chamber and of the cap before screwing them on.

-  Ensure that the powder chambers are closed tight. Otherwise air and powder could leak.
-  Do not turn the powder chambers upside down after filling.
-  The PERIO powder being specially fine, tamping it down the powder chamber must be avoided when filling it.

-  The powder should not remain over a determined time inside the powder chamber. Read the instructions delivered with the powders.

-  The PERIO powder level will go down slightly a few minutes after the filling (powder compaction).

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
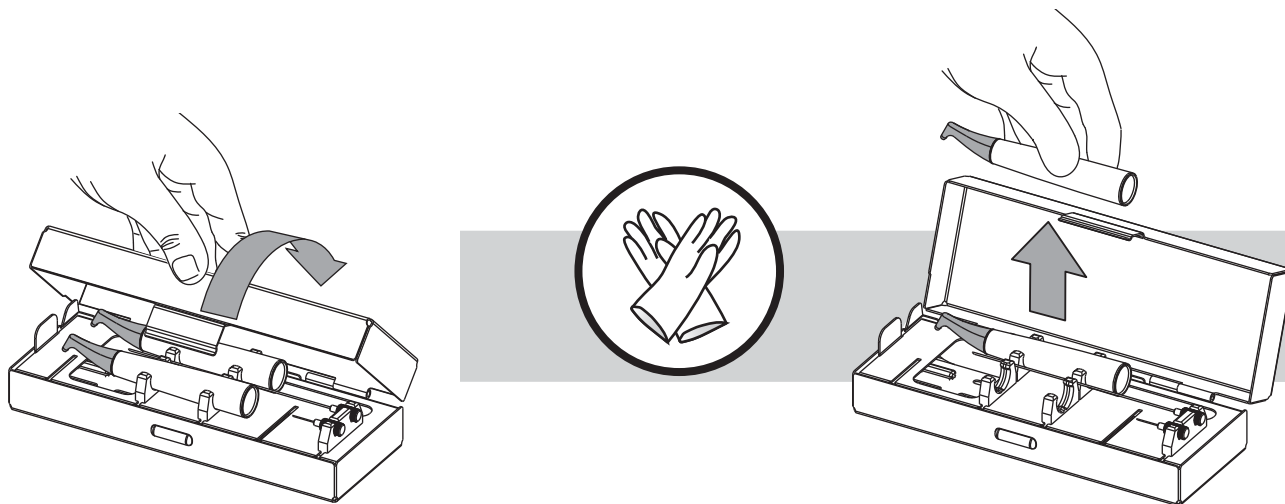
Connecting the AIR-FLOW handpiece

 For further details concerning the use of the handpiece, please refer to the operating instructions of the AIR-FLOW systems.

 The AIR-FLOW handpiece can be used in combination with a AIR-FLOW PERIO+PLUS powder chamber in some applications.

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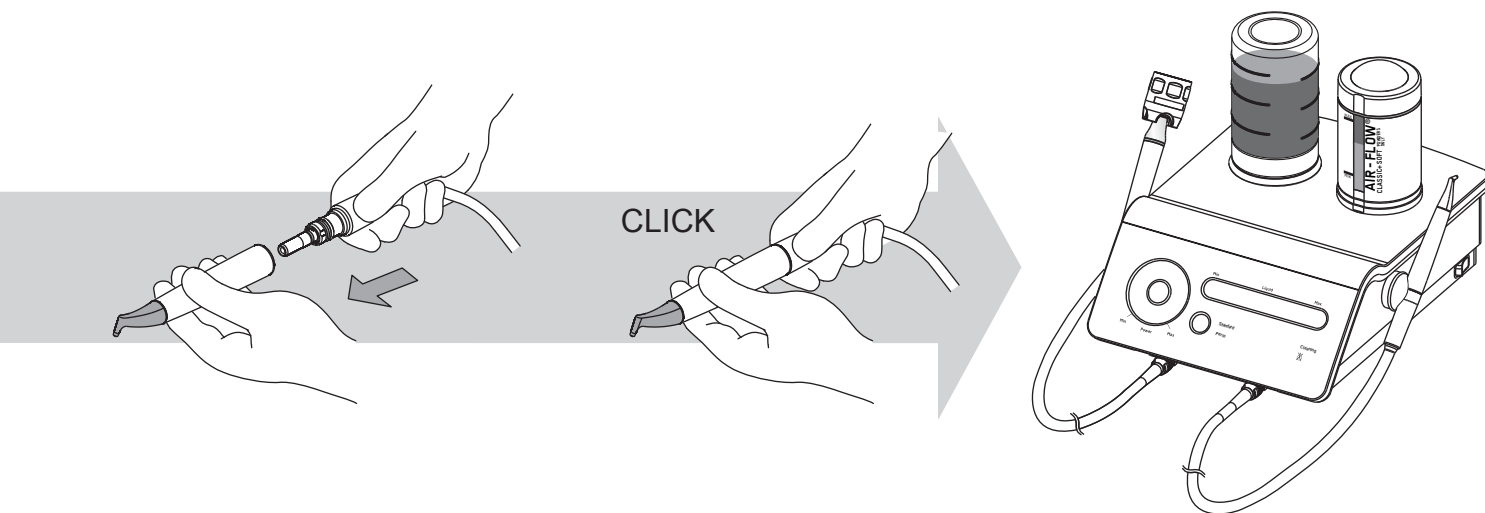
Operating the unit

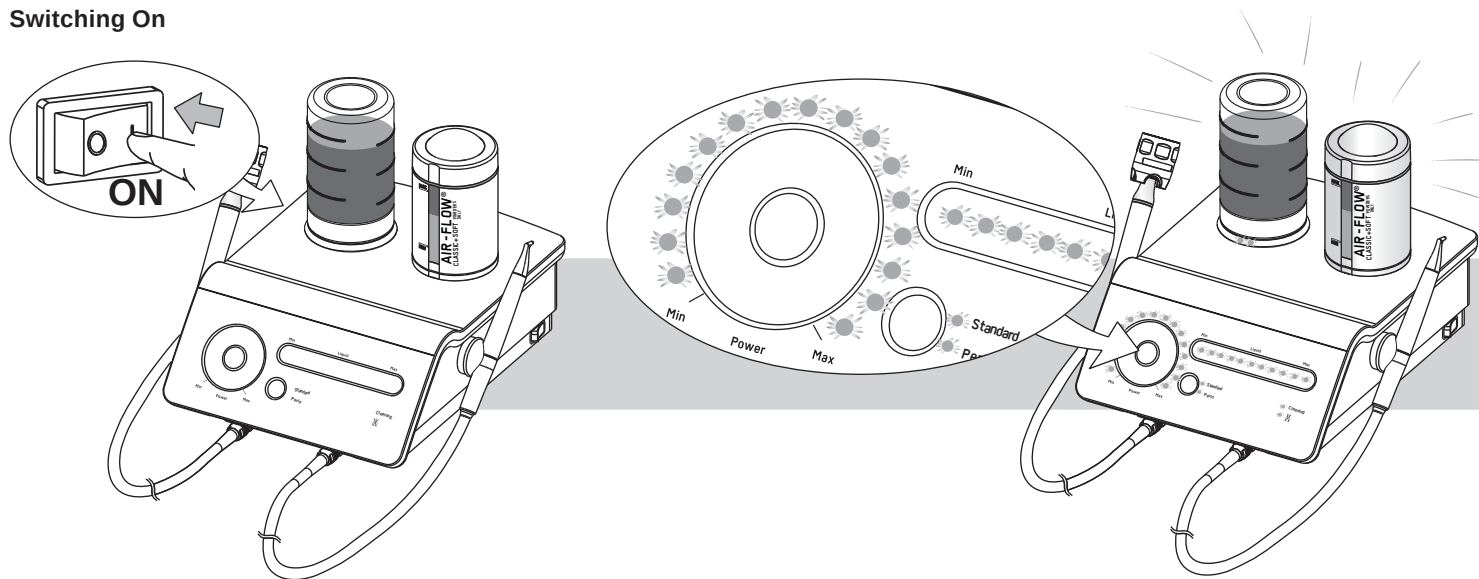Switching On


When the unit is turned on, the control lamps light up and then the unit returns to its previous configuration and the powder chamber is under pressure.

 Before treating the patient, rinse the eventual residues of the disinfecting solution used by the Piezon system out of the circuit of the AIR-FLOW system. Put the handpiece above a sink and press the foot pedal.

 After 15 min. of inactivity, the unit switches to standby mode. Placing a finger on the touch-sensitive zone wakes the unit from standby mode. The unit returns to its previous configuration.

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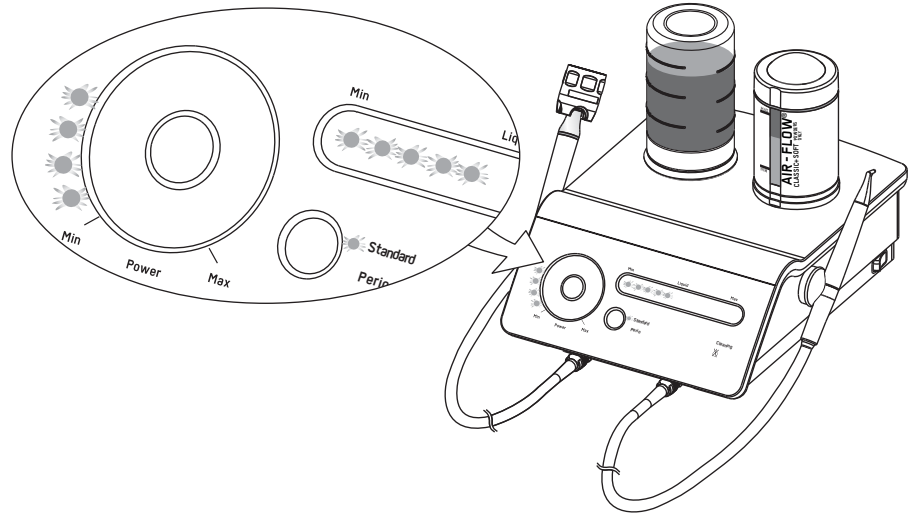
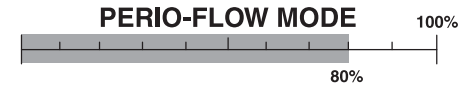
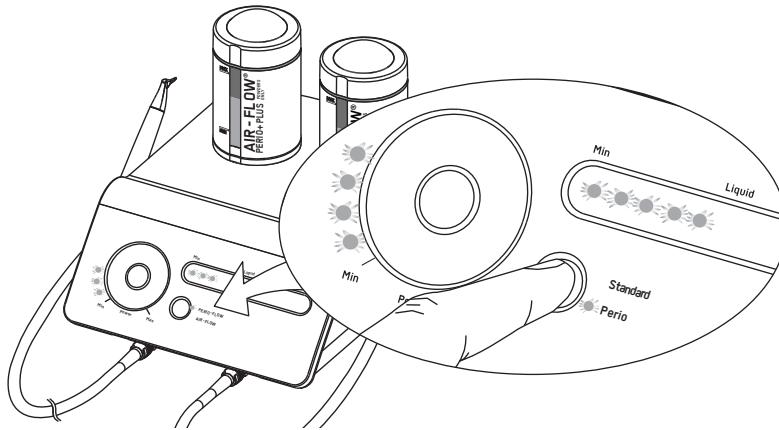
Working mode selection

In the PERIO-FLOW mode, the available air pressure is reduced compared to the AIR-FLOW mode. The air pressure changes automatically with the selection mode.

In PERIO-FLOW mode, one beep sound is generated every 5 seconds (recommended duration per single pocket treatment).

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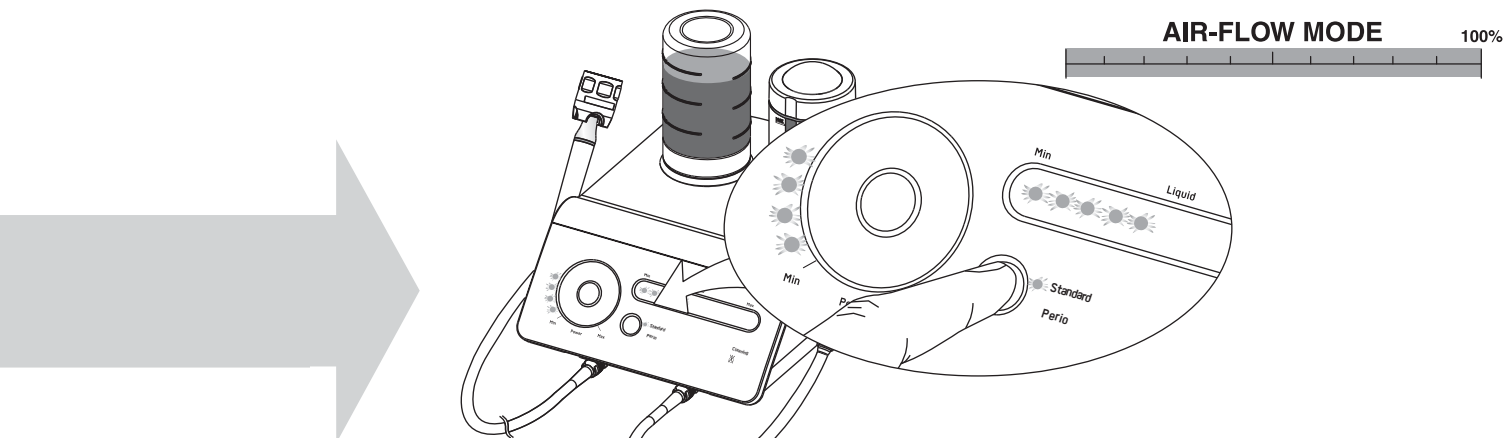
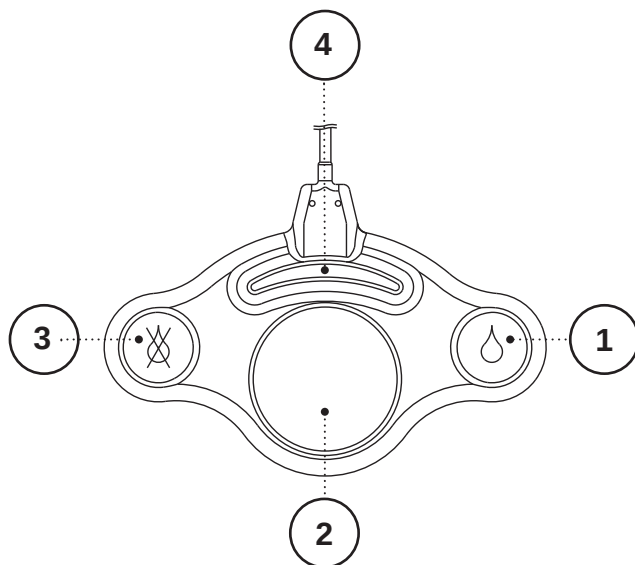
*Temporary air pressure increase of 30% (up to 100%)

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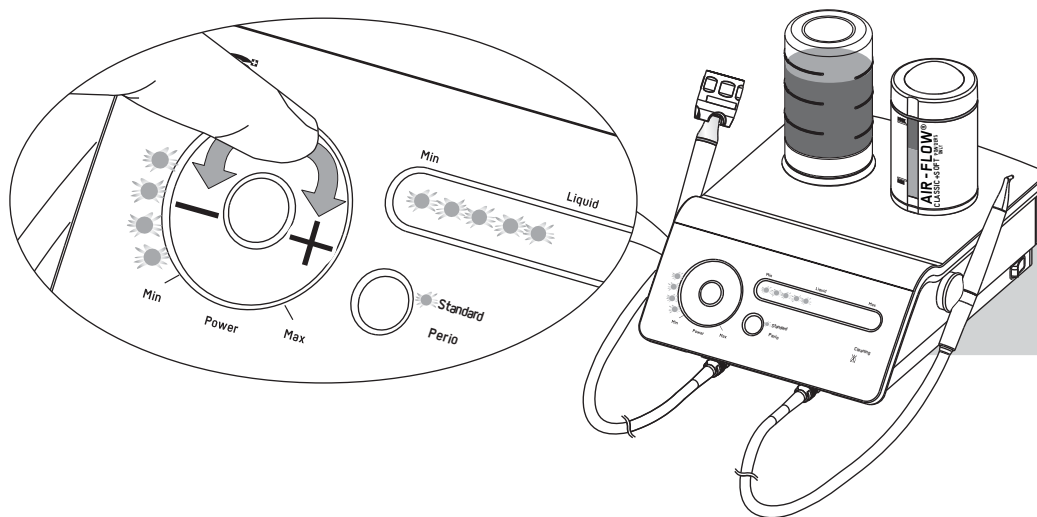
Setting the air/powder flow rate

The adjustments can be obtained either by sliding or by placing a finger on the sensitive zone. To place a finger directly at the desired location is a quick-setting alternative.

The unit has a memory function that recalls the last settings that were entered, even if the unit has been turned off.

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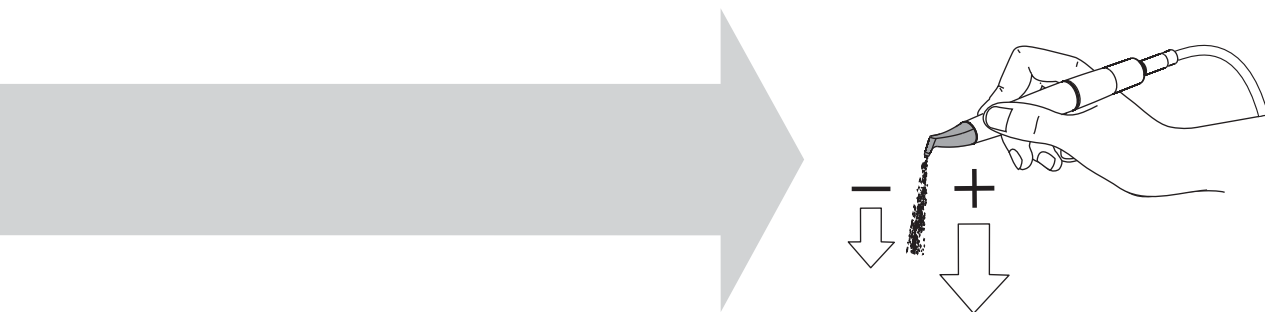
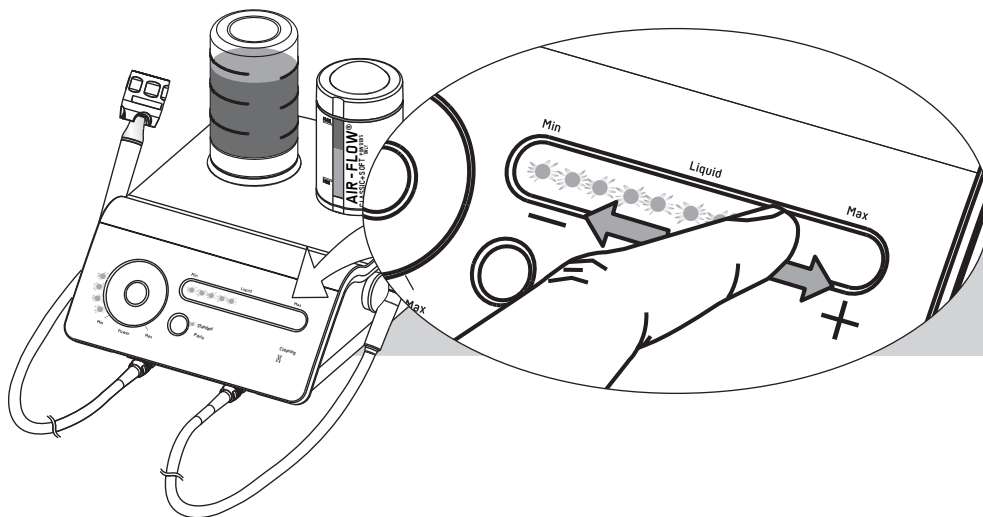
Setting the water flow rate

 Information regarding how to use and adjust the water/air setting can be found in the operation instructions of the AIR-FLOW systems.

 Using the unit without irrigation will generate a substantial cloud of powder that could make the patient feel uncomfortable.

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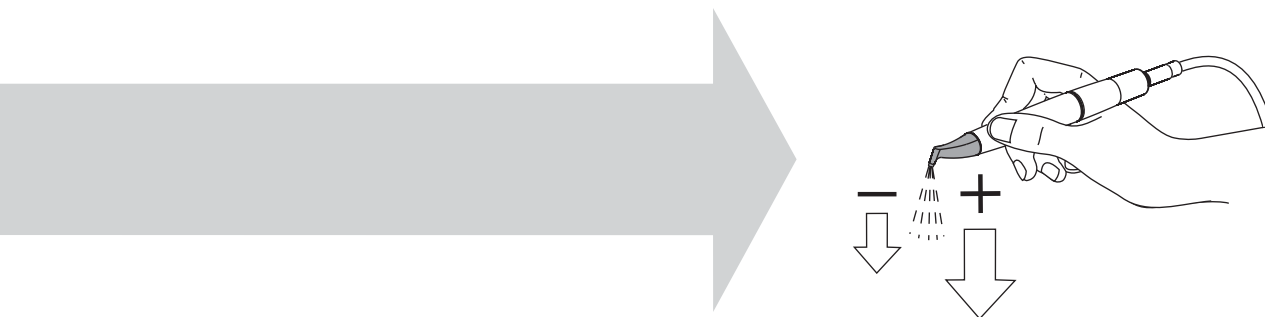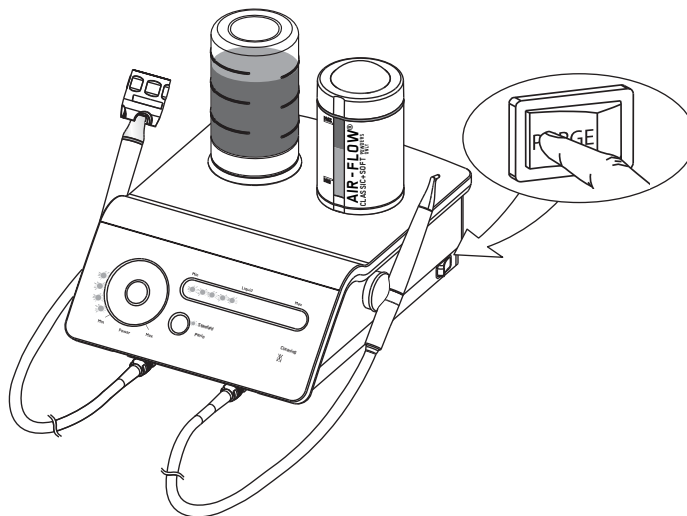
Purge switch and mode

The purge switch allows the depressurization of the powder chamber for safe removal and to repressurize the chamber once it is back on the unit.

A beep sound will be emitted when the purge switch is pressed. After approximately 1 minute the unit will return to its previous configuration (or when pressing the switch a second time).

The purge switch also controls the "Cleaning" function (see following pages).

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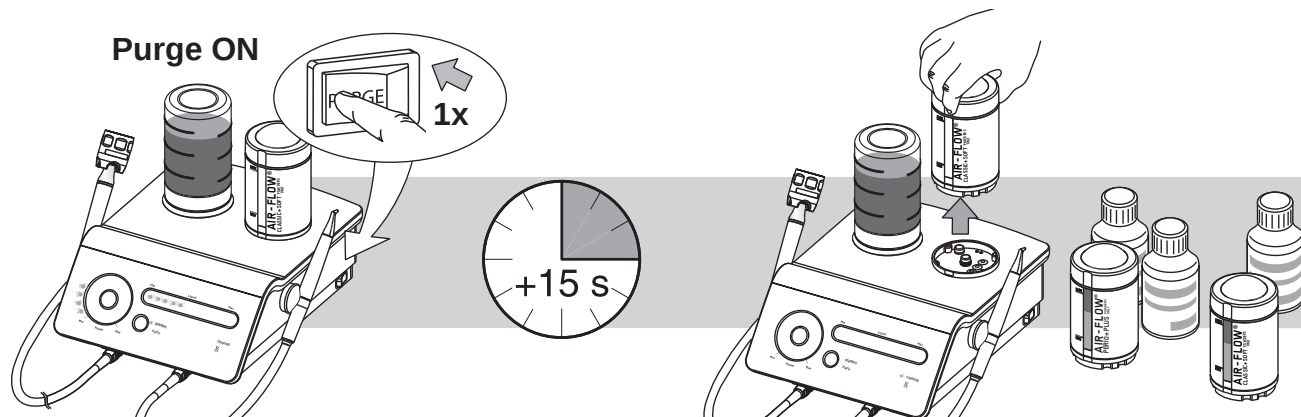
Powder exchange or refill

 Do not remove or open the powder chamber when it is under pressure.

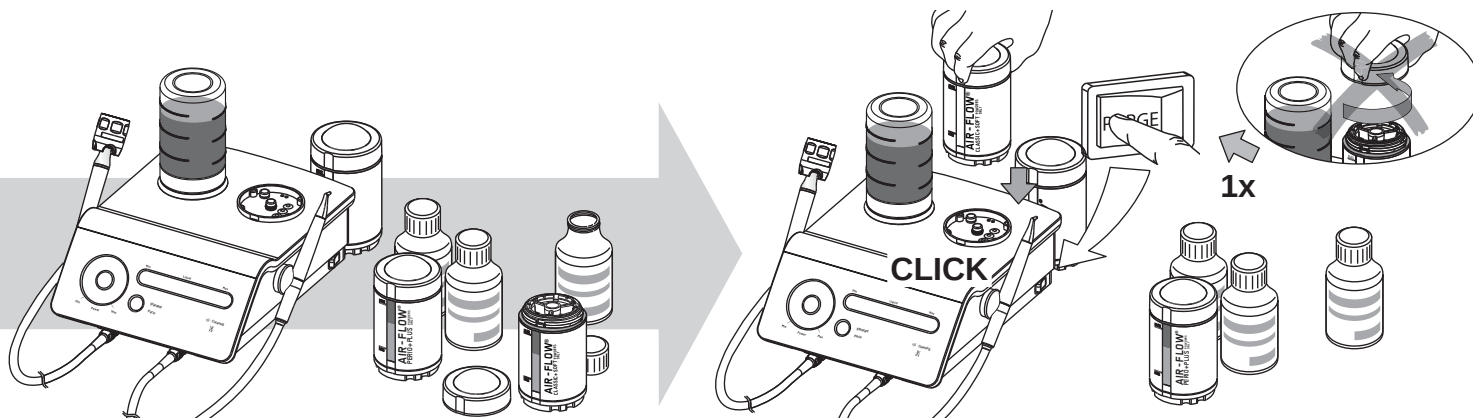
Press once on the purge switch to release the air pressure from the powder chamber.

Fill the chambers as indicated in chapter "Filling the powder chamber".

 Never unscrew the cap of the powder chamber when it is on the unit.

When the chamber is on the unit, press again on the purge to put the chamber under pressure.

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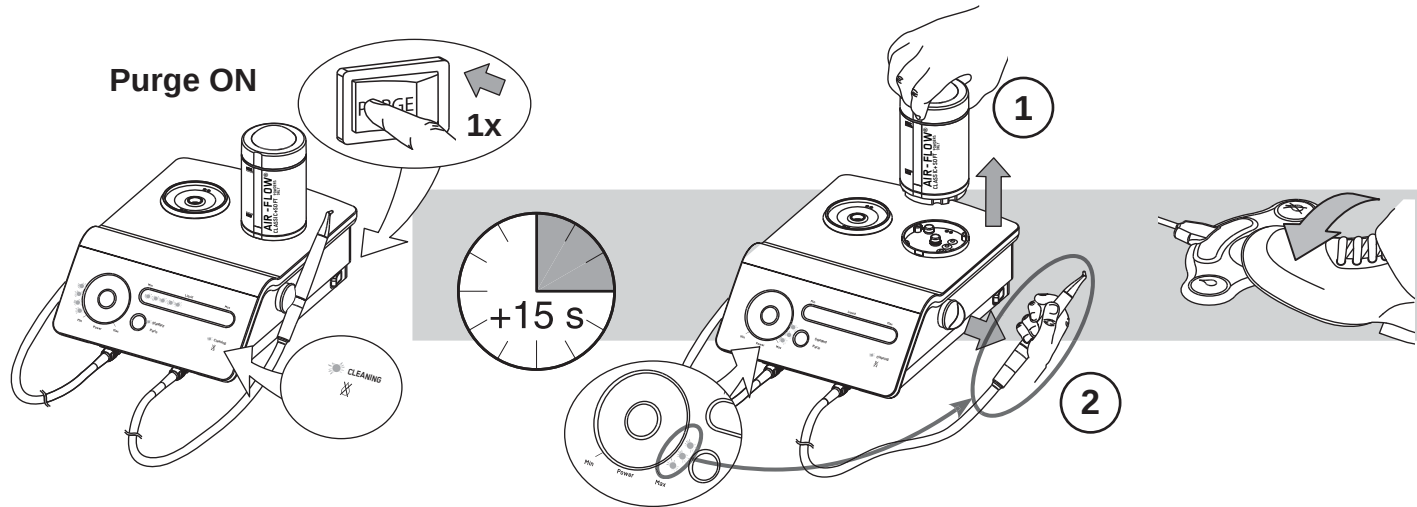
End of the treatment

Powder and water purge


Remove the powder chambers before or after the purge of the AIR-FLOW circuit.

 After each treatment a purge is necessary to remove all residues of powder and liquid inside the cords and the handpieces.

 If the unit is not going to be used for an extended period, it is important to empty the irrigation circuit to avoid the formation of biofilm.

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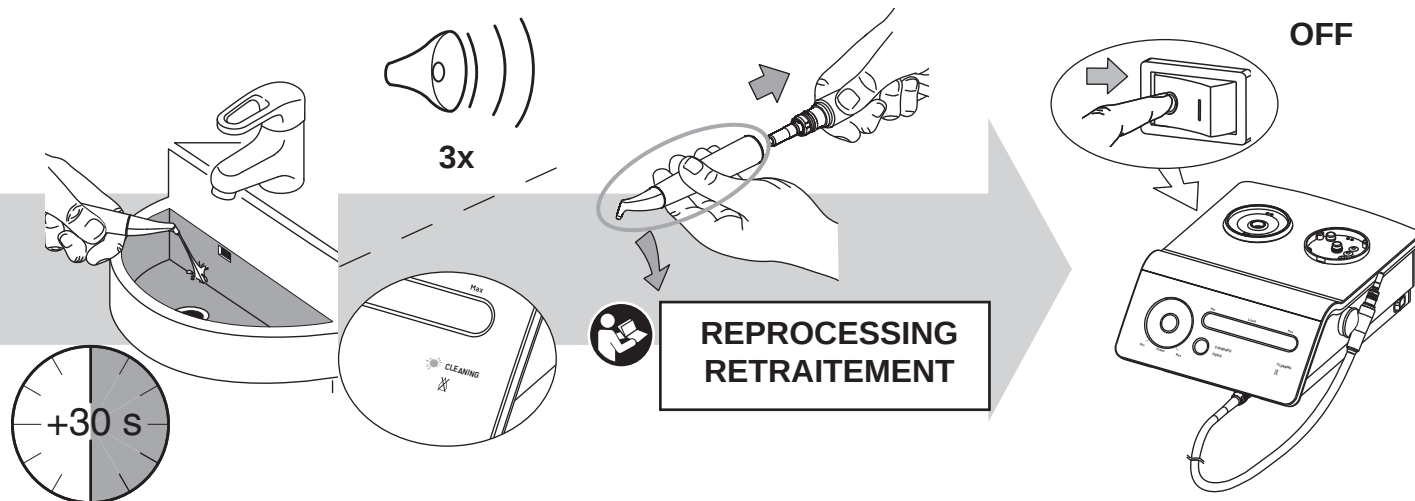
PERIO-FLOW SYSTEM

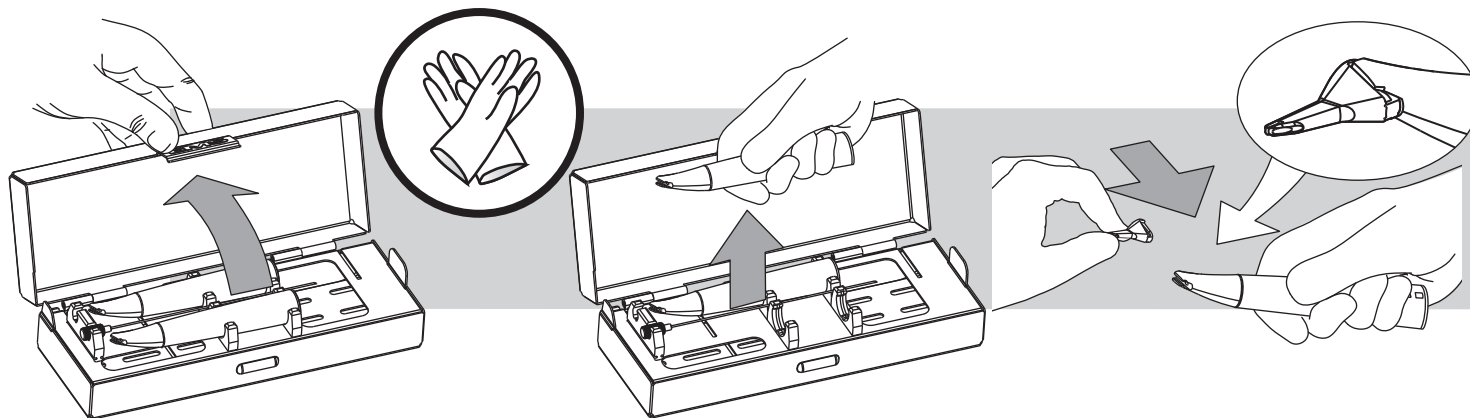Installing the nozzle and connecting the PERIO-FLOW handpiece


 For further details concerning the use of the handpiece please refer to the operating instructions of the PERIO-FLOW system and in the chapter System AIR-FLOW of this manual.

 The PERIO-FLOW handpieces must be used only with the AIR-FLOW PERIO+PLUS powder chambers.

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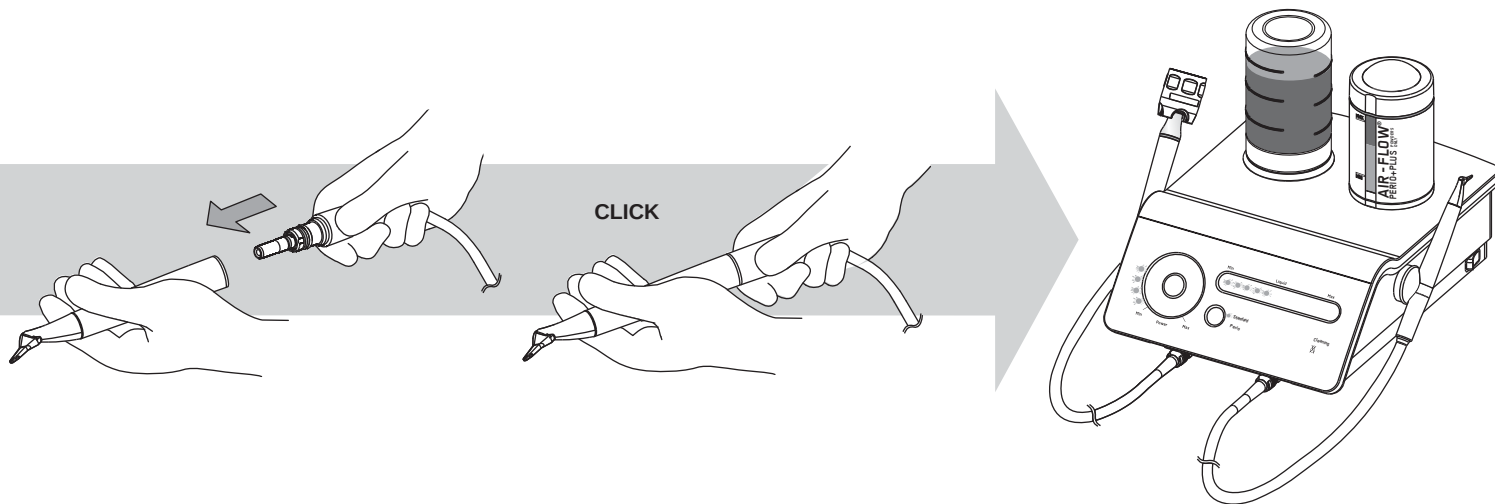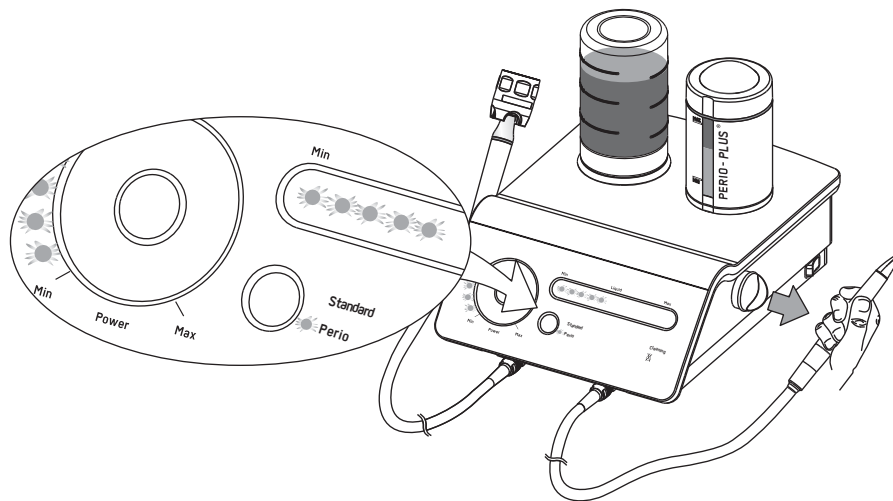
PERIO-FLOW mode

The PERIO-FLOW mode is automatically selected when the handpiece used with the AIR-FLOW PERIO+PLUS powder chamber is lifted from its support.

 Press the "working mode selection" switch to return to AIR-FLOW mode when the AIR-FLOW handpiece is used.

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


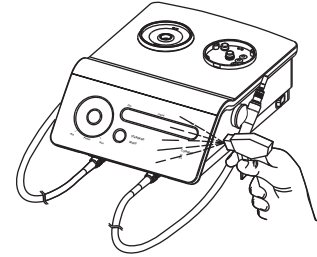
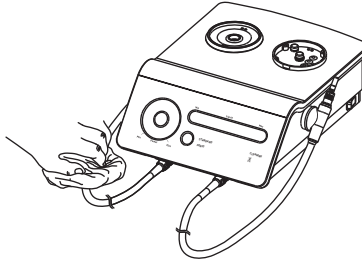
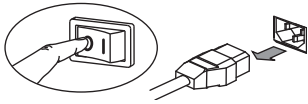
CLEANING, DISINFECTING AND STERILIZING

-
-  Please conform to the recommendations of the Reprocessing Instructions chapter of this manual.
 -  Follow present-day regulations enforced in the country about reprocessing.
 -  Clean the unit only with an alcohol-based, commercially available (ethanol, isopropanol), colorless disinfectant. The use of scouring powder or an abrasive sponge will damage its surface.
 -  Never remove the silicone cap from the powder chamber holder when cleaning the unit.
 -  Do not use needles on the silicone cap of the powder chamber holder.

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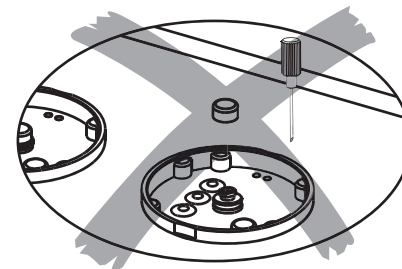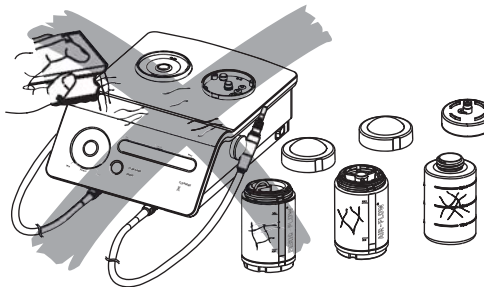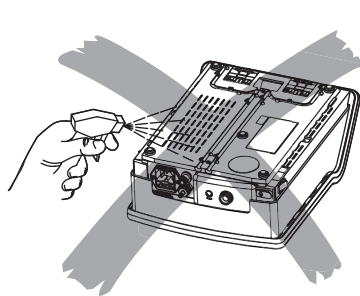All instruments must be cleaned and sterilized before the first use. Reusable instruments must be cleaned and sterilized following each use. A listing of the reusable instruments is provided in the table below. Diamond-coated instruments are indicated for single use only and must not be reused.

Reusable Instruments:

Description	Description
INSTRUMENTS	Instrument PI
Intrument A	Soft Instrument
Instrument C	Instrument RT4
Instrument B	Cap SP
Instrument H	File ISO 15
Instrument D	File ISO 20
Instrument E	File ISO 25
Instrument F	File ISO 30
Instrument G	File ISO 35
Endochuck 120°	Flat wrench
Instrument P	COMBITORQUE
Endochuck 90°	Combitorque
Instrument SP	HANDPIECE HOLDER
Instrument PS	Handpiece Holder diam 16
Instrument PL1	HANDPIECE
Instrument PL2	Handpiece Piezon LED
Instrument PL3	AIR-FLOW handpiece
Instrument PL4	PERIO-FLOW handpiece
Instrument PL5	CLEANING NEEDLE
Instrument PI	Cleaning needle
Endo-chuck 180°	PERIO-FLOW NOZZLE EXTRACTOR
Endochuck 120°	Nozzle extractor
Instrument RT3	NEEDLE GUIDE
Instrument RE2	Needle guide

Recommended cleaning, packaging for sterilization, and sterilization procedures

Manual cleaning

- Remove any externally attached soiling by brushing carefully with a soft surface brush or cloth.
- Remove the instruments from handpieces.
- Soak all parts in the cleaning solution for, at a minimum, the time and the concentration specified by the manufacturer of the cleaning agent (ENZOL, 2 %, 10 min).
- Remove any externally attached soiling by brushing carefully with a soft surface brush or cloth.
- Place all parts in an ultrasound bath filled with cleaning solution (ENZOL, 2 %).
- Treat the parts with ultrasound for at least 3 minutes.
- Brush the parts in the cleaning solution (ENZOL, 2 %) if any visible soiling is detected.
- Submerge all components completely in deionized water.
- Rinse through all product lumens with deionized water.
- Repeat the cleaning process if the last rinse does not run clear, or if stains are still visible on the product.
- Dry all components at room temperature.

Handpiece

- Remove any externally attached soiling by brushing carefully with a soft surface brush or cloth.
- Remove the handpiece from the connector. Remove the gaskets, caps and other removable parts.
- Soak all parts in the cleaning solution for, as a minimum, the time and the concentration specified by the manufacturer of the cleaning agent (ENZOL, 2 %, 10 min).
- Brush all lumens and surfaces with a suitable lumen / surface brush until all visible soiling is removed.
- Rinse through all product lumina (e.g. irrigation and aspiration connection) with the cleaning solution (ENZOL, 2 %), at least 5 times in the flow direction (no back rinsing), using a disposable syringe (min. volume 50 ml) applied to the nozzles of the product.
- Submerge all components completely in deionized water.
- Rinse through all product lumina (e.g. irrigation and aspiration connection) with deionized water at least 3 times in the flow direction (no back rinsing), using a disposable syringe (min. volume 50 ml) applied to the nozzles of the product.
- Repeat the cleaning process if the last rinse does not run clear, or if stains are still visible on the product.
- Dry all components at room temperature.

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Packaging for Sterilization

The clean, dry instruments should be placed in the Steribox sterilization container. The container should be bagged in a sterilization pouch that is validated for use in a 132°C pre-vacuum steam sterilization cycle. The sterilization pouch should be sealed according to the manufacturer's instructions.

Sterilization

The reusable Hu-Friedy & EMS handpieces, instruments, and accessories must be steam-sterilized prior to the first use and following each patient use. Hu-Friedy & EMS recommend sterilization in a fractional prevacuum cycle at 132°C for 4 minutes with a drying time of 20 minutes. Hu-Friedy & EMS recommend use of biological indicators and chemical indicators that have been validated for use in a 132°C pre-vacuum steam sterilization cycle.

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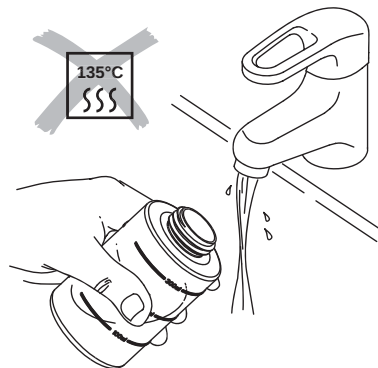
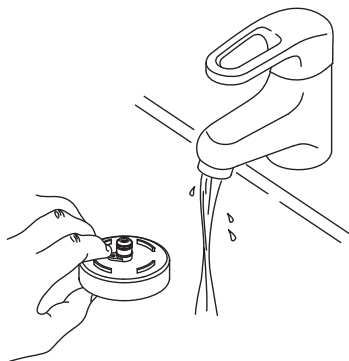
Cleaning the accessories



 The bottle can be washed in a dishwasher and disinfected up to 95°C in a washer disinfectant.

The bottle cannot be sterilized.

 Please refer to the operating instructions of the Piezon systems to remove, clean, disinfect and sterilize the instruments and the handpiece.

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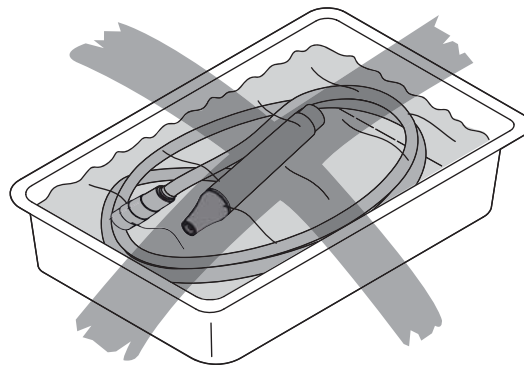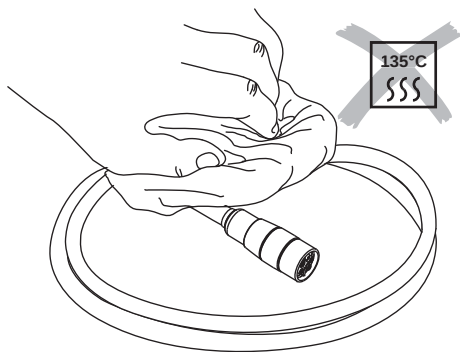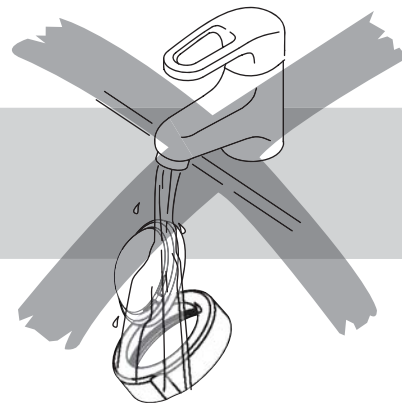
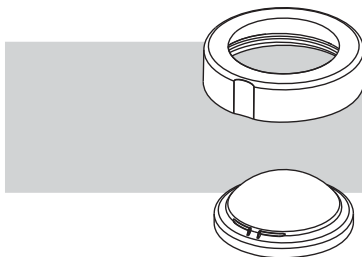
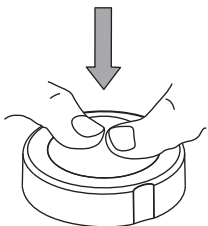
Cleaning the dome


Disassemble the dome to facilitate cleaning.

Regularly clean and disinfect the cap.

⚠ The ribs of the dome must be aligned with the lugs of the ring. The two pieces must be correctly aligned and interconnected.

⚠ The cap must be completely dry before closing.

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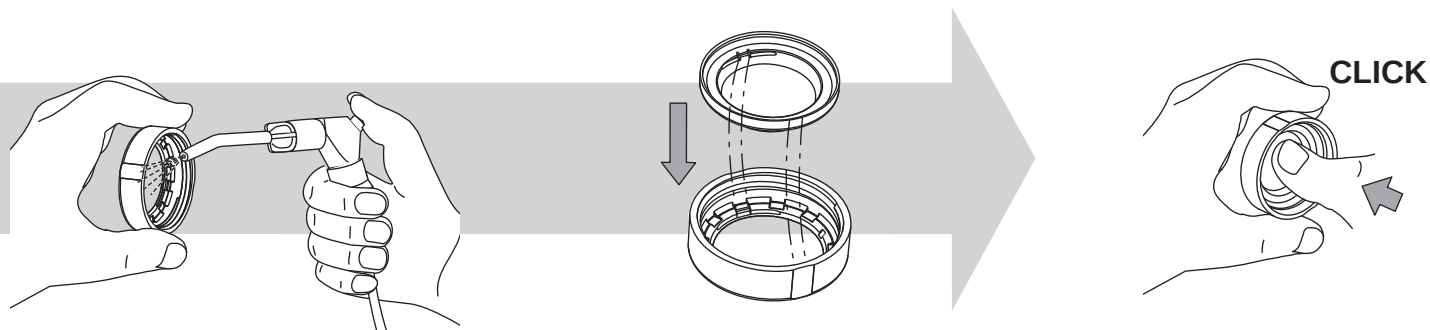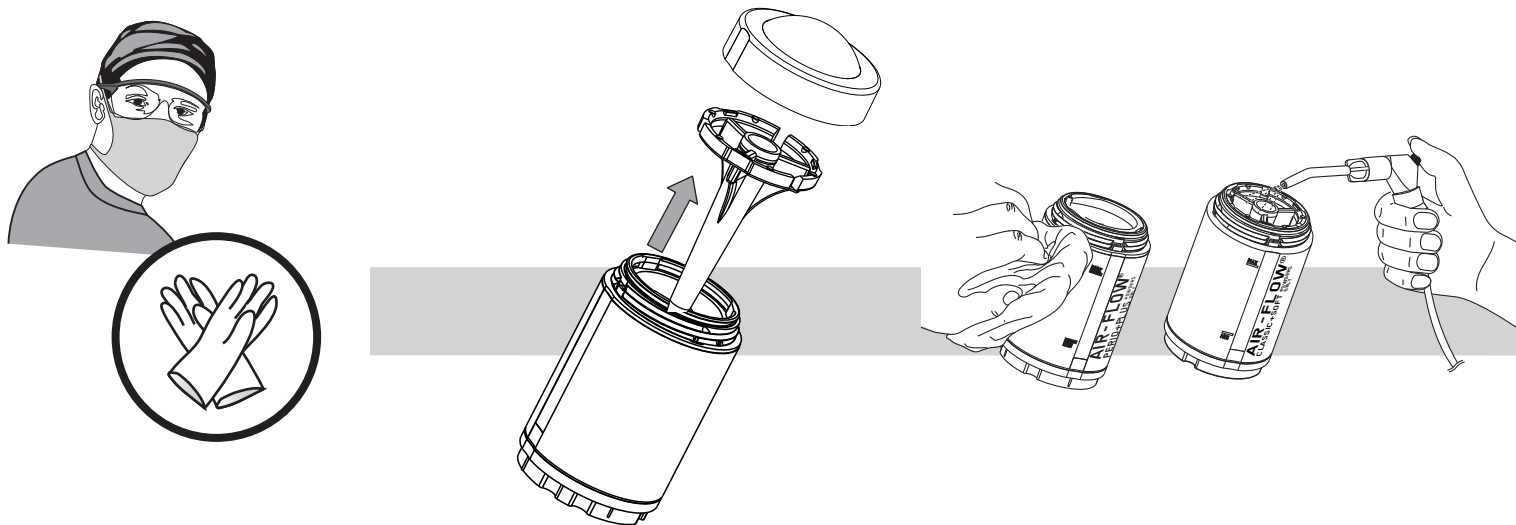
Cleaning the Powder Chamber


Unscrew the dome and pull out the powder diffuser.

Regularly clean this part.

- ⚠ The cap and the powder diffuser must be absolutely dry before reassembling.
- ⚠ Do not switch both powder diffusers during the reassembling.
- ⚠ Do not use needles to clean the powder chamber valve (risk of perforation)

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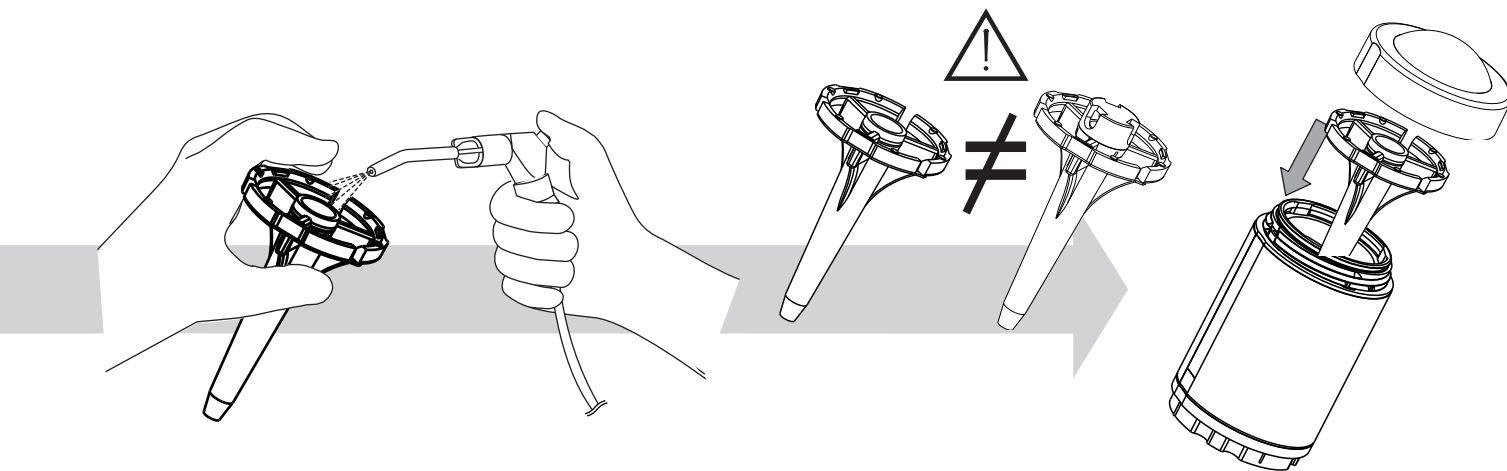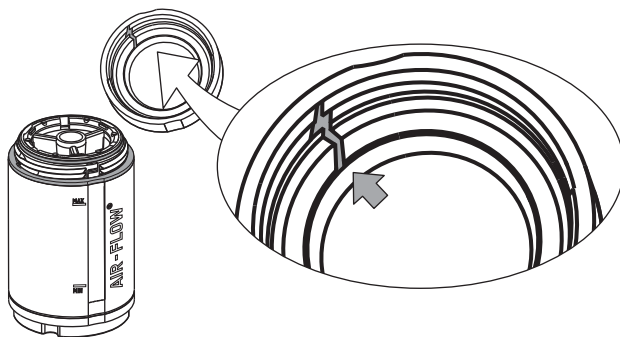
MAINTENANCE

Replacing the dome


 Check the condition of the threads on the powder chamber and cap. The powder chamber is pressurized during use. The condition of the powder chamber and cap (ring and dome) is an important safety factor.

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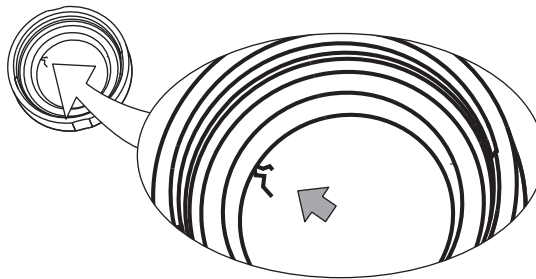
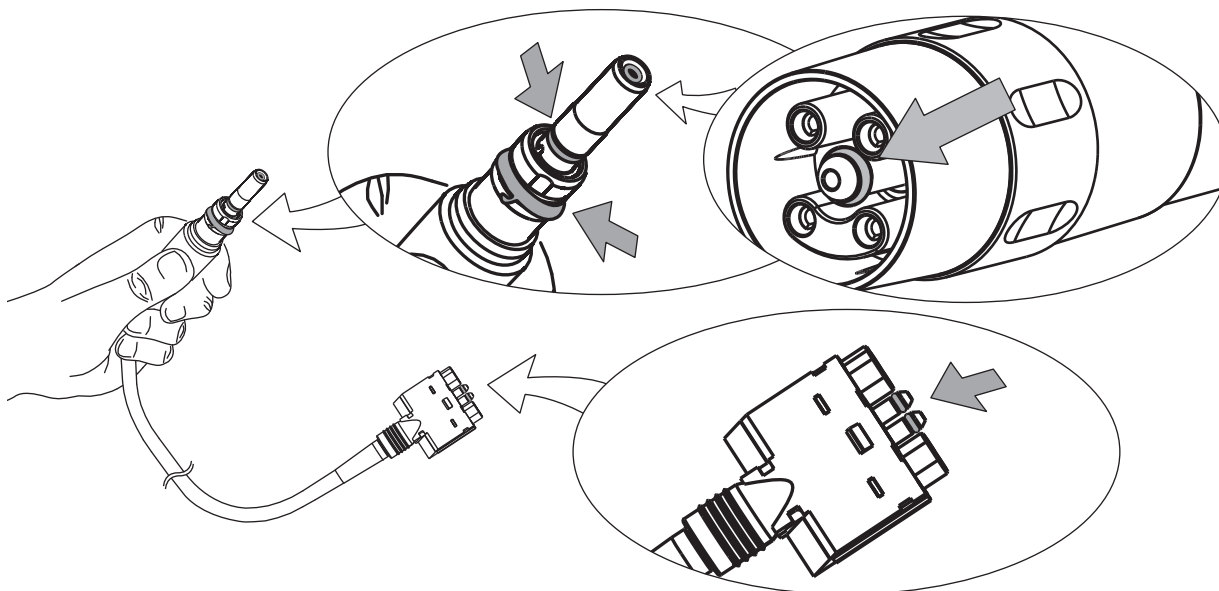
O-rings

-
-  If an O-ring is damaged or worn out, it must be replaced immediately.
 -  Please refer to the instructions provided with the maintenance set.
 -  Always use Hu-Friedy & EMS original parts. Using non- original instruments may damage the unit and the practitioner or patient may be injured.

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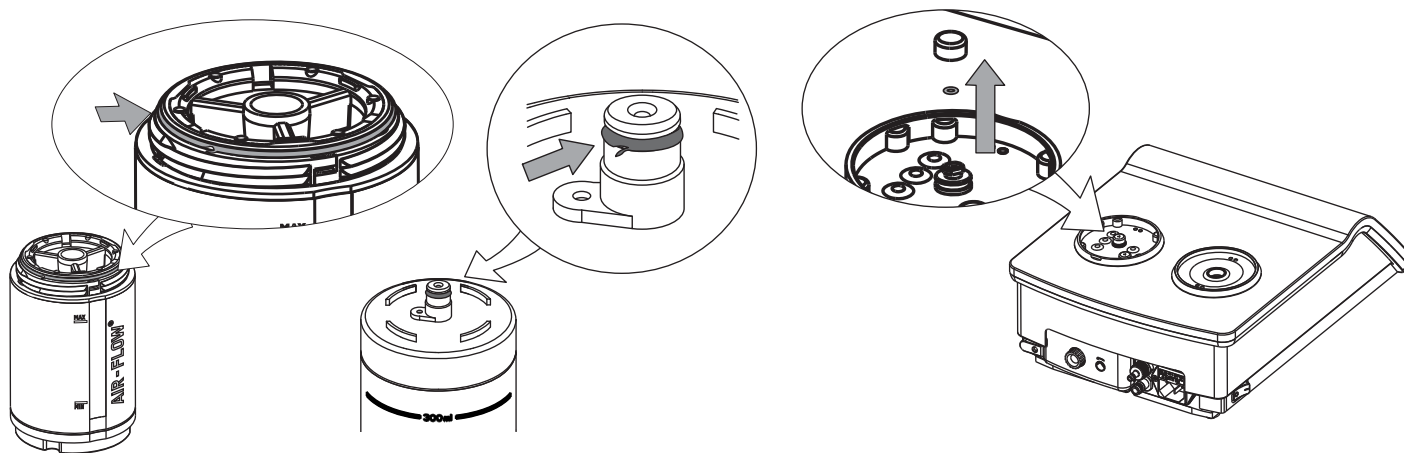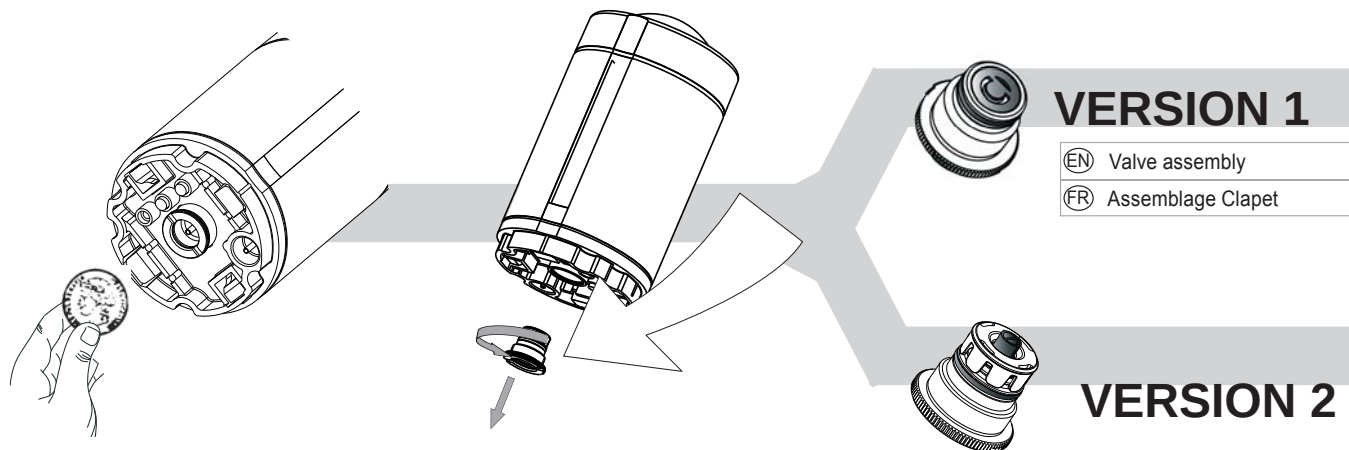
Valve assembly


⚠ If the valve is damaged or worn out, it must be replaced immediately.

To replace the powder chamber valve:

- Do not turn the powder chamber upside down.
- Unscrew the valve assembly under the powder chamber with a coin, unclip the cap and extract the valve.
- Place the new valve in the correct position on the support and screw the assembly under the powder chamber.

⚠ If an O-ring is damaged or worn out, it must be replaced immediately.

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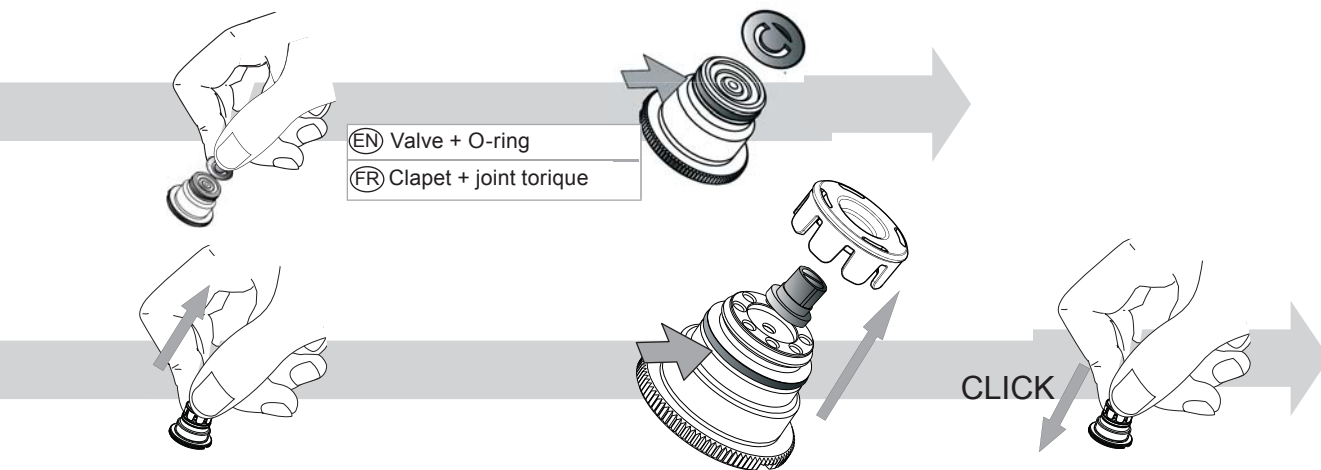
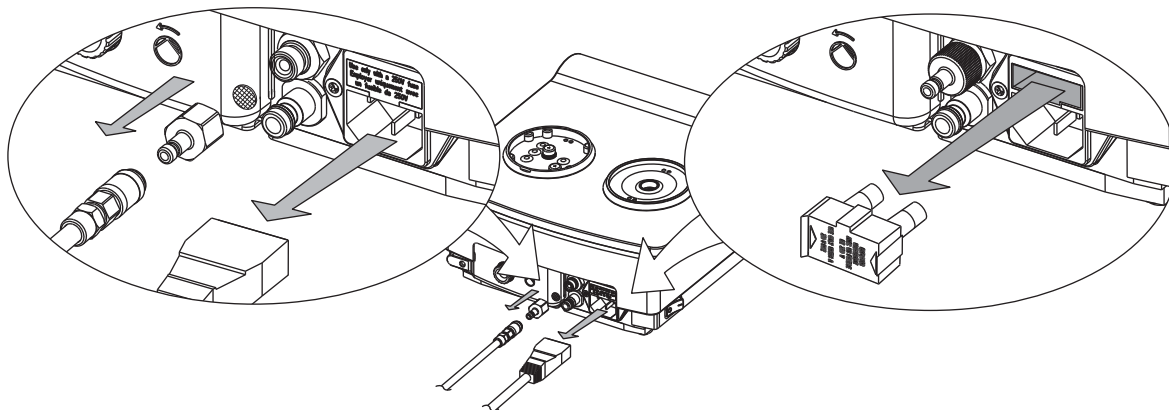
Replacing the water filter and the fuses

Regularly check the filter. Clean or change the filter if necessary.

 Only replace with the type of fuse indicated on the back of the unit.

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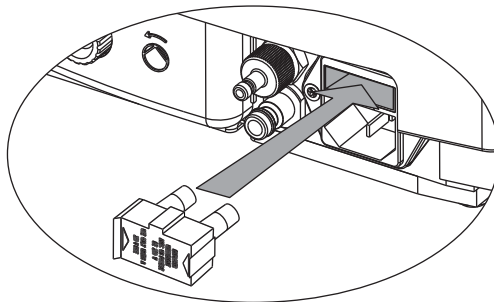
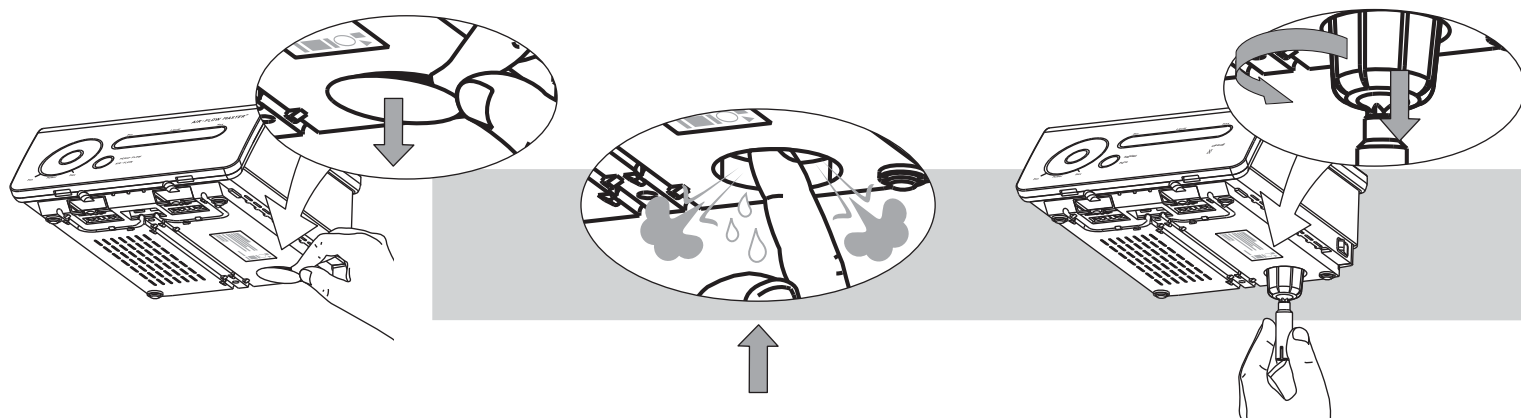
Air filter

Regularly check and purge the air filter.

The air filters are not reusable. Always install a new Hu-Friedy & EMS filter.

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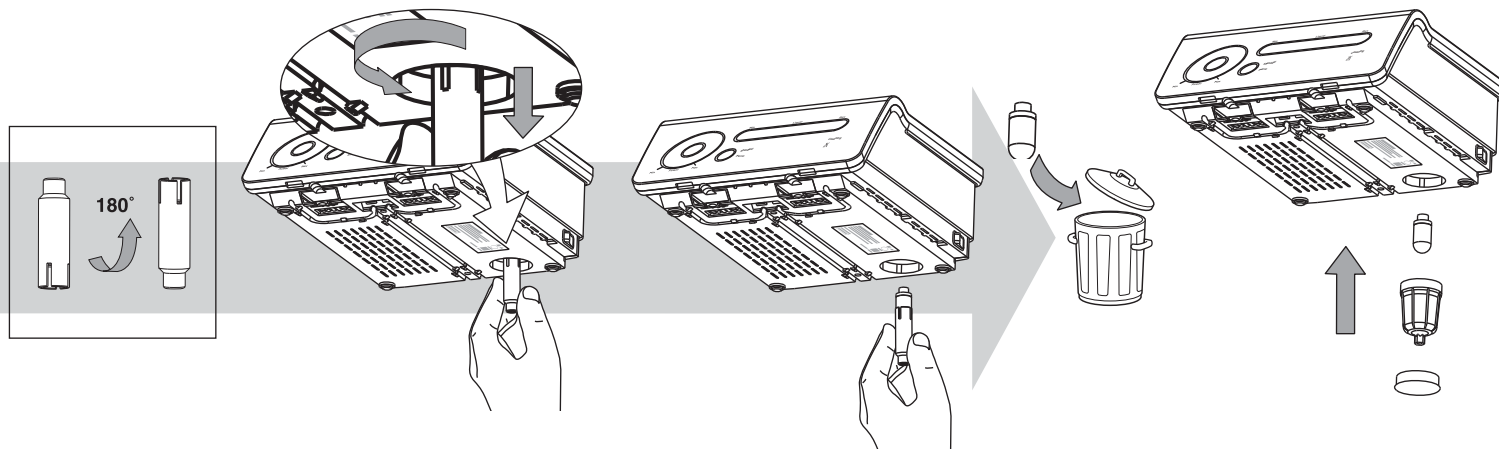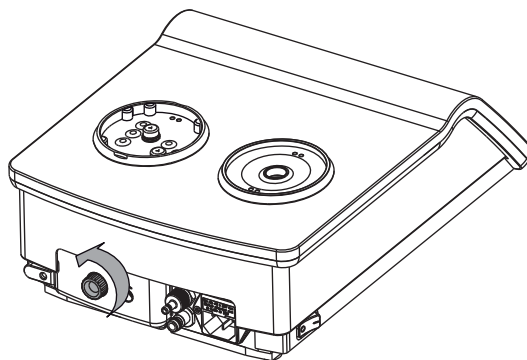
Replacing the pump of the unit

The pump of the unit is subject to wear. It is recommended to replace it every 18 months.

Please refer to the instructions provided with the spare pump.

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SAFETY PRECAUTIONS

Hu-Friedy & EMS and the distributor of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 **Contraindication:** ultrasonic oscillations may prevent cardiac pace makers and defibrillators from functioning properly. Therefore, we recommend that patients with a cardiac pace maker or a defibrillator should not be treated with this product.

 **Use for the intended purpose only.** Before using the product, make sure the operating instructions have been carefully studied. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged.

 **Risk of explosion:** do not use this product in the presence of inflammable anaesthetics or gases.

 **This product must be used only by trained and qualified personnel.**

 **Before each use, clean, disinfect and sterilize the different pieces and accessories of this product.** Please refer to the information provided in the operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 **Use only the powder recommended by Hu-Friedy & EMS for the subgingival application when the PERIO-FLOW handpiece is connected to the unit.** Any other powder could cause premature wear of the nozzle and would consequently increase the risk of emphysema and damage to the unenameled dentine.

 **The powders provided by Hu-Friedy & EMS are specially designed for use with the unit.** Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

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-  Never use an abrasive powder i.e. based on silica or alumina. This would damage the unit.
-  Always examine the product for damage before starting treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original Hu-Friedy & EMS spare parts and accessories only.
-  Close the powder chamber after use, risk of device obstruction
-  This product must only be repaired by an approved Hu-Friedy & EMS repair center.
-  This product has been investigated with regard to safety from electrical shock and fire hazard. CSA (Canadian Standard Association) has not investigated physiological effects.
-  Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the note "Electromagnetic compatibility" manual must be taken into account.

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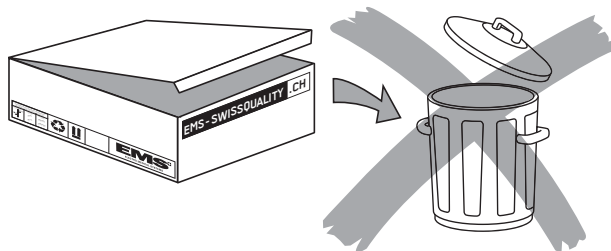
STORING THE PRODUCT

 Keep the original packaging until the product is to be disposed of permanently. It can be used for shipping or storing at any time.

If the unit is not going to be used for an extended period of time:

- Proceed as described in the chapter "Cleaning, disinfecting and sterilizing",
- Proceed to a "Cleaning" cycle without bottles to empty the circuits,
- Remove the screw of the water filter to empty the unit,
- Pack the product and all accessories in the original packaging.

 The powder should not be stored near acids or heat sources.

Please refer to the storage and transport conditions in the chapter "Technical data".

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PRODUCT DISPOSAL

The product must not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

Waste Electrical and Electronic Equipment belonging to customers located in the European Union may be shipped to Hu-Friedy & EMS for recycling in accordance to the WEEE regulations. The costs of recycling, exclusive of shipping fees, are covered by Hu-Friedy & EMS.

WARRANTY

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from Hu-Friedy & EMS or any authorized dealers. Please contact your customer service directly.

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HU-FRIEDY & EMS-SERVICE

Should your product need servicing and repairs, please send it to your dealer or to an authorized Hu-Friedy & EMS Repair Center.

Hu-Friedy & EMS decline responsibility for the safety of the device and declare the warranty null and void if service or repair is carried out by an unauthorized third party or if non-genuine spare parts are used.

It is best to ship the product in the original packaging. It protects the product against damage during shipment.

 Please clean, disinfect and sterilize the product and all accessories as described in the operating instructions before dispatching them.

When sending the product directly to an approved Hu-Friedy & EMS repair center, please include the name of your dealer. This simplifies processing.

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SYMBOLS

SYMBOLES

	Manufacturer's logo
	Distributor's logo
	Caution!
	Manufacturer
	Applied part, type BF
IP ..	Degree of protection against water permeability
	CE marking: refers to directive 93/42 EEC, including EN 60601-1 and EN 60601-1-2
	Disposal of old electrical & electronic equipment (applicable in the European Union and other European countries with separate collection systems)
	Read the operation instructions

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	Power switch "ON"
	Power switch "OFF"
	Input
	Output
	Footpedal connection
	CSA marking with "C" identification for products in conformance with Canadian standards and "US" for products in conformance with US standards
	Irrigation
	Without irrigation

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Max	Maximum setting
Min	Minimum setting
PURGE	Purge switch
Cleaning	"Cleaning" mode
	GOST R marking for products in conformance with Russian standards
Segurança  	DEKRA INMETRO identification for products in conformance with Brazilian electrical standards

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TECHNICAL DATA

DESCRIPTION

Manufacturer	EMS ELECTRO MEDICAL SYSTEMS SA, CH-1260 Nyon, Switzerland
Model	AIR-FLOW MASTER PIEZON®
Classification EN 60601-1	- Class I - Applied part, Type BF - IP 20 unit - IP X1, footpedal
Classification 93/42/EEC	Class IIa
Operating mode with irrigation: without irrigation:	Continuous operation Duty cycle 10% during 10 min. maximum
Supply voltage	100-240 VAC 50-60Hz
Power consumption	max. 110 VA
Fuses	T 2.5 A 250 VAC (Ø5 X 20 mm, (time lag, low breaking capacity)
Water supply	1 to 5 Bar (100 to 500 kPa)
Compressed air supply	5.5 to 7.5 Bar (550 to 750 kPa)
Ultrasound specifications Maximum output: Frequency range:	12 Watts 24-32 kHz
Weight (approx.)	4.5 kg
Dimensions in mm (HxWxD)	225 x 280 x 295
Operating conditions	+10 °C to +40 °C 30% to 75% relative humidity Altitude max. 3000 m
Storage and transport conditions	-10 °C to +40 °C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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ELECTROMAGNETIC COMPATIBILITY

 This product needs special precautions regarding electromagnetic compatibility and needs to be installed and put into service according to the note "Electromagnetic compatibility".

 Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the note "Electromagnetic compatibility" must be taken into account.

 This product should not be used adjacent to or stacked with another unit. If adjacent or stacked use is necessary, this product should be observed to verify normal operation in the configuration in which it will be used.

 The use of accessories and cables other than those specified or sold by Hu-Friedy & EMS as replacement parts may result in increased emissions or decreased immunity of this product.

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TROUBLESHOOTING

Symptom	Solution
The unit does not start or stop	Check the electrical connections and power supply at the mains Check the fuses on back panel Check main switch function: at least one LED must be visible on adjustments and working mode areas The unit is in "standby" mode, tap on any sensitive zone to exit standby Send the unit to an authorized Hu-Friedy & EMS Repair Center
The foot pedal does not activate the unit	Check that the foot pedal is properly connected Replace the foot pedal Send the unit to an authorized Hu-Friedy & EMS Repair Center
The unit is still activated when the foot pedal is released	The foot pedal is active, check for jam at foot pedal The switch in the pedal is blocked. Turn the unit off at the power switch Replace the foot pedal Send the unit to an authorized Hu-Friedy & EMS Repair Center
User interface displays an error: there is one acoustic signal and all LEDs are blinking	Powder chamber pressure is too low or too high The air or liquid regulator is switched off Foot pedal and purge commands are not recognized by system. Check connections
The handpiece does not remain on the magnetic support	Install the magnetic support on a metallic part of the unit The support is demagnetized Replace magnetic support
The handpiece does not hold on the handpiece cord	Check for presence of the O-ring seal and replace if necessary Replace the handpiece Replace the handpiece cord Send the handpiece and the handpiece cord to an authorized Hu-Friedy & EMS Repair Center
No ultrasound oscillations or insufficient power	Make sure that the Piezon instrument is correctly tightened Check the wear of the instrument and replace it if necessary Check the connections between the unit, the handpiece cord and the handpiece Dry the connectors of the handpiece cord and of the handpiece Check the connection of the foot pedal Check the settings and the working mode Check that when the handpiece is lifted from its support, the control lamps of the bottle holder are turned on; if it is not the case, send the handpiece to an approved Hu-Friedy & EMS repair center Replace the handpiece cord and/or the handpiece Send the unit to an approved Hu-Friedy & EMS repair center
Insufficient lighting of the handpiece	Replace the light guide
Cleaning Mode Blinking LED	Make a cleaning of the unit and follow the instructions under the "cleaning and disinfecting circuit" section

No water jet coming from the Piezon handpiece	Check the water supply and water filter Make sure the liquid flow rate is set at maximum level Check the connections between the unit, the cord and the handpiece Remove the instrument and activate the foot pedal for a few minutes until the liquid flows to the handpiece Check for presence of water at the cord outlet port Send the handpiece to an authorized Hu-Friedy & EMS Repair Center
No powder/air jet coming from the AIR-FLOW or PERIO-FLOW handpiece	Check the compressed air supply and air filter Check for presence of powder/air at cord outlet port Eliminate residual powder present in the channels using the cleaning needles provided Check the quantity of powder in your unit Send the system to an authorized Hu-Friedy & EMS Repair Center
Water leakage between the handpiece cord and the handpiece	Check the condition of the handpiece connecting O-rings Send the system and the handpiece cord to an authorized Hu-Friedy & EMS Repair Center
Water leakage between the PERIO-FLOW handpiece and the nozzle	Verify the position of the nozzle and its position on the handpiece Verify that no dirt remains between the nozzle and the handpiece Verify that nothing affects the alignment or prevents fixing the nozzle. Replace the nozzle Replace the handpiece Send the system and the handpiece cord to an certified Hu-Friedy & EMS Repair Center
Air and powder leakage on the main unit	Verify the positioning of the bottles Check the O-rings under the silicone caps of the powder chamber holders Verify the cap of the powder chamber and the O-ring positioning Verify the check valve and the O-ring under the powder chamber Remove all powder dust between the bottle holder and the powder chamber Send the unit to an authorized Hu-Friedy & EMS Repair Center
Air and powder is leaking at powder chamber	Check the seal condition and the cleanliness of the thread of the powder chamber cap Replace the seals if necessary Clean the sealing surfaces carefully Replace the powder chambers Send the unit to an authorized Hu-Friedy & EMS Repair Center
Handpiece AIR-FLOW or PERIO-FLOW efficiency decreases	Verify the quantity of powder in your unit Eliminate residual powder present in the channels using a cleaning needle Send the system and the handpiece cord to an certified Hu-Friedy & EMS Repair Center
Leakage on the unit water supply	Check the water supply connection and inlet pressure Send the unit to an authorized Hu-Friedy & EMS Repair Center

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Hu-Friedy Mfg. Co., LLC
EMS Electro Medical Systems S.A.