



# Clear Collection™ Instrument Usage Guide

## Clockwise and Counter Clockwise Wedges

The Clockwise and The Counter Clockwise Wedge instruments are used to produce indents into the facial and the lingual plastic for the rotational movement of a specific tooth. From an occlusal view, evaluate the tooth you wish to turn or rotate: Clockwise or Counter Clockwise (i.e. anti-clockwise). Select the appropriate wedge instrument to produce the desired rotation. Position it at the line angle near the embrasure, and indent the labial and the lingual to create a rotational couple for that tooth.

### Clockwise Wedge 678-808

### Counter Clockwise Wedge 678-807

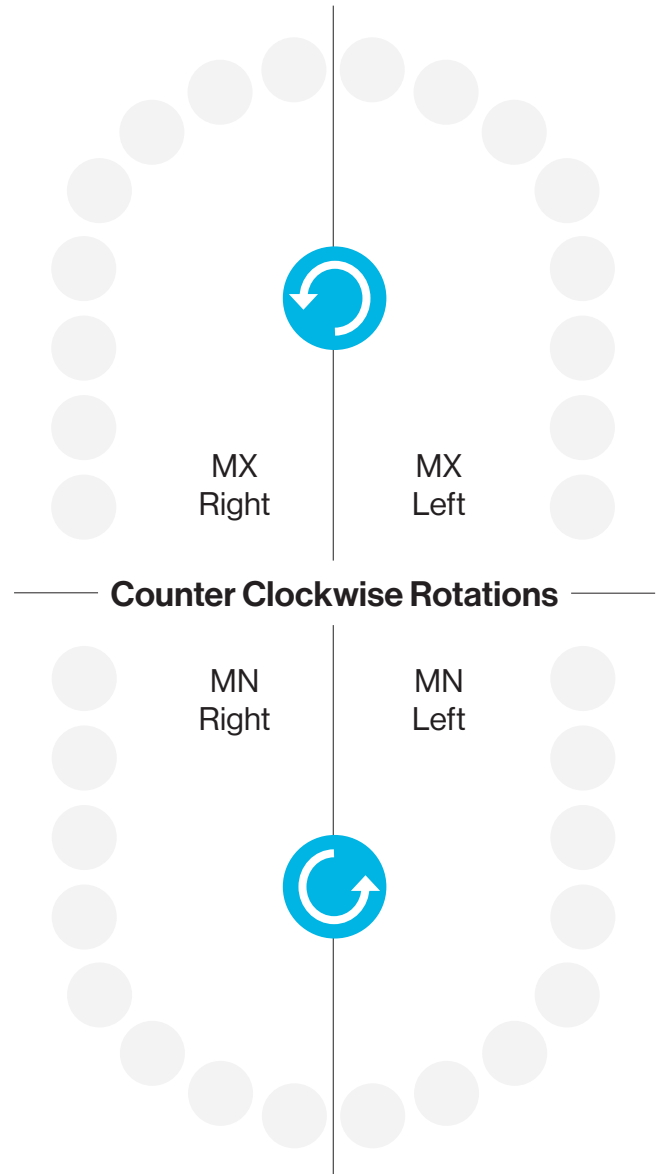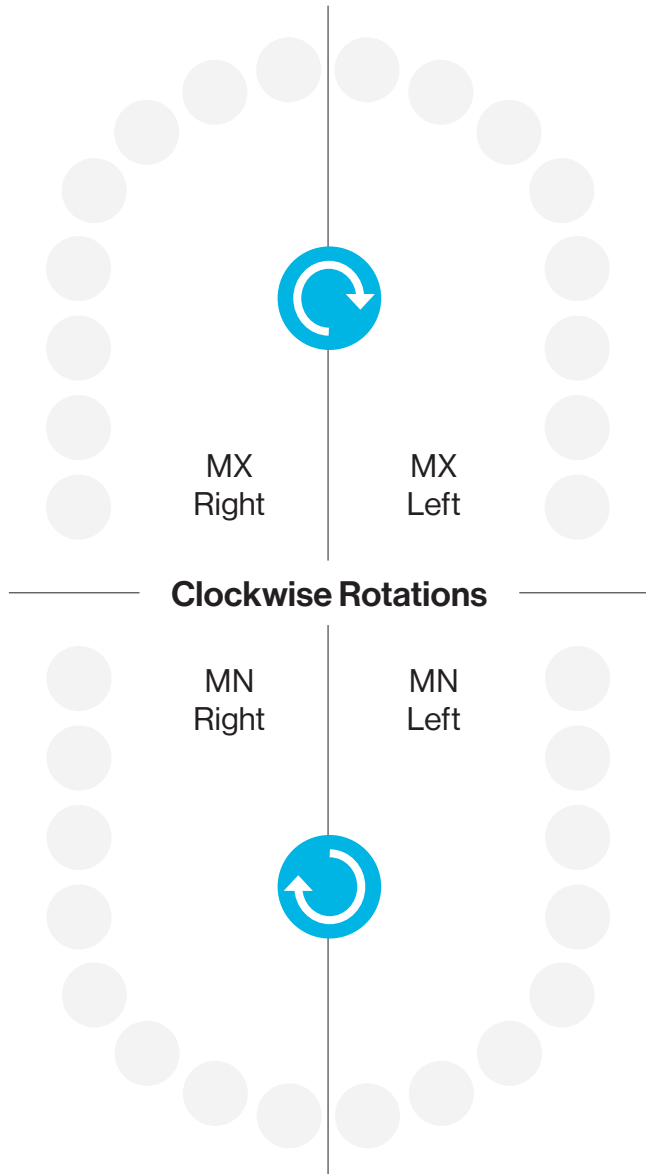

For more information visit [Hu-Friedy.com/ClearCollection](https://www.hufriedy.com/ClearCollection)

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted.

©2021 Hu-Friedy Mfg. Co., LLC. All rights reserved. HFL-573/0321