

# SKOOLTOOLZ

## BEAUTIFUL BIOMIMETIC RESTORATIONS

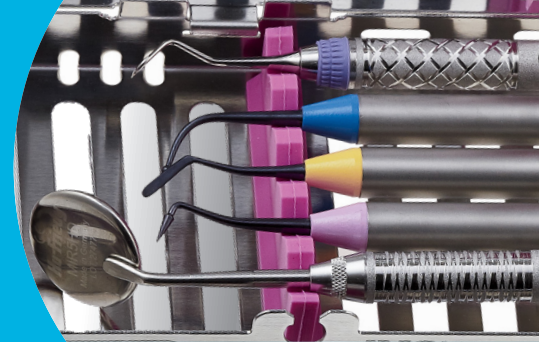


FIGURE 1



### PREOPERATIVE VIEW

Defective amalgam and composite patch restorations which need to be replaced.

FIGURE 2



Amalgams removed and marginal ridge composite base layer shaped in the distal prep using **TNTAM2**.

FIGURE 3



Brown tint delicately applied with the ultra-pointed fine end of **TNTAM1**.

FIGURE 4



Final layers of composite placed and sculpted with the conical end of **TNTAM1**.

FIGURE 5



Access for matrix burnishing and initial layers of packable composite made simple using the extended shank and burnisher end of **TNTAM3**.

FIGURE 6



Composite formed and finessed with the thin paddle of **TNTAM2**.

FIGURE 7



### POSTOPERATIVE VIEW


Beautifully and biomimetically-completed restorations.

FIGURE 8



# SKOOLTOOLZ KIT

## TNTAMKIT

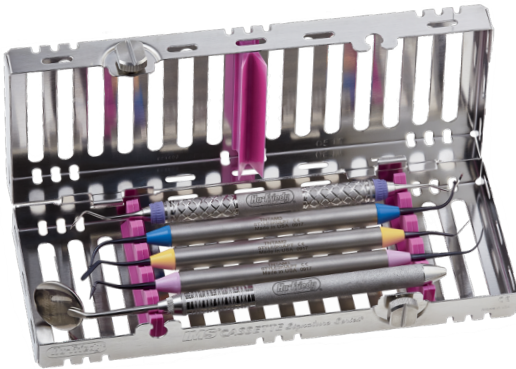

The SkoolToolz Kit (TNTAMKIT) contains instruments in 5-instrument cassette.

**Dr. Clarence Tam** is originally from Toronto, Canada, where she completed her Doctor of Dental Surgery and General Practice Residency at the University of Western Ontario and the University of Toronto, respectively. Clarence's practice is limited to cosmetic and restorative dentistry. She is well-published to both the local and international dental press, writing articles, reviewing and developing prototype products and techniques in clinical dentistry. She frequently and continually lectures internationally.

Clarence is the Chairperson and Director of the New Zealand Academy of Cosmetic Dentistry. She is currently one of two individuals in Australasia to hold Board-Certified Accredited Member Status with the American Academy of Cosmetic Dentistry. Clarence is an Opinion Leader for multinational dental companies Kuraray Noritake, J Morita Corp, Henry Schein NZ, Ivoclar Vivadent, DentsplySirona, 3M ESPE, Kerr, GC Australasia, SDI, Coltene, Triodent and is the only Voco Fellow in Australia and New Zealand. She holds Fellowship status with the International Academy for DentoFacial Esthetics. She is a passionate and approachable individual, and is committed to having an interactive approach with patients in all of her cases to maximize predictability.

SkoolToolz Kit	TNTAMKIT
Cone Socket Mirror Handle, Satin Steel	<b>MH6</b>
#5 HD Mirror	<b>MIR5HD</b>
SkoolToolz 1, Posterior Occlusal Shaper	<b>TNTAM1</b>
SkoolToolz 2, Universal Thin PFI	<b>TNTAM2</b>
SkoolToolz 3, Curved Ball Burnisher	<b>TNTAM3</b>
Posterior Sickle Scaler, EverEdge 2.0	<b>S204S9E2</b>



SkoolToolz 1,  
Posterior Occlusal  
Shaper  
**TNTAM1**



SkoolToolz 2,  
Universal Composite  
Instrument  
**TNTAM2**



SkoolToolz 3,  
Curved Ball  
Burnisher  
**TNTAM3**

Learn more about SkoolToolz and Dr. Tam at [Hu-Friedy.com/Community/Clarence-Tam](https://www.hu-friedy.com/Community/Clarence-Tam)

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted. Marks not registered in all jurisdictions.  
©2021 Hu-Friedy Mfg. Co., LLC. All rights reserved. HFL-046/0221