

Why do I need to maintain and clean my waterlines?

The CDC recommends that dental unit waterlines be tested on a regular basis to maintain its established standard of less than 500 colony-forming units per milliliter.

If dental unit waterlines are not maintained on a regular basis, bacteria will develop on the inner surfaces of the tubing, blocks and other system components.

Simply maintaining the waterlines is not enough; it is a two part process of cleaning and maintenance. Over time, bacteria can overwhelm the waterline environment and begin to create microbial deposits. Using a daily irrigant, like VistaClean, and cleaning the waterlines regularly with an EPA approved antimicrobial, like VistaTab, will clean and control microbial contaminants to provide improved water quality while helping to protect patients, staff, and equipment.

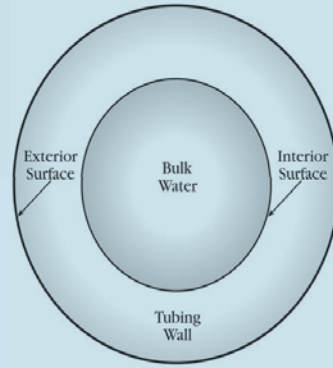


Figure 1. Clean dental waterline tubing. No biofilms or free-floating organisms present.

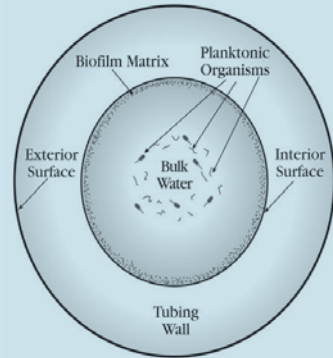


Figure 2. Contaminated dental waterline. Biofilm on interior surface with abundant free-floating organisms.

Team Vista™ Dental Waterline Cleaner by Hu-Friedy



Team Vista Dental Waterline Cleaner IMS-1450

This complete system contains:
1 – One Ounce Bottle of VistaClean Irrigant Solution
15 – VistaTab Cleaner Tablets
(One-year supply per operator)



VistaTab Dental Waterline Cleaner Tablets IMS-1451

This box contains: 15 Tablets (1.7 grams)
Net Weight 22.5 grams (0.90 oz.)



Vista Clean Irrigant Solution Concentrate IMS-1452

One Fluid Ounce Bottle (29.57 ml)

Conversions:
500 ml = 16.91 oz. 750 ml = 25.36 oz.
1.7 grams = 0.06 oz. 28.35 grams = 1 oz.

VistaClean is a Class I Medical Device registered with the FDA.
VistaTab is EPA registered in all 50 states.



Hu-Friedy Mfg. Co., Inc.
3232 North Rockwell Street
Chicago, IL 60618

www.hu-friedy.com
1-800-HU-FRIEDY

Take Waterline Treatment
TO THE NEXT LEVEL



Introducing Team Vista™ by Hu-Friedy

The complete waterline security system

Team Vista provides a more effective and economical way to keep waterlines safer and cleaner. This easy-to-use system is designed for the quick cleaning and control of microbial contamination in dental unit waterlines. It's the only system that provides both an organic irrigant and registered antimicrobial cleaner for enhanced patient care.

- For use in independent water bottle systems
- Provides continuous cleaning of waterlines for up to one week
- Non-corrosive - gentle and safe on equipment
- One year supply per operator



VistaTab™

Dental Waterline Cleaner Tablets

VistaTab is an antimicrobial tablet used for the cleaning and control of microbial contaminants in dental unit waterlines. When placed in water, the solid tablet quickly dissolves to create a non-corrosive solution that is seven times more effective than bleach.

"It is important to properly treat your dental unit waterlines as a component of a total infection control program. Team Vista is an easy, effective system to both periodically clean and maintain dental unit waterlines."

—John Molinari, PhD

VistaClean™

Irrigant Solution Concentrate

VistaClean is a daily irrigant solution derived from organic citrus botanicals and is safe for use with patients. The VistaClean solution helps protect waterlines and equipment from deposits and scale. The solution works to emulsify organic and inorganic contaminants that can build up in waterlines. VistaClean leaves no taste or odor and is completely non-toxic, environmentally friendly, non-corrosive, and has no negative effect on bond strength.

